

Recommended Computer Requirements for BioGraph Infiniti System version 6.12

Processor	Quad-core CPU or better, Intel i5 or equivalent. ARM64*
Operating Systems	Windows 11
Memory (RAM)	Minimum 8 GB
Hard Disk space (Primary C drive)	500 GB or more, to save video and sessions data on the primary drive
Video / Graphics card	At least 4 GB of dedicated memory with 1280x1024 resolution and higher. For recent generations of Intel 64-bit systems with a minimum of 8 GB RAM.
Monitor	Resolution of 1280x1024 and higher; option for two monitors

Other necessary components:

- Adequate number of USB ports depending on the intended system usage (1-4 for ProComp or FlexComp Infiniti encoders and accessories)
- 64-bit compatible sound card & speakers.
- Microsoft® Office with Word® and Excel® (for report generation and printing).
- High-speed Internet access (for streaming videos, updating software, and receiving technical support)

Optional components, depending on intended system usage:

- DVD-ROM drive for playing movies from a DVD; note: many non-DVD video options are available.
- Compact Flash Reader (for use with compact flash card only).
- Up to two webcams with built-in microphone (if recording video). Frame rate 30fps, Microsoft® recommended.
- Bluetooth receiver/radio (Bluetooth version 5 or higher), if using TPS sensor and other Bluetooth-enabled devices.

PLEASE NOTE:

- Tele-Infiniti CF (T9600) version 3.2.1 or higher is compatible with 64-bit Windows® operating systems.
- Tele-Infiniti CF (T9600) is not supported on ARM64 processor. *
- AND blood pressure monitoring unit (UA767PC) is not supported on ARM64 processor. *
- ARM64 processor computer requires the latest version of TT-USB. *
- Low speed CPUs (1.8 GHz) may be sluggish when running screens with a large number of instruments per screen.
- If using 2 webcams for video recording, it is strongly suggested to use recommended PC requirements.
- If using TT-AV Sync functionality, please refer to possibly additional system requirements from appropriate suite manual.
- BioGraph Infiniti is not designed for use in a networked environment.

The following apply to BioGraph Infiniti 6.12 and its application suites:

- We recommend 64-bit version of Windows OS to be able to have 8+ GB of RAM.
- BioGraph Infiniti software and application suites are installed for **All Users** on a per system basis.

- The BioGraph Infiniti system must be installed on the PC where the database is/will be located.