



Reference Manual



Thought Technology Ltd.

8205 Montreal/ Toronto Blvd. Suite 223, Montreal West, QC
H4X 1N1 Canada
Tel: +1 (800) 361-3651 · +1 (514) 489-8251 Fax: +1 (514) 489-8255
E-mail: mail@thoughttechnology.com
Webpage: <http://www.thoughttechnology.com>

For use with  version 6.1 or later



CLASSIFICATION



- Type BF Equipment
- Internally powered equipment
- Continuous operation



- Read Instruction Manual

CAUTION

RxOnly

- US Federal Law restricts this device to sale by or on order of licensed health care practitioners.

WARNING

- Do not operate active sensor within 10 feet (3m) of an operating cellular phone, similar radio transmitting device, other powerful radio interference producing sources such as arc welders, radio thermal treatment equipment, x-ray machines or any other equipment that produces electrical sparks.
- All encoders are totally isolated from line (110 or 220VAC) power due to battery operation and fiber optic connections to computers. However, many hospitals and the FDA require that computers, printers and any other equipment used with medical devices be electrically isolated from line voltage to UL or CSA medical safety standards.
- Do not connect inputs or outputs of the encoder or sensors to line powered devices, except through the fiber optic cable.
- The PC used with the encoder must be placed outside the patient/client environment (more than 3 meters or 10 feet) or the PC must comply with EN60601-1.1 (system safety).
- After use, the disposable electrodes may be a potential biohazard. Handle and, when applicable, dispose of these materials in accordance with accepted medical practice and any applicable local, state and federal laws and regulations.
- To diminish the risk of spreading communicable diseases, always use good hygiene practices with reusable EMG electrodes, particularly if abrasive substances are used. In all cases, refer to your facility's infection control procedure.000
- Do not use in the presence of a flammable anesthetic mixture with air or with Oxygen or Nitrous Oxide.
- Not to be immersed in water.
- Take care in arranging patient and sensor cables to avoid risk of patient entanglement or strangulation.
- The operator is responsible for ensuring the safety of any devices controlled or triggered by Infiniti equipment or software, or by any software or hardware receiving data from Infiniti equipment. Infiniti equipment must not be configured or connected in such a way that failure in its data acquisition, processing or control functions can trigger patient feedback stimulus that poses an unacceptable level of risk.
- Use of any equipment in a biofeedback context should be immediately terminated upon any sign of treatment-related distress or discomfort.
- Not to be connected to a patient undergoing MRI, Electro surgery or defibrillation.

ATTENTION

- To prevent static discharge from damaging the sensor and/or encoders, use anti-static mats or sprays in your working area. A humidifier may also be used to help prevent static environments by conditioning hot, dry air.
- Not for diagnostic purposes. Not defibrillator proof. Not for critical patient monitoring.
- To prevent voiding warranty by breaking connector pins, carefully align white guiding dot on sensor plug with slot on sensor input.
- Sharp bends or winding the fiber optic cable in a loop smaller than 4 inches (10cm) may destroy the cable.
- A fiber optic cable not fully pushed into its receptacle may cause the unit not to operate; make sure that both ends of the cable are fully inserted into their receptive jacks and the nuts are tightened firmly.
- Make sure to remove electrodes from sensor snaps immediately after use.
- Apply conductive gel only to electrodes; never put gel directly on sensor snaps.

- Always use electrodes between the subject and the sensor.
- Do not plug third party sensors directly into instrument inputs. Plug only Thought Technology active sensor cable connectors into instrument inputs. All EMG electrodes and third party sensors must be connected to active sensors, either directly or through an adapter.
- Remove batteries when the device is not being used for extended period of time. Please dispose of battery following national regulations.

INTENDED PURPOSE

- Biofeedback, relaxation and muscle re-education purposes.

CONTRAINDICATIONS

- None.

NOTE

- No preventative inspections required; maintenance must be performed by qualified personnel.
- The supplier will make available, upon request, circuit diagrams, component parts lists and description or other information required for the repair of product by qualified personnel.
- If a fiber optic or patient cable is damaged or breaks, please replace it.
- Due to the essential performance and intended use of the device, testing for immunity to electromagnetic disturbances was not required and was not performed. The device may be susceptible at levels below IEC60601-1-2 immunity test levels.
- The operator must be familiar with typical characteristics of signals acquired by this equipment, and be able to detect anomalies in the acquired signal that could interfere with treatment effectiveness. Depending on the importance of signal integrity, it may be advisable to continuously monitor the raw signals, in time and/or frequency domain, while the device is being used for biofeedback or other purposes. If anomalies are observed on acquired signals, and if you suspect a problem with electromagnetic interference, contact Thought Technology for a technical note on identification and remediation.

MAINTENANCE AND CALIBRATION

- Wipe encoder with a clean cloth.
- Factory testing and calibration ensure equipment accuracy and frequency response. The user may invoke a self-calibration function that will recalibrate certain device parameters (see section in hardware manual). Contact Thought Technology for factory recalibration if necessary.

STORAGE

- Store in its original case.
- Temperature -23 to +60C
- Humidity (Non-condensing) 10% to 90%
- Atmospheric Pressure 70 to 106 Kpa

TRANSPORTATION

- Transport in its original case.
- Temperature -23 to +60C
- Humidity (Non-condensing) 10% to 90%
- Atmospheric Pressure 70 to 106 Kpa

Guidance and manufacturer's declaration – electromagnetic emissions

The Infiniti system is intended for use in the electromagnetic environment specified below. The customer or the user of the Infiniti system should assure that it is used in such an environment.

Emissions test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The Infiniti system uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment. The Infiniti system is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
RF emissions CISPR 11	Class B	
Harmonic emissions IEC 61000-3-2	Not applicable	
Voltage fluctuations/flicker emissions IEC 61000-3-3	Not applicable	

Manual No. SA7972 ver. 6.1 (April 2016)
© copyright Thought Technology Ltd. 2003-2016

TABLE OF CONTENTS

INTRODUCTION	1
REQUIREMENTS	1
FIRST TIME RUN	2
STARTING THE PROGRAM	2
VERIFYING SETTINGS	2
Temperature and Force Units	3
Notch Filter.....	3
Text Report Options	3
Confidential Setting	3
ENTERING KEY CODES	3
HARDWARE SET-UP	5
THE ENCODER	5
Connecting TT-USB and the encoder	5
Battery Placement.....	6
Caring For the Fiber Optic Cable	6
Electromagnetic Electrical Interference.....	6
SENSORS AND ACCESSORIES	6
Electromagnetic interferences.....	6
Electrostatic discharges	7
Connecting the sensors	7
Correct orientation of sensors	7
Sensors, Electrodes, and Accessories Used with the System	8
MyoScan/ MyoScan-Z EMG Sensor (P/N: SA9503M/SA9503Z).....	8
MyoScan-Pro EMG (P/N: SA9401M-60 or SA9401M-50).....	8
InclinoTrac/Dual-InclinoTrac (P/N: T7650/T7655)	8
ForceTrac (P/N: T7640)	8
HR/BVP Sensor (P/N: SA9308M)	9
EKG-Z Sensor (P/N: SA9306M).....	9
Respiration Sensor (P/N: SA9311M).....	9
Skin Temperature Sensor (P/N: SA9310M)	10
Skin Conductance Sensor (P/N: SA9309M).....	10
EKG Wrist Straps (P/N: SA9325).....	10
Disposable EKG/EMG electrodes	10
Static EMG electrodes (P/N: SA9327)	10
Foot Pedal (P/N: SA7551).....	11
Tele-Infiniti CF™ (P/N: SA9600) - Optional.....	11
SOFTWARE OVERVIEW	12
RECORDING SESSIONS	12
Start the Program	12
Starting a session from a Favorite.....	12
Starting a Script Session.....	13
Starting an Open Display Session.....	14

Sensor Connections	15
STOPPING A SESSION	16
REPLAYING SESSIONS	17
REVIEWING SESSIONS	18
Review	18
Rejecting Artifacts	19
Calculating Statistics	19
Setting In/Out Markers	20
Generating Session Reports	20
TOOL BAR OPTIONS	22
Toggling Between Screens	22
Adjusting Scales and Threshold.....	22
USING A WEB CAMERA	23
USING A DVD	23
SENSOR CALIBRATION AND ZEROING	23
IMPEDANCE CHECK	25
GENERAL SEMG ASSESSMENT	26
OVERVIEW	26
CLINICAL GUIDE REFERENCE	26
GENERAL PROCEDURE	26
Starting the Session	26
Measuring	27
Reviewing the session	27
Generating a report	27
G.SEMG01 – BASELINE ASSESSMENT	27
G.SEMG02 – MAXIMAL FORCE ASSESSMENT	28
G.SEMG03 – ENDURANCE ASSESSMENT	28
G.SEMG04 – FAST FLICK ASSESSMENT	29
G.SEMG05 – GLOBAL SEMG ASSESSMENT 1 MUSCLE	30
SEMG ASSESSMENT – GENERAL PURPOSE (G.SEMG06 and up)	30
G.SEMG06 - SEMG Assessment with 2 EMG	30
G.SEMG07 - SEMG Assessment with 4 EMG (FlexComp Infiniti only).....	31
G.SEMG08 - SEMG Assessment with 6 EMG (FlexComp Infiniti only).....	32
G.SEMG09 - SEMG Assessment with 10 EMG (FlexComp Infiniti only).....	32
STATIC SEMG ASSESSMENT	33
OVERVIEW	33
CLINICAL GUIDE REFERENCE	33
PROCEDURE	33
Starting the Session	33
Measuring	34
Generating a report	35
DYNAMIC SEMG ASSESSMENT	37
OVERVIEW	37
CLINICAL GUIDE REFERENCE	37
GENERAL PROCEDURE	37
Starting the Session	37

Measuring	38
Reviewing the session	38
Generating a report	40
PROCEDURE FOR LUMBAR SPINE ASSESSMENT	40
Starting the Session	40
Measuring	41
Reviewing the session	42
Generating a report	43
PROCEDURE FOR CERVICAL SPINE ASSESSMENT	44
Starting the Session	44
Measuring	45
Reviewing the session	46
Generating a report	47
PROCEDURE FOR ANTERIOR KNEE ASSESSMENT	49
Starting the Session	49
Measuring	50
Reviewing the session	51
Generating a report	51
MUSCLE FATIGUE MONITORING	53
OVERVIEW	53
CLINICAL GUIDE REFERENCE	53
GENERAL PROCEDURE	53
Starting the Session	53
Measuring	54
SEMG BIOFEEDBACK	55
OVERVIEW	55
CLINICAL GUIDE REFERENCE	55
GENERAL PROCEDURE	55
Starting the Session	55
CATEGORY: PRO/FLEX SUITE – SEMG BIOFEEDBACK – RELAXATION	56
CATEGORY: PRO/FLEX SUITE – SEMG BIOFEEDBACK – STRENGTHENING	59
CATEGORY: PRO/FLEX SUITE – SEMG BIOFEEDBACK – CONTROL	62
CATEGORY: PRO/FLEX SUITE – SEMG BIOFEEDBACK – EQUILIBRATION	64
CATEGORY: PRO/FLEX SUITE – SEMG BIOFEEDBACK – TRAINING	68
UNSTABLE SHOULDER	70
MOTOR FUNCTION RESTORATION	71
RESPIRATION TRAINING	73
OVERVIEW	73
CLINICAL GUIDE REFERENCE	73
PROCEDURE	73
HEART RATE MONITORING AND HRV	76
OVERVIEW	76
CLINICAL GUIDE REFERENCE	76
PROCEDURE	76

SKIN CONDUCTANCE AND TEMPERATURE BIOFEEDBACK	79
OVERVIEW	79
CLINICAL GUIDE REFERENCE	79
PROCEDURE	79
STATIC RANGE OF MOTION ASSESSMENT	82
OVERVIEW	82
CLINICAL GUIDE REFERENCE	82
GENERAL PROCEDURE	82
Preparing the instrumentation	82
Measuring	84
Generating a report	84
DYNAMIC RANGE OF MOTION ASSESSMENT	87
OVERVIEW	87
CLINICAL GUIDE REFERENCE	87
GENERAL PROCEDURE	87
Starting the Session	87
Measuring	88
Reviewing the session	88
Generating a report	90
PROCEDURE FOR LUMBAR SPINE ASSESSMENT	90
Starting the Session	90
Measuring	91
Reviewing the session	92
Generating a report	93
PROCEDURE FOR CERVICAL SPINE ASSESSMENT	93
Starting the Session	93
Measuring	94
Reviewing the session	95
Generating a report	95
PROCEDURE FOR ANTERIOR KNEE ASSESSMENT	95
Starting the Session	95
Measuring	97
Reviewing the session	97
Generating a report	98
RANGE OF MOTION TRAINING	99
OVERVIEW	99
CLINICAL GUIDE REFERENCE	99
PROCEDURE	99
ROMT01 – ROM Training with 1 inclinometer	100
ROMT02 – ROM Training with 2 inclinometers	101
MANUAL MUSCLE TESTING	103
OVERVIEW	103
CLINICAL GUIDE REFERENCE	103
PROCEDURE	103

Starting the Session	103
Muscle Testing Training (MMT01 & MMT02)	104
Unilateral/Bilateral Muscle Testing (MMT03 & MMT04)	104
Measuring	105
Unilateral/Bilateral Muscle Testing with EMG (MMT05 & MMT06).....	106
Generating a report	106
ALGOMETRY	109
OVERVIEW	109
CLINICAL GUIDE REFERENCE.....	109
PROCEDURE	109
Starting the Session	109
Measuring	111
Generating a report	112
HARDWARE SPECIFICATIONS.....	114
PLACING ORDERS	117
TECHNICAL SUPPORT	117
WARRANTY	117
RETURNING EQUIPMENT FOR REPAIR	118
REPAIR RETURN FORM	119

Introduction

Welcome to the reference manual for Rehab Suite. This manual provides information intended to help you get the most out of your system.

The manual is organized into chapters as follows:

- **Installation and System Requirements** lists the computer requirements recommended for running the software.
- **First Time Run** explains how to start the software and verify its settings prior to using it for the first time.
- **Hardware Set-up** provides general information about the encoder, sensors and accessories included in the system. It also shows how to connect the hardware elements.
- **Software Overview** introduces the main functions of the BioGraph Infiniti software: recording, replaying, and reviewing sessions. It discusses the analytical processes of artifact rejection, statistics calculation, and report generation. In addition this section explains how to use a web camera with the software, how to calibrate and zero sensors, and how to run an impedance check.

The next chapters explain the protocols run by the system. These are:

- **General SEMG Assessment**
- **Static SEMG Assessment**
- **Dynamic SEMG Assessment**
- **Muscle Fatigue Monitoring**
- **SEMG Biofeedback**
- **Respiration Training**
- **Heart Rate Monitoring and HRV**
- **Skin Conductance & Temperature Biofeedback**
- **Static Range of Motion Assessment**
- **Dynamic Range of Motion Assessment**
- **Range of Motion Training**
- **Manual Muscle Testing**
- **Algometry**

Each protocol chapter presents the general principles of one specific protocol. It lists relevant references in the clinical guide. It provides step-by-step instructions for running the protocol. Finally, it describes how to generate a report.

The remaining sections of the manual provide a list of **Hardware Specifications**, and information about the **Warranty**, obtaining **Technical Support**, and **Returning Equipment** for repair if necessary.

Note: *Important safety information about operating the equipment is provided in the front matter of the manual starting on page i.*

REQUIREMENTS

Please refer to *IS7915 – BioGraph Infiniti PC System Requirements* for a detailed list of requirements for running BioGraph Infiniti 6.1.

First Time Run

Once the software is installed, you have to configure your system. This takes only a few minutes.

STARTING THE PROGRAM


To start the program, simply double-click on the BioGraph Infiniti icon on the Desktop.



The program always starts by showing the Main Menu screen.


From here you can set global settings that affect the performance of BioGraph Infiniti.

Note: *Global settings remain in effect unless you change them.*

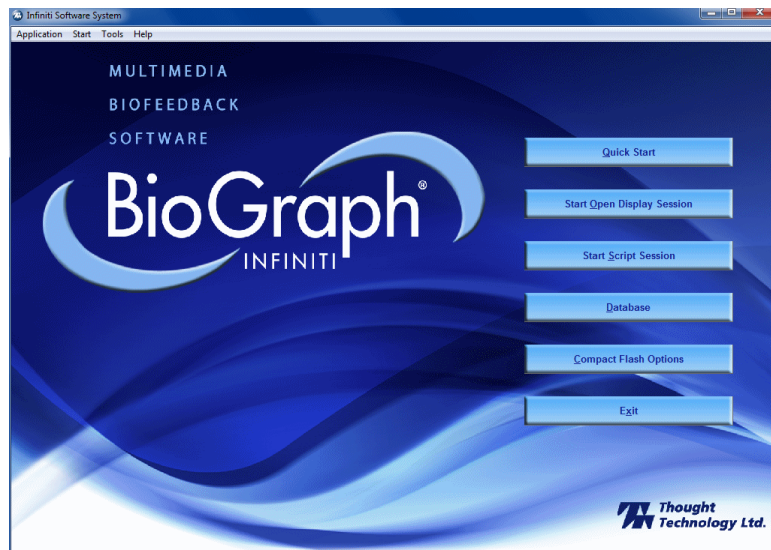

VERIFYING SETTINGS

Select **Preferences** from the **Tools** menu in the Main Menu Screen. In the **General** tab verify temperature and force units, notch filter settings and text report options.

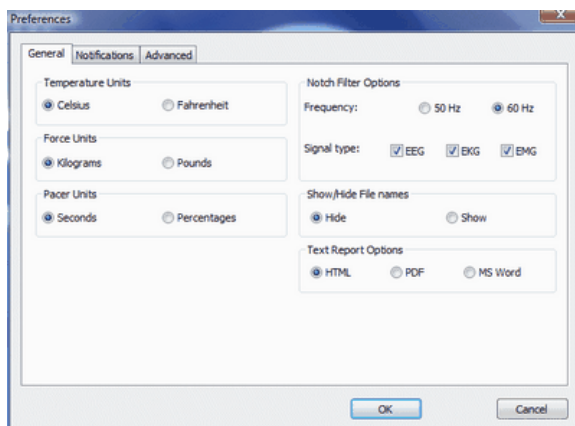

Temperature and Force Units

Default values for Temperature and Force are determined from your computer's Regional Settings when BioGraph Ininiti is run for the first time. Ensure that the correct values are selected, **Kilograms** or **Pounds** for **Force** and **Fahrenheit** or **Celsius** for **Temperature**.

Notch Filter

Default values for the notch filter Frequency are determined from your computer's Regional Settings when BioGraph Ininiti is run for the first time. Select each **Signal type** that you plan to use (**EEG**, **EKG** or **EMG**) and ensure that the **Frequency** is set to **50Hz** or **60Hz**, depending on the transmission frequency used in your country (60Hz for North America, for instance).

Text Report Options



You can generate text reports in one of three different formats, **HTML**, **PDF**, or **Microsoft Word**. Select your preferred format here.

Note: *Microsoft Word, a PDF reader, and a web browser **must** be installed on your system for these functions to work. You print and save text reports using the **Print** and **Save** functions of the appropriate software.*

Confidential Setting


Optionally, enable the **Confidential** setting. This helps protect the privacy of your clients by masking their names in the client database.

- Click **Database** to open the Main Database window.
- From the **Settings** menu, select **Confidential**. The check mark indicates that the setting is enabled.
- If you need to disable the **Confidential** setting, repeat these steps to remove the check mark.

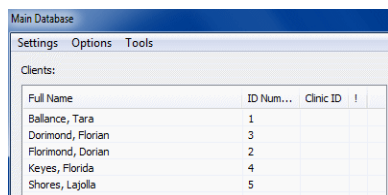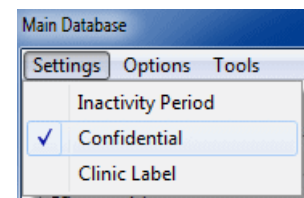

Full Name	ID Num...	Clinic ID	!
Ballance, Tara	1		
Dormond, Florian	3		
Florimond, Donian	2		
Keyes, Florida	4		
Shores, Lajolla	5		

With the **Confidential** setting disabled

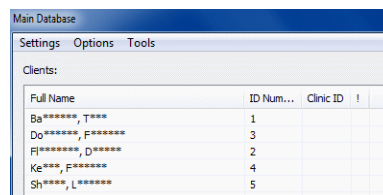

Full Name	ID Num...	Clinic ID	!
B***** T****	1		
D***** F*****	3		
F***** D*****	2		
K***** F*****	4		
S***** L*****	5		


With the **Confidential** setting enabled

ENTERING KEY CODES

The first time you start to record a session with BioGraph Ininiti, the software will check for the key codes and ask you to enter them. You will also be asked to enter key codes if you have uninstalled and reinstalled the software, or if this is the first time you are using your encoder with your existing installation.


There are three ways to start recording a session. They are described fully in the chapter [Software Overview](#) beginning on page 12.

- **Quick Start:** Click this option to record a Favorite session (page 12). This is the preferred method for recording sessions.
- **Start Open Display Session:** Click this option to record an unscripted session using your own selection of feedback screens (page 14).
- **Start Script Session:** Click this option to record a script session (page 13).

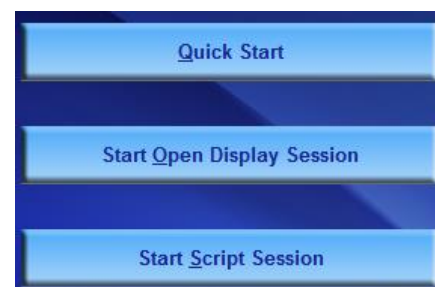


1. Click **Quick Start**, select a name from the **Clients** list and a protocol from the **Favorites** list, and then click **OK**.

2. The **Encoder Key Codes** dialog box opens.
3. Click to highlight your unit's serial number in the left table (in this example, DA1002; this is the same number as on the back of the unit).

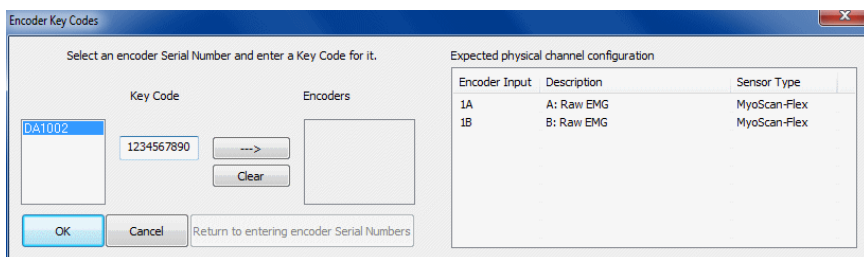


The serial number is found on the back of the encoder.

4. Enter the **Encoder Key Code** (found inside the encoder battery compartment) in the **Key Code** text box.
5. Click  to move the serial number from the left table to the right table. The serial number should now appear in the **Configuration Table**.
6. When done, click **OK**.
7. Now the **Application Key Code** dialog box opens.

8. Enter the **Application Key Code**.

This number is found in the battery compartment with the encoder key code.

9. When done, click **OK**.

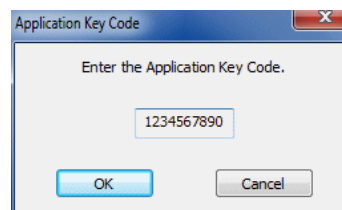

Note: Even if you are using more than one encoder, you need to enter only one Application Key Code.

If the key codes have been entered properly and there is no problem with your encoder setup or your sensor connections, the recording screen will open.

Hardware Set-up

Before you can start recording a session, you have to set up your hardware. The following section describes how to connect a FlexComp Infiniti encoder but the process is similar when using ProComp Infiniti and ProComp5 Infiniti. For more information on using your particular encoder, please refer to the device's hardware manual.

THE ENCODER

Connecting TT-USB and the encoder



Insert one end of the fiber optic cable carefully into the fiber optic port on the encoder. Tighten the nut gently so that the cable won't slip out.



Do the same with the other end of the fiber optic cable and the fiber optic port of the TT-USB interface unit.



Insert the small connector of the USB cable into the USB port on the TT-USB interface device.

Insert the large connector of the USB cable into the USB port of your PC.



USB ports on a PC are generally located at the back of the base unit. You may also find a USB port at the front of your base unit; you can connect the other end of the USB cable to it. On a laptop, USB ports are usually located at the side or the back of the laptop. It is recommended to always use the same USB port for connecting the TT-USB to your computer.



Connected hardware components

Battery Placement

Opening the Compartment

Looking at the underside of the encoder, you will see a long door in the plastic enclosure. Holding the encoder with the connectors facing you and with the door up, push lightly down and back with your thumbs on the door to slide it open.

Attention: *The internal workings of the encoder are not intended to be opened for repair except by qualified service personnel. The tamper-evident seal under the batteries in the battery compartment should not be removed or broken. Thought Technology may refuse to honour the unit's warranty if the seal is broken.*



Polarity

When the compartment cover has been slid out, place four AA batteries in the slots, observing the correct battery polarity as embossed on the inside surface of the compartment.

Closing the Compartment

Slide the door back into the encoder case, gently pushing it in until you feel the click of the locking mechanism.

Keeping an Eye on the Battery Level

Since each sensor draws a small amount of power from the batteries when connected to the encoder, it is better to connect only the sensors that are going to be used for a session before you start recording; this will ensure maximal battery life. Most Thought Technology software applications will display a battery power indicator; we recommend that you replace the batteries as soon as this indicator falls below about 50% of the battery power.

Caring For the Fiber Optic Cable

A fiber optic (FO) cable is used for transmitting the sensor data to the computer. Although this technology provides maximal electrical isolation, signal fidelity and freedom of movement, some care has to be taken when handling the fiber optic cable, as it is much less flexible than a regular electric wire.

Be careful not to bend it sharply or wind it in a loop smaller than 4 inches in diameter.



Electromagnetic Electrical Interference

When performing a biofeedback session, it is best to **avoid placing the encoder near the computer, display monitor or any other electrical device**, where it can be exposed to electromagnetic fields.

SENSORS AND ACCESSORIES

The instrumentation system includes a group of sensors used for the various protocols. Before a session can be recorded, the correct sensors must be connected to the encoder.

Electromagnetic interferences

The SEMG and EKG sensors are capable of detecting very tiny electrical signals (millionths of a Volt) generated by muscle activity. Therefore they are very sensitive to electromagnetic fields generated by other devices in the exam room, such as radio transmitting devices, computer monitors, medical devices (for example x-ray machines), and fluorescent, halogen or neon lights.

These devices should be turned off, if they are not needed for the examination. If the situation arises, keep the instrumentation 10 feet away from radio transmitting devices and 3 feet away from electronic devices (including monitors) and fluorescent, halogen or neon lights.


Disconnect all the unused sensors from the encoder. If not connected to the examinee, they may act as antennas and capture unwanted signals that would corrupt the signal.

Electrostatic discharges

To prevent static discharge from damaging the sensor and/or encoder, use anti-static mats or sprays in your working area. A humidifier may also be used to help prevent static environments by conditioning hot, dry air.

Connecting the sensors

When connecting a sensor to the encoder, make sure to properly line up the guiding dot on the top of the plug with the notch in the device's input socket.

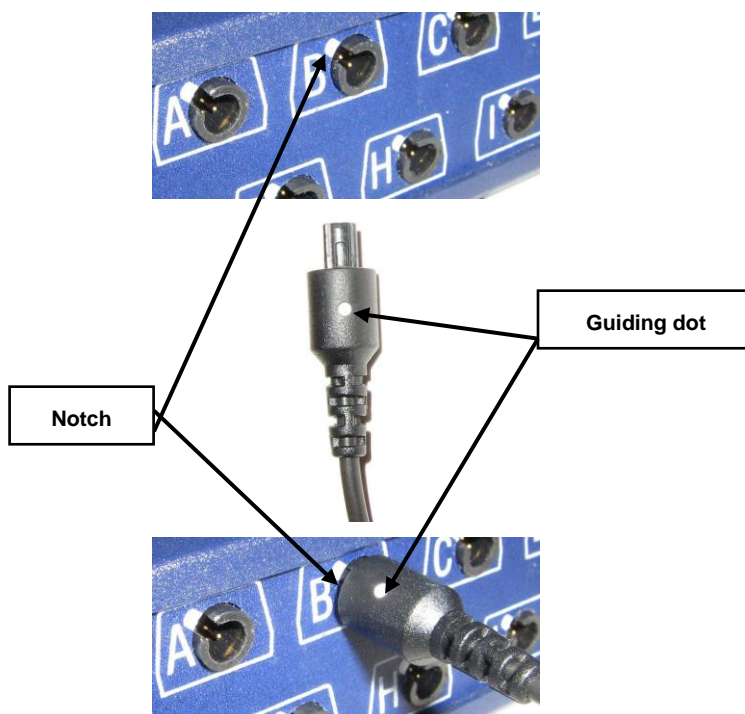


Forcing the plug into the jack in any other position may damage your equipment.

Correct orientation of sensors

For some sensors, such as the temperature and skin conductance sensors, either the connection cable or the electrode cable forms a permanent part of the item. Other sensors, for example the push button switch, have removable cables but only a single input entrance. This makes it impossible to connect these sensors in the wrong orientation to the encoder.

Certain sensors, however, have removable cables and two input entrances. It is important to orient these sensors to the encoder correctly so that the physiological signal is properly received. These sensors include the EKG, EEG, and EMG sensors, all of which are shaped like the illustration at the right.

Such a sensor is correctly oriented to the encoder when the cable joining them is connected to the bottom entrance of the sensor. If an extender cable with electrodes needs to be attached to such a sensor, it should be connected to the top entrance of the sensor.

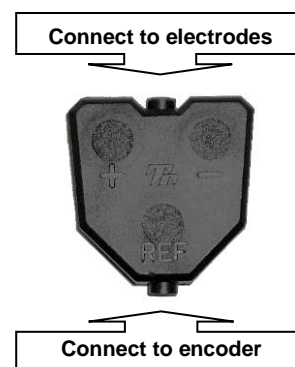

Sensors, Electrodes, and Accessories Used with the System

MyoScan/ MyoScan-Z EMG Sensor (P/N: SA9503M/SA9503Z)



The MyoScan is a pre-amplified surface electromyography sensor, sensing raw SEMG. MyoScan-Z also allows impedance check.

- Input Range 0 -2000 μ V
- Bandwidth 20 Hz – 500Hz.

Used for:

- SEMG Assessment
- Muscle Fatigue Monitoring
- SEMG Biofeedback
- Dynamic Range of Motion
- Respiration Training
- Manual Muscle Testing

MyoScan-Pro EMG (P/N: SA9401M-60 or SA9401M-50)




The MyoScan-Pro is a pre-amplified surface electromyography sensor for use on low sampling rate inputs of the ProComp5 and ProComp Infiniti, and designed to measure RMS SEMG.

Used for:

- SEMG Assessment
- SEMG Biofeedback

InclinoTrac/Dual-InclinoTrac (P/N: T7650/T7655)

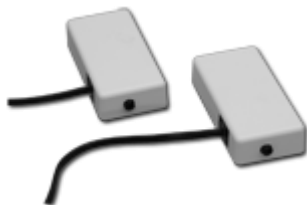

The Dual-InclinoTrac is a dual inclinometer that measures relative angle (between back and head, for instance).

Used for:

- Static Range of Motion Assessment
- Dynamic Range of Motion Assessment
- Range of Motion Training

ForceTrac (P/N: T7640)



The ForceTrac can be used as an algometer when used with its rod attachment.

ForceTrac can be used as a manual muscle tester when used with its rubber-coated probes (a flat one, and a curved one).

Used for:

- Manual Muscle Testing
- Algometry



Accessories to the ForceTrac are (from left to right):

- Rod attachment
- Curved muscle testing probe
- Flat muscle testing probe

HR/BVP Sensor (P/N: SA9308M)



The HR/BVP sensor is a blood volume pulse (BVP) detection sensor (also known as a photoplethysmography – PPG – sensor) housed in a small finger worn package, to measure heart rate (HR) and provide BVP amplitude, BVP waveform, HR and heart rate variability (HRV) feedback.

Used for:

- Heart Rate Monitoring and HRV

EKG-Z Sensor (P/N: SA9306M)



The EKG sensor is a pre-amplified electrocardiograph sensor, for directly measuring the heart's electrical activity. Connects via extender cables for a single channel hook up.

Used for:

- Heart Rate Monitoring and HRV

Respiration Sensor (P/N: SA9311M)



The respiration sensor is a sensitive girth sensor worn using an easy fitting high durability woven elastic band fixed with a length adjustable webbing belt. It detects chest or abdominal expansion/contraction and outputs the respiration waveform.

The sensor is latex-free, magnet-free, and Velcro-free, and can be worn over clothing.

Used for:

- Respiration Training
- Relaxation
- Heart Rate Monitoring and HRV

Skin Temperature Sensor (P/N: SA9310M)



The Temperature sensor measures skin surface temperature between 10°C – 45°C (50°F - 115°F). It is supplied with a self adhering band for easy finger placement.

Used for:

- Stress Assessment
- Relaxation

Skin Conductance Sensor (P/N: SA9309M)



The Skin Conductance sensor measures the conductance across the skin, and is normally connected to the fingers or toes. Supplied with two finger bands.

Used for:


- Stress Assessment
- Relaxation

EKG Wrist Straps (P/N: SA9325)




EKG Wrist Straps come in a kit containing reusable medical-grade, non-latex straps and replaceable silver/silver-chloride electrodes.

They provide fast, convenient placement of electrodes for measuring EKG from the wrists.

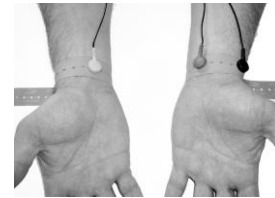

Disposable EKG/EMG electrodes

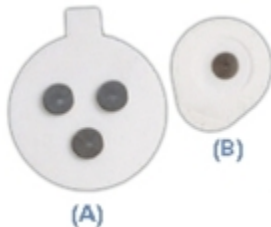

(A) T3402M – Triode™ electrode, with standard 2cm spacing of silver-silver chloride electrodes, backed with nickel plated brass snaps to prevent corrosion.

(B) T3425 – UniGel electrodes, for use with the EKG sensor or EMG, in case of sensitive placements on dry skin.

Static EMG electrodes (P/N: SA9327)



Standard silver-silver chloride re-usable electrodes, backed with nickel plated brass snaps to prevent corrosion when connected to pre-amplifiers for extended periods.

Foot Pedal (P/N: SA7551)



The Foot Pedal connects to the encoder and allows the user to remotely drive the software (initiate the recording).

Used for:

- SEMG para-spinal scanning
- Thermal scanning
- SEMG para-spinal scanning with thermographic scanning
- Range of Motion assessment
- Manual Muscle Testing
- Algometry

Tele-Infiniti CF™ (P/N: SA9600) - Optional



The Tele-Infiniti CF turns the encoder into a long range monitoring device. The unit disappears into the Compact Flash slot, leaving only the antenna visible to transmit data up to 300 feet (100 m).

This accessory is optional.

Software Overview

This chapter provides you with a rapid overview of the main functions of the BioGraph Infiniti software. Refer to the on-line help manual for a more detailed description of all the functions and features of the program.

To open the on-line help, press the F1 function key, on your keyboard, from any dialog box in the program. Because the on-line help manual is designed to provide contextual information, it will automatically open at the section that discusses the dialog box from which the help was called.



The information in this chapter is of a general nature. Each protocol chapter explains how to record, review and report upon data using that particular protocol.

RECORDING SESSIONS

With the BioGraph Infiniti software, two types of sessions can be recorded: **Open Display** and **Script** sessions. Both types of session can be recorded with BioGraph Infiniti.

- **Scripts** are predefined protocols that group together varied types of activities with specific display screens. They are used for evaluation and assessments because they produce statistics that can be used to determine the type of exercise and/or practice that is needed by the patient. These statistics can be trended across sessions to reflect patient progress.
- **Open Display** sessions are used for biofeedback training, when flexibility is important and you need to be able to rapidly change the session parameters, depending on the client's reactions. The kinds of feedback include various types of audio, such as music, tones, sounds and voice prompts. There are also visual media, such as animations, and different graphs that change color when the signals pass their threshold markers.

Start the Program

After making the necessary connections described in the previous chapter, turn on your encoder and start the BioGraph Infiniti software.

Starting a session from a Favorite


A **Favorite** is a predefined session configuration that contains all the settings you need for recording, replaying, and reviewing a session. Depending on the session type (script or open display), these settings include scripts, channel sets and screens, statistics for session reports, auto-rejection settings for review mode, and Excel report options. Favorites simplify and standardize the process of recording and reviewing sessions.

Rehab Suite includes a Favorite for every protocol. You find them listed in the Favorites table when you click the **Quick Start** button on the main BioGraph screen.


To start a session from a **Favorite**:

1. Click the **Quick Start** button on the main screen.
2. Optionally, click **Add New Client**, in the **Client Data** screen enter at least the person's first and last name, and click **OK**.
3. Select a name from the list of **Clients**.
4. Choose a **Category** from the drop-down list. Only the favorites belonging to the category will display.
5. Select a **Favorite** from the list.
6. Click **OK**.

If there is no problem with your encoder or sensor connection, the recording screen will open.

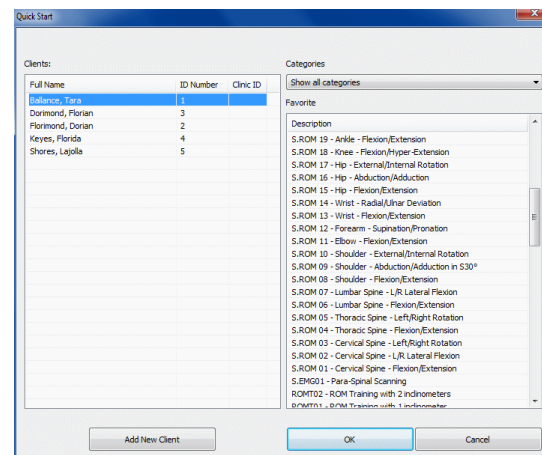


Note: If this is the first time you are recording a session, at this point the system will prompt you to enter key codes. See page 3 for more details.


7. Start recording by clicking on the **Start** button  in the toolbar.

Starting a Script Session

Select **Start Script Session** from the Main Menu screen. This opens the **Start Session** screen where you go through the process of selecting a client file and script, setting up session parameters, and launching recording mode.

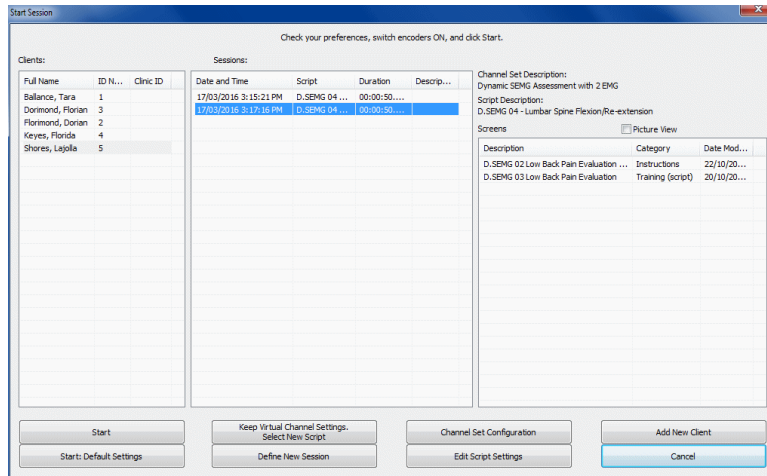

Start Session

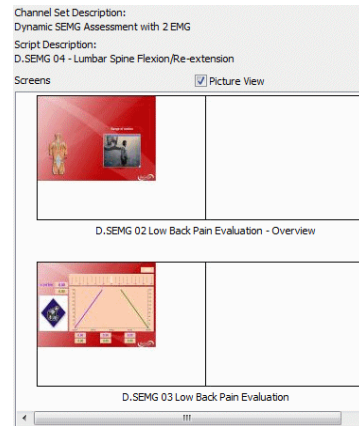


To see thumbnail images of the screens used in the chosen script, select **Picture View**.

To record with a new script:

1. Select an existing **Client**.
2. Optionally, select a previously recorded script session from the **Sessions** list.
3. Click one of the following **New Script** options:
 - **Define New Session** to select a new script.
 - **Keep Virtual Channel Settings. Select New Script** to keep the client's channel set and settings but select a different script.


The **Script Database** window opens.

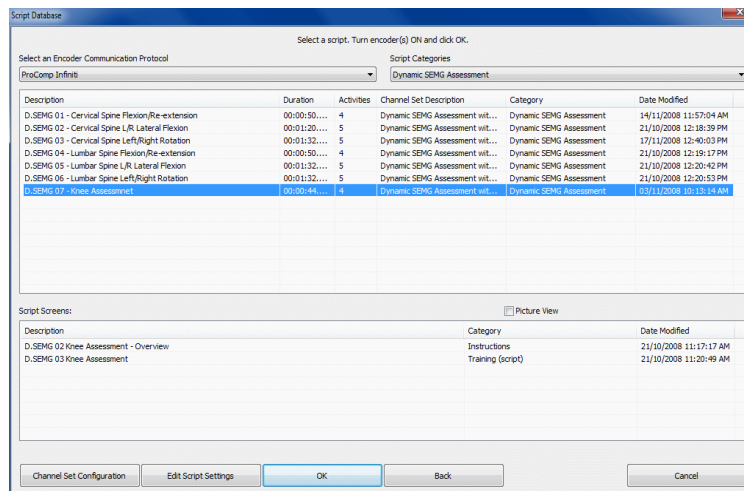

Script Database

4. Select **FlexComp Infiniti** or **ProComp Infiniti** as the encoder communication protocol from the drop-down list at the top.
5. Optionally, choose a **Script Category** from the drop-down list. Only the scripts belonging to the category will display.
6. Select a script from the list, and click **OK**.


Note: *If this is the first time you are recording a session, at this point the system will prompt you to enter key codes. See page 3 for more details.*

7. When the recording screen opens, start the script by clicking on the **Start** button  , and follow the on-screen instructions.

To record with a previously recorded script:

1. Select an existing **Client**.
2. Select a previously recorded script session from the **Sessions** list.
3. Click one of the following **Start** options:
 - **Start** to rapidly start a new session with the same display screens and script settings as the selected session. These include any changes you made to the client settings.
 - **Start: Default Settings** to start a new session with the default channel set settings instead of any changes you made to the client settings for the selected session.

Note: *If this is the first time you are recording a session, at this point the system will prompt you to enter key codes. See page 3 for more details.*


4. When the recording screen opens, start the script by clicking on the **Start** button  , and follow the on-screen instructions.

To record with a new client:

1. Click **Add New Client**, in the **Client Data** screen enter at least the person's first and last name, and click **OK**.
2. Select the new **Client** from the list.
3. Click **Define New Session** and when the **Script Database** opens, follow the remaining steps to record with a new script.

Starting an Open Display Session

Select **Start Open Display Session** from the Main Menu screen. This opens the **Start Session** screen where you go through the process of selecting a client file and channel set, setting up session parameters, and launching recording mode.

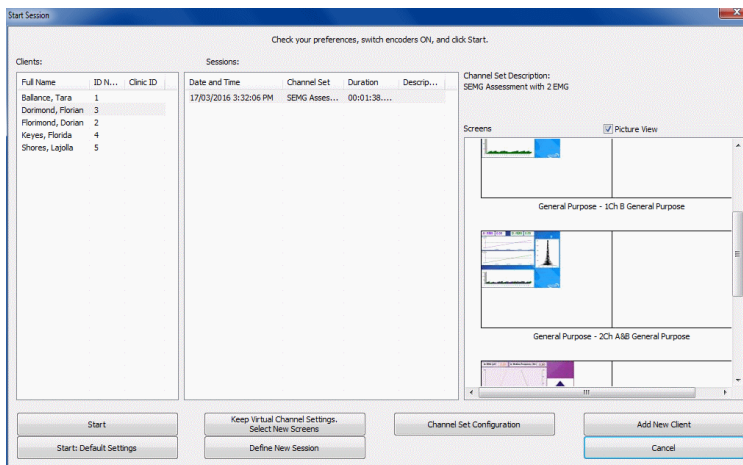


Start Session


To record a new session:

1. Create a new client as described above, or select an existing client from the list.
2. Select **Define New Session** to open the window.

The **Select up to 5 Screens for Open Display Mode** window opens.

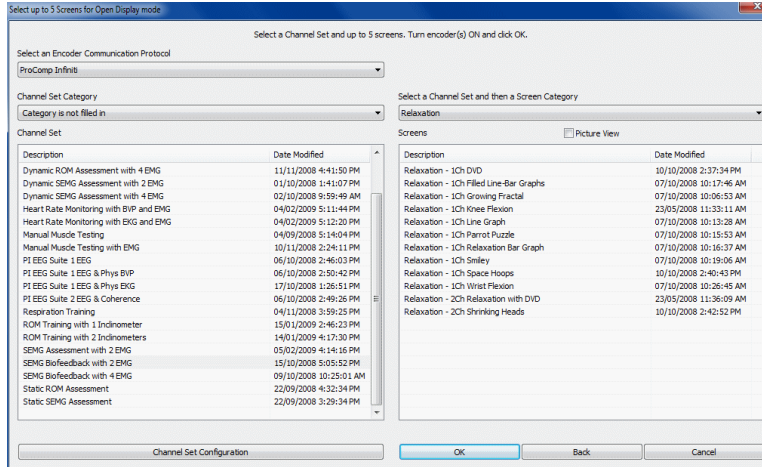

Select up to 5 Screens for Open Display Mode

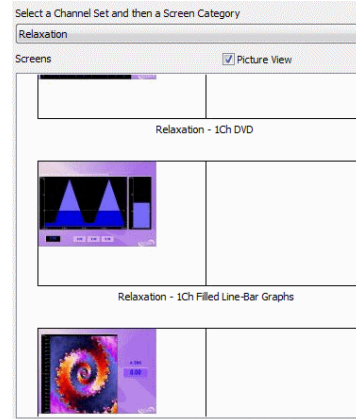

To see thumbnail images of the screens available for the chosen channel set, select **Picture View**.

3. Select **FlexComp Infniti** or **ProComp Infniti** as the encoder communication protocol from the drop-down list at the top.
4. Highlight a **Channel Set** on the left.
5. Optionally, choose a **Screen Category** from the drop-down list. Only the screens belonging to the category will display.
6. Choose one to five **Screens** from the list, and click **OK**.

To select more than one screen, press down the CTRL key while clicking with the left mouse button.


Note: If this is the first time you are recording a session, at this point the system will prompt you to enter key codes. See page 3 for more details.

7. When the recording screen opens, start recording by clicking on the **Start** button

Sensor Connections

If there are problems with your sensor connections, the program will report them prior to accessing the recording screen. If, for example, you have sensors that are connected but are not required for this channel set, you will get a warning message.

You can remove the sensor if you want, or continue recording by clicking **Yes**. Keep in mind that no data is recorded for sensors that are not defined in the channel set.

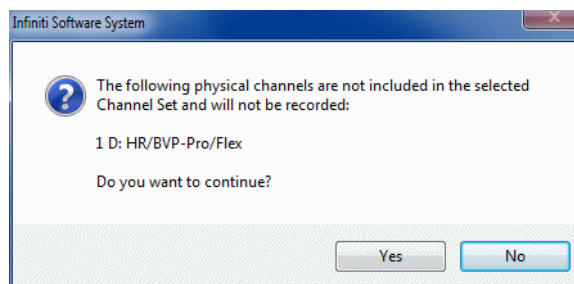


Note: If there are sensor connection problems in addition to extra sensors, this information won't appear as a separate message. It will display in the **Sensor Connections** window.

If you have connected wrong sensor types, you will see the **Sensor Connections** window. All the sensors that are expected will be listed in the **Expected Sensor** column.

The **Wrong Sensor Type** column shows which sensor type was actually detected. You can remove the wrong sensor or replace it with the right one.


Click **Repeat** to check the sensor connections again until the system no longer detects a wrong sensor type. The program will not allow you to record a session with the wrong sensors.

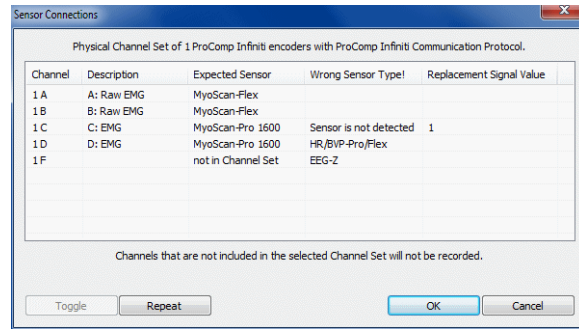

Note: If the **Wrong Sensor Type** column is empty, the correct sensor has been connected.

If you do not connect all the sensors that are required for the selected channel set, the message "Sensor is not detected" appears in the **Wrong Sensor Type** column. When a sensor is not connected, the program replaces its signal value by a constant value of 1. You can record a session when some sensors are missing, but the corresponding signal will be a flat line. Any calculations based on that signal will not reflect real physiological changes.

STOPPING A SESSION


No duration is set for an **Open Display** session. It must be stopped manually by clicking the **Stop** button .

The duration of a **Script** session is set at the time the script is defined. When the script reaches its end, recording will automatically stop.

When the session ends, the following series of message prompts will appear and guide you through the process of saving data, adding notes to the session file, and, optionally reviewing the data or recording another session.

1. Save session data:


You can save the session in compressed format but, unless your hard drive has very little space available or you do not need to review the saved sessions, we recommend that you always **Save**, which uses non-compressed format.

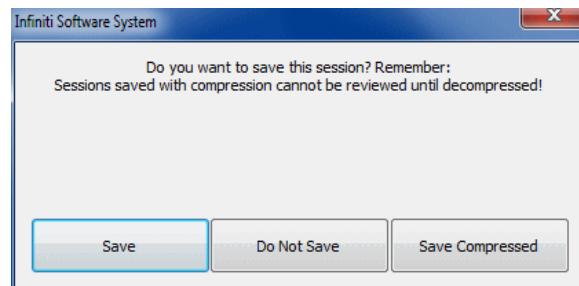

Compressed files can't be replayed, reviewed or trended until they have been decompressed.

2. Enter **Session Notes**:


Here you can enter, if you want, a treatment code, a session description and some notes. Click **OK** to continue.

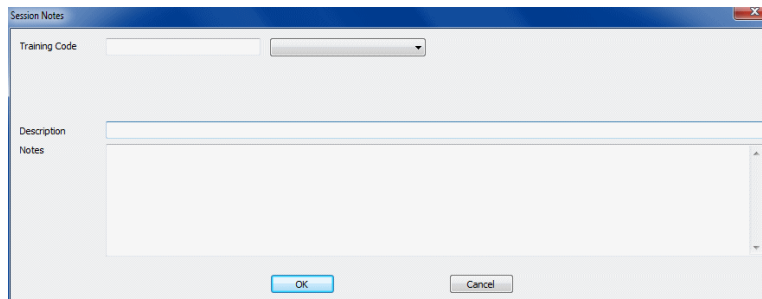

3. Review or replay the session:


You have the option to analyze the session data at this point. You can switch to review mode or replay mode.

After you review the data and close the viewing screen, the program displays the next message prompt.

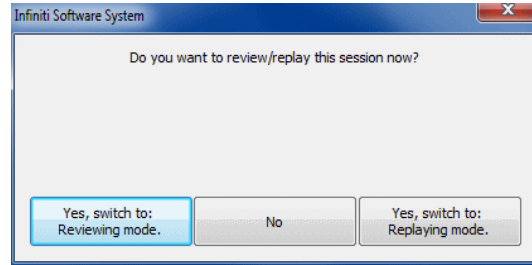

4. Record another session with the same settings:


- If you selected **No** at the previous message, the program asks if you want to record another session with the same settings.
- If you select **Yes** at this point, the recording screen will reopen.

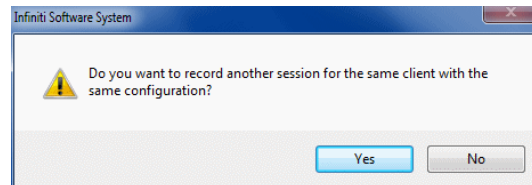

5. Turn off your encoder:

If you selected **No** at the previous message, the program prompts you to turn your encoder off.


Click **OK** to return to the Main Menu screen.

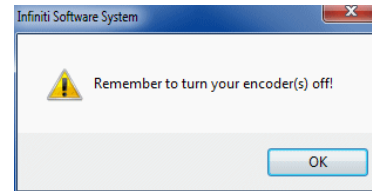

You can also manually stop recording a script session by clicking the **Stop** button .

In this case, you are prompted to confirm the command.


- If you select **No**, the script continues recording normally.
- If you select **Yes**, no data is saved for the session, and you are prompted to record another session with the same settings (message 4, above).

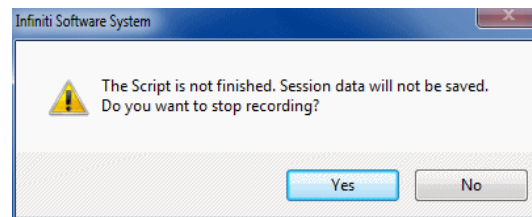

REPLAYING SESSIONS

Replaying a session allows you to play back a recorded session like a movie. This is useful if you have recorded video data. You can replay the session from any point in time and skip to event markers or script activities and steps.

1. Select **Database** from the Main Menu screen.
2. In the **Main Database** window select the client whose session you want to replay.
3. Select the session from the right-hand table.
4. Click on **Replay Session** at the bottom.

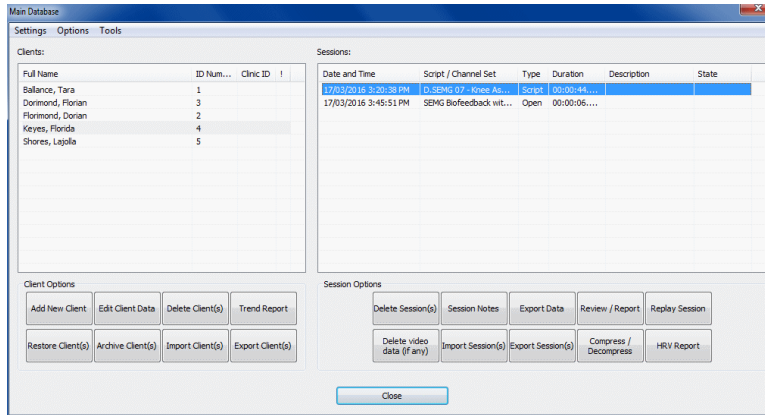


Main Database

5. The next step depends on whether you selected a **Script** session or an **Open Display** session to replay.
 - If you selected a Script session, the screen enters replaying mode.
 - If you selected an **Open Display** session, the window **Review/Replay Session Confirmation** opens.

To replay the session using different screens from those used to record it, click **Select Screens** to open the **Select Screens** window. Select up to 5 screens and click **OK** to return to **Review/Replay Session Confirmation**.

Click **OK** to enter replay mode.

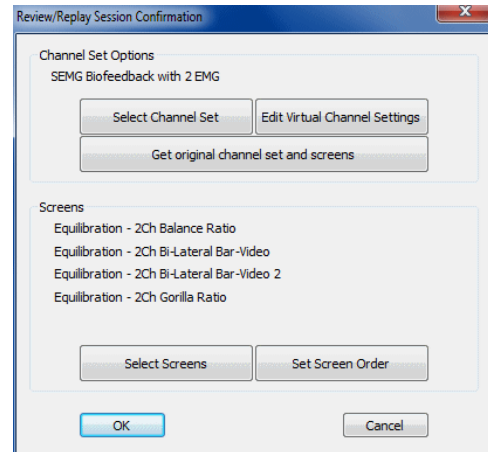

Review/Replay Session Confirmation

6. Click on the **Start** button  to replay the session.


REVIEWING SESSIONS

After recording a session you can analyze the data by going into review mode. This mode allows you to scroll through the graphs, look at statistics, and print out a report displaying the data in graphical and/or statistical form.


Review

1. Select **Database** from the Main Menu screen.
2. In the **Main Database** window select the client whose session you want to review.
3. Select the session from the right-hand table.
4. Click on **Review/Report** at the bottom.

The **Review/Replay Session Confirmation** window opens.

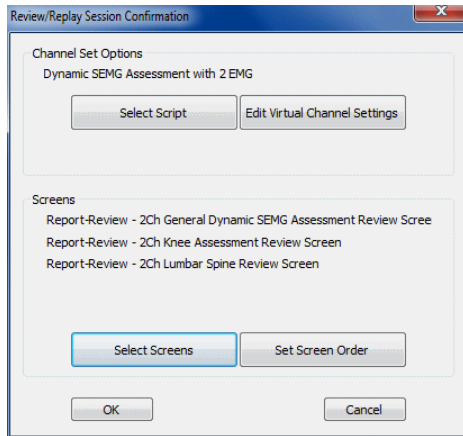

For a Script session

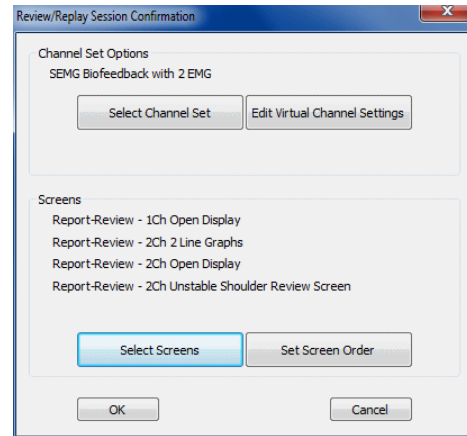

For an Open Display session


If you are reviewing a session that was recorded using a Favorite, then review screens will already be selected by default, and you can click **OK** to enter review mode.

5. Otherwise, to select review screens, do the following.

- Click on **Select Screens**.
- From the **Screen Category** drop-down list, select **Report-Review**. Only screens from the selected category will be listed.
- Select up to 5 screens and click **OK** to return to **Review/Replay Session Confirmation**.
- Click **OK** again to enter review mode.

Rejecting Artifacts

The purpose of rejecting artifacts is to have reliable statistics, with the least amount of noise affecting the recording. Rejecting artifacts can be performed while reviewing sessions. It can be done manually by the person reviewing the data. First review the session to determine where the recording is most affected by noise. Then place artifact rejection segments at these locations, by following these steps.

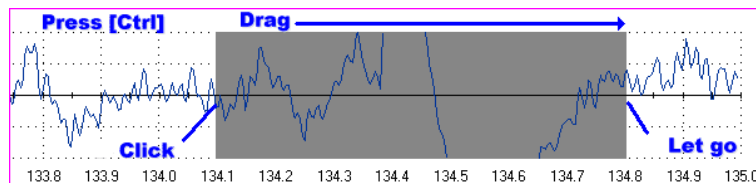

1. Select the line graph containing the segment to be rejected. The selected graph will display a magenta border.
2. Press and hold the **[Control]** Key.
3. Place the cursor over one end of the rejected segment.
4. Click and hold the left mouse button. (You can release the **[Control]** key at this point.)
5. Drag the cursor across the segment to be rejected. The selected segment is highlighted in gray.
6. At the end of the segment, release the mouse button.

Note: You can place artifact rejection segments only on single-line graph instruments. If you get an error message, you have to select a different screen for reviewing.

Calculating Statistics


You have two options for calculating statistics.

The option to **Calculate statistics for the whole session** is always available. It calculates statistics for all data recorded during the session, with the exception of data contained within rejection segments.

If you have set at least one **In/Out** area on a line graph, you have the option to **Calculate statistics for the selected areas**. It calculates statistics for all data contained within these areas. (A brief description of In/Out markers and how to set them is provided below.)

- To calculate statistics for a session, click the **Calculate Statistics** button  in the tool bar and select a calculation option.

A message confirms that the program is processing session data and then the **Statistics** window opens.

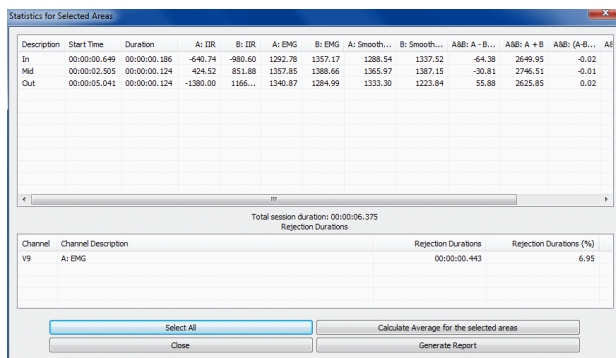


Description	Start Time	Duration	A: IIR	B: IIR	A: EMG	B: EMG	A: Smooth...	B: Smooth...	A&B: A - B...	A&B: A + B	A&B: (A-B)...	A&B
In	00:00:00.649	00:00:00.186	440.74	-980.60	1252.78	1357.17	1288.54	1337.52	44.38	2649.95	-0.02	
Mid	00:00:02.505	00:00:00.124	424.52	851.88	1357.85	1388.66	1365.97	1387.15	-30.81	2746.51	-0.01	
Out	00:00:05.041	00:00:00.124	-1380.00	1166...	1340.87	1284.99	1333.30	1223.84	55.88	2625.85	0.02	

Total session duration: 00:00:06.375
Rejection Durations

Channel	Channel Description	Rejection Durations	Rejection Durations (%)
V9	A: EMG	00:00:00.443	6.95

Buttons: Select All, Calculate Average for the selected areas, Close, Generate Report

Calculate statistics for the selected areas

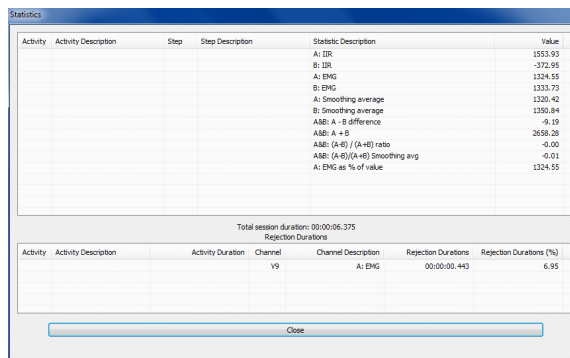

Activity	Activity Description	Step	Step Description	Statistic Description	Value
				A: IIR	1553.93
				B: IIR	-372.95
				A: EMG	1324.55
				B: EMG	1333.73
				A: Smoothing average	1320.42
				B: Smoothing average	1380.84
				A&B: A - B difference	-9.19
				A&B: A + B	2658.28
				A&B: (A-B) / (A+B) ratio	-0.00
				A&B: (A-B)/(A+B) smoothing avg	-0.01
				A: EMG as % of value	1324.55

Total session duration: 00:00:06.375
Rejection Durations

Activity	Activity Description	Activity Duration	Channel	Channel Description	Rejection Durations	Rejection Durations (%)
			V9	A: EMG	00:00:00.443	6.95

Buttons: Close

Calculate statistics for the whole session

If you selected **Calculate statistics for the selected areas**, you can select two or more areas and click **Calculate Average for the selected areas**.

To create an area statistics report, click **Generate Report** in this window. BioGraph Infiniti displays your report as a document in HTML, PDF, or Microsoft Word format, depending upon the setting in **Preferences**. The report can be saved or printed using the **Print** and **Save** functions of the appropriate software.

Note: When this window is open, the session report function in the Main Frame tool bar creates a report for the whole session.

- Click **Close** to close the window.

Setting In/Out Markers

You can perform this function on single line and multiple line graphs. It allows you to define specific areas where you can perform statistical analysis.

- Place your cursor at a specific point on the line graph, click the right mouse button, and select **Set In Marker for Area**.
- When **Event Marker Labels** opens, select or enter a name for the area and click **OK**. An **In** marker is placed on the graph.
- Place your cursor at a subsequent point on the graph, click the right mouse button again, and select **Set Out Marker for Area** to place an **Out** marker.

The two markers together indicate an area of the graph which is uniquely identified by the name you entered.

You can set as many **In/Out** areas as you want, but areas cannot overlap.

Generating Session Reports

In review mode BioGraph Infiniti 6.1 can generate two types of session report.



- Excel reports** have been predefined and customized especially for the Rehab Suite protocols. An Excel report transfers session statistics from the session data file into a Microsoft Excel workbook and subjects them to further analysis. Excel reports can be saved or printed using the Excel functions. Excel reports have not been defined for open display sessions.


The protocol chapters contain detailed information about their respective Excel reports.

- **Text reports** include general information about the session as well as optional information (session notes, statistics, markers) that you select. A text report can be generated from any screen which includes line graphs or trend graphs. Text reports can be generated in HTML, PDF, or Microsoft Word format. They can be saved or printed using the **Print** and **Save** functions of the relevant software. Text reports are preferred for open display sessions, but they can also be generated for script sessions.

To generate a session report, enter review mode either immediately after recording a session (see [Stopping a Session](#) on page 16), or from the Database window by selecting the session from the database and clicking on **Review/Report** (see [Reviewing a Session](#) on page 18).


Select review screens, and then follow these steps:

1. If this is an open display session, from the **Options** menu select **Set Open Display Statistics** and choose the channels for which statistics should be calculated.
2. If necessary, perform [artifact rejection](#) on the recorded data.
3. Click the **Calculate Statistics** icon  in the tool bar to recalculate the statistics.
4. If you have loaded more than one screen, open the screen from which you want to generate the report. Different screens can generate different reports.
5. Click the **Session Report** icon  in the tool bar and select the type of report that you want to generate.

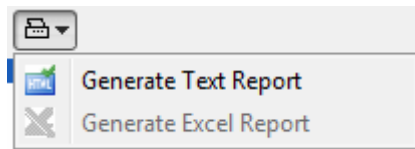


The icon beside **Generate Text Report** indicates whether the report will be created in HTML, PDF, or Microsoft Word format. To change this setting, go to the **Options** menu and open **Text Report Options**, where you can select a different format.


- If you select **Generate Excel Report**, Microsoft Excel opens and displays your report as a workbook. You can print or save the report using Excel's **Print** and **Save** functions.
 - If you select **Generate Text Report**, the **Text Report Options** window opens, and you must continue to the next step.
6. Select report components by placing a check mark in the desired boxes.
 7. Click **Generate Report** to generate the report. Your report displays as a document in HTML, PDF, or Microsoft Word format, depending upon the setting in **Preferences**. The report can be saved or printed using the **Print** and **Save** functions of the relevant software.

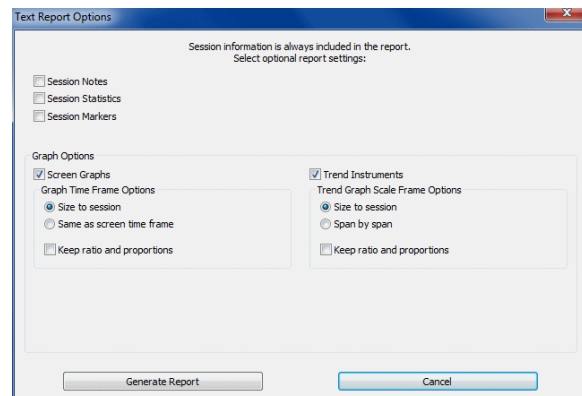

Note: Microsoft Word and Excel, a PDF reader, and a web browser **must** be installed on your system for the reporting functions to work.

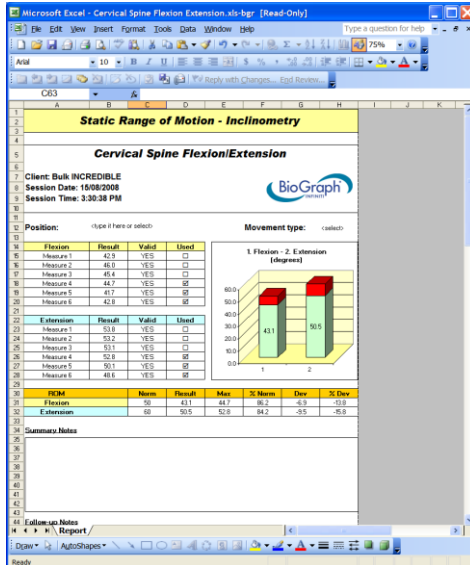

Excel-generated session report

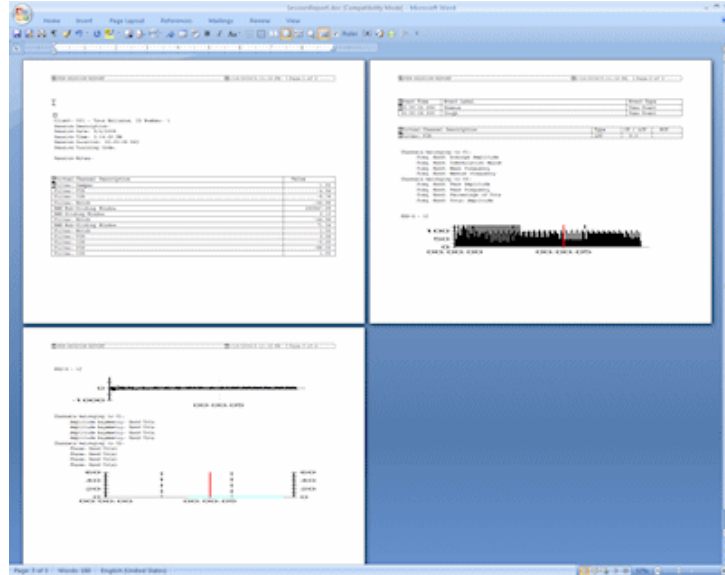

Word-generated session report

TOOL BAR OPTIONS

While recording, replaying or reviewing an open display session there are certain options that are available on a tool bar found near the top of the display screen. To learn more about the tool bar, refer to the online help by pressing F1.


Toggle Between Screens

If you have chosen more than one screen you can view each one by clicking on the numerical buttons on the toolbar at the top. Up to five screens can be viewed (in the example below, there are four, where the third screen is currently on display).




Adjusting Scales and Threshold

By clicking on a graph instrument you can enable any of the scale and threshold options on the toolbar shown below.

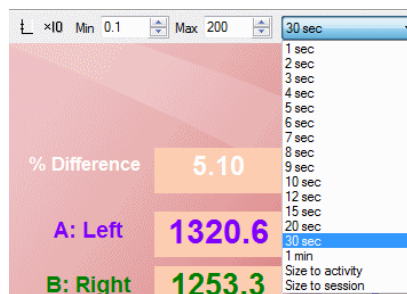

- **Vertical scale:** The minimum and maximum scale values can be changed by clicking on the up and down arrows to the right of the text boxes labeled with **Max** and **Min**.

The numbers increase or decrease by 0.1, but when you click on the **Scaling Coefficient** button , the numbers change by 1. (You can also type the desired value into the text box and then click anywhere on the screen to save the change.)

- **Multi line-graph Y1 and Y2 axes toggle:** The multiple line graph has a left and a right vertical scale that can be adjusted independently of each other by clicking on the toggle button.



Left vertical scale enabled.



Right vertical scale enabled.

- **Threshold:** The threshold line on a bar graph, line graph and animation scale can be adjusted in the same way as the vertical scale. The text box labeled **Thr1** is for single threshold graphs, while **Thr2** is for double threshold graphs like the multi-line graph that can have up to two guidelines.
- **Time scale:** The default time for all the screens is 30 seconds. You can select a new time scale by clicking on the drop-down menu.

USING A WEB CAMERA




When you see this logo on a screen in the manual, it means you can use a webcam with this screen. You only need to connect your webcam to a USB port of your computer and turn it on before you start the session.

Note: *Your computer system must be connected to a video capture device, such as a webcam, and the proper software installed, in order to use this function.*

The software searches for a connected video capture device and activates it automatically. A preliminary testing window will pop up, in order to adjust the camera. The video will be recorded with the session and can be reviewed or replayed.

If you have more than one video capture device connected to your system, and want to use a specific one, place the cursor on the video image and click the right mouse button, then select the device from the list:

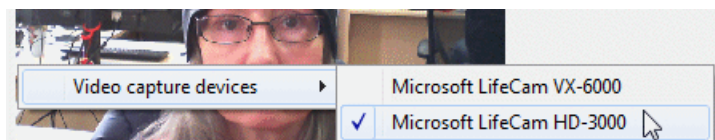

USING A DVD



When you see this logo on a screen in the manual, it means a DVD can be used with this screen. You need only insert the DVD in the DVD player of your computer.

When the option **Start DVD but Pause session** is selected, you can launch a DVD, access the DVD menu and start a movie while the program waits in Pause.

When you are ready to start recording data, click the **Pause** button. Recording will begin.


Note: *Your computer system must include a functioning DVD drive in order to use this function.*

SENSOR CALIBRATION AND ZEROING

The ForceTrac and Dual-InclinoTrac sensors may require calibration or zeroing. You do this in the Recording Screen before starting to run the protocol.

From the **Hardware** menu of the Recording Screen, select **Sensor Calibration**.

This opens the **Sensor Calibration & Zeroing** window. Here you can run a Wizard to zero or calibrate your sensor.


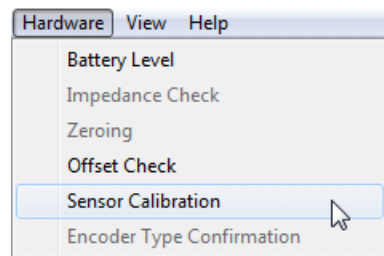


Encoder, Channels, and Sensors

The left pane of the window identifies the specific encoder by its serial number (in this case, CA1422), lists the channels for the encoder, and indicates the calibration and zeroing status of the sensor attached to each channel.

If a sensor has previously been zeroed or calibrated, the value entered in *Sensor serial no.* displays beside its channel identifier.

Click **Display Legend** to open a window explaining the symbols used to indicate sensor status.

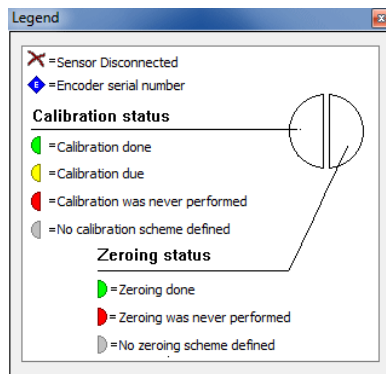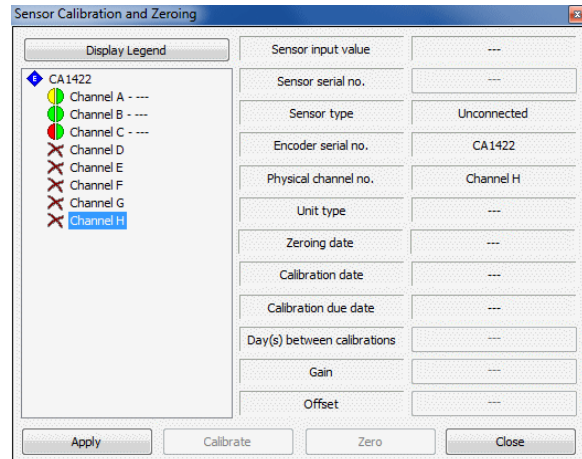

In the screen shot above, for example:

- For the sensor connected to Channel A, calibration and zeroing schemes exist. Zeroing has been performed, but calibration has not, and the sensor is due to be calibrated.
- For the sensor connected to Channel B, calibration and zeroing schemes exist. Both calibration and zeroing have been performed.
- For the sensor connected to Channel C, calibration and zeroing schemes exist. Zeroing has been performed, but calibration has never been performed.
- Sensors have not been connected to the other channels.


Command Buttons

The command buttons are in a row at the bottom of the window. Depending on whether you are calibrating or zeroing the sensor, some button labels will change.

Calibrate now: Click this button to open the Calibration Wizard, which provides step-by-step instructions leading you through the calibration procedure. The wizard opens the right pane of the window. When the Zeroing Wizard is active, this button is disabled.

Zero now: Click this button to open the Zeroing Wizard, which provides step-by-step instructions leading you through the zeroing procedure. The wizard opens the right pane of the window. When the Calibration Wizard is active, this button is disabled.

Note: If a sensor requires both calibration and zeroing, calibrate the sensor **before** you zero it.

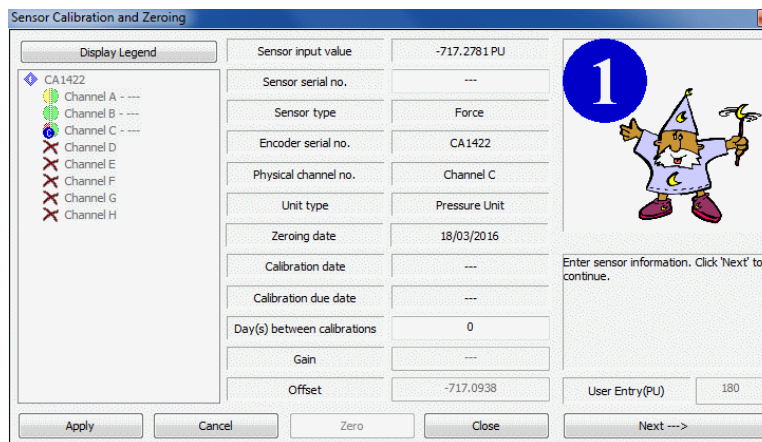

Apply: Click this button to save changes to the settings.

Abort calibration: In the Calibration Wizard, click this button to exit the wizard without saving changes.

Abort zeroing: In the Zeroing Wizard, click this button to exit the wizard without saving changes.

Next: After completing each step, click this button to proceed to the next step.


Close: At any step, click this button to discard any changes to the settings and close the wizard. You will be prompted to confirm that you want to discard the changes.

Sensor Status Details


The list in the middle panes of the window shows specific values for the sensor on the selected channel. For a description of these values, press F1 to consult the on-line help.

IMPEDANCE CHECK

It may be necessary to check impedance values for MyoScan-Z sensors. These values indicate the quality of contact between the electrodes and the skin. You do this in the Recording Screen before starting to run the protocol.

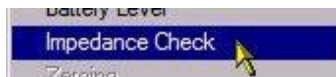

From the **Options** menu of the Recording Screen, select **Impedance Check**. This opens the **Impedance Check** window.

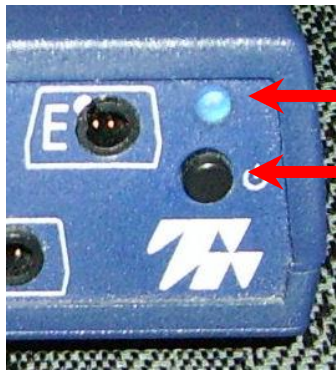

Then, trigger impedance checking from your encoder. To do this:

- Ensure that the encoder's **power light** is on.
- Press and hold down the **power button** for about 3 seconds, or until the power light starts blinking.


When the power light begins to blink, impedance checking mode is active.

The graphic display shows the impedance values of the selected sensor in green, orange or red.

- Green indicates a low value.
- Orange indicates a medium value.
- Red indicates a high value.

Ideally, all three measurements should display in green.

For more information about impedance checking, press F1 to consult the on-line help.

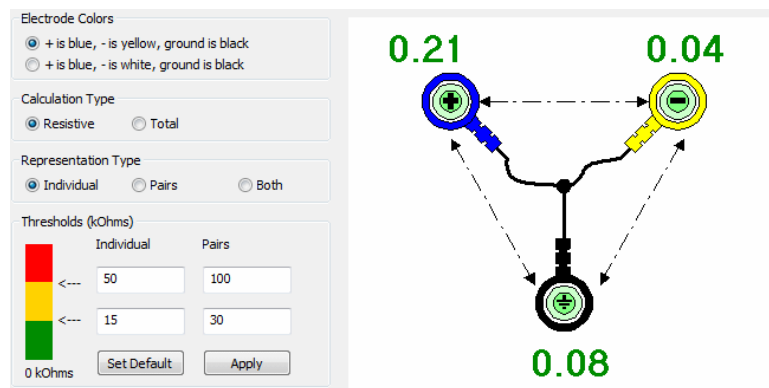

The screenshot shows the 'Impedance Check' window with the following settings:

- Electrode Colors:** + is blue, - is yellow, ground is black; + is blue, - is white, ground is black
- Calculation Type:** Resistive; Total
- Representation Type:** Individual; Pairs; Both
- Thresholds (kOhms):**

	Individual	Pairs
<---	50	100
<---	15	30

0 kOhms

Buttons: Set Default, Apply

The diagram shows three electrodes connected to a central ground point. The impedance values are: 0.21 (top left, blue), 0.04 (top right, yellow), and 0.08 (bottom, black).

To exit impedance checking mode, press the power button down for about 3 seconds, or until the power light stops blinking.

General SEMG Assessment

OVERVIEW


The following protocols are very common SEMG assessment techniques. They can be performed at the beginning of each visit. They allow you to quickly assess the examinee's muscle condition and determine the training parameters of the day (since an examinee's condition can change over time).



This logo on a screen in a protocol indicates that you can use an optional webcam with the screen to record video of the session. You need only connect your webcam to a USB port of your computer and turn it on before you start the session.

Note: *Your computer system must be connected to a video capture device, such as a webcam, and the proper software installed, in order to use this function.*

CLINICAL GUIDE REFERENCE

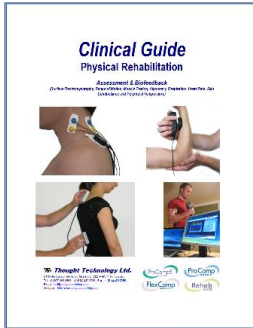

The clinical guide gives general guidelines and recommendations for the accurate use of the instrumentation. It also explains how to prepare the examinee and the procedure to follow.

Chapter: SEMG ASSESSMENT; **section:** SEMG SIGNAL ANALYSIS: GENERAL CONCEPTS

Also recommended:

- INTRODUCTION TO SURFACE ELECTROMYOGRAPHY
- SKELETAL MUSCLE PROPERTIES

GENERAL PROCEDURE

Starting the Session

If you are going use a webcam to make a video recording of this session, connect your webcam to a USB port of your computer and turn it on.

- Connect the sensors to the encoder as follows:
 - **MyoScan/MyoScan-Z sensor** to channel **A**.
 - Optional **MyoScan/MyoScan-Z sensors** to channel **B**, and channels **C to J** for FlexComp Ininiti.

A	B	C	D	E	F	G	H	I	J
Myo	Myo (optional)	Myo (optional for FlexComp)	Myo (optional for FlexComp)	Myo (optional for FlexComp)	Myo (optional for FlexComp)	Myo (optional for FlexComp)	Myo (optional for FlexComp)	Myo (optional for FlexComp)	Myo (optional for FlexComp)

- Turn the encoder on.

1. Click **Quick Start**.



- From **Categories**, select **Pro/Flex Rehab Suite - General SEMG Assessment**.
- From **Clients**, select a name.
- From **Favorites**, select the desired **Assessment Protocol**.
- Click **OK**.

Categories

Pro Rehab Suite - General SEMG Assessment

Full Name	ID Number	Clinic ID
Shores, Lajolla	5	

Favorite

Description
G.SEMG01 - Baseline
G.SEMG02 - Maximal Force
G.SEMG03 - Endurance
G.SEMG04 - Fast Flick
G.SEMG05 - Global SEMG Assessment 1 Muscle
G.SEMG06 - General SEMG Assessment with 2 EMG



Measuring

- Click the **Start** button .

Reviewing the session

When the session ends, a series of message prompts guides you through the process of saving data, adding notes to the session file, and switching to reviewing mode so that you can generate a session report.


In review mode, a screen similar to this will appear:

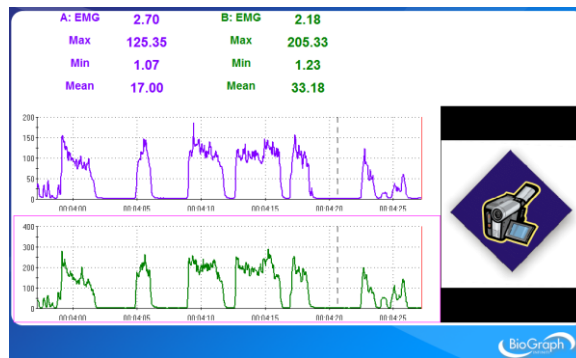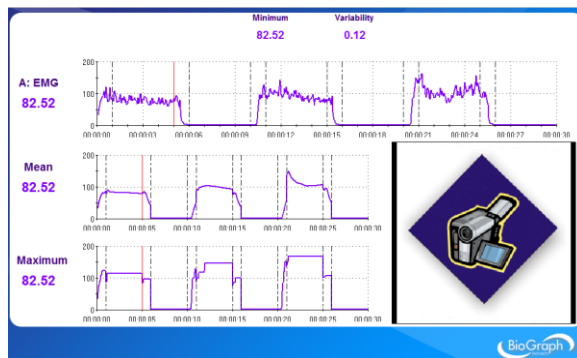

If you used a webcam to make a video recording of the session, during review the video playback is synchronized to the EMG signal as you move the Time Mark back and forth along the EMG signal line graph.

Generating a report

- After the review screen opens, you can generate a Dynamic SEMG Assessment report. This is a Text report.

- Click the **Session Report** icon  in the tool bar.
- Select **Generate Text Report**, select report components in the **Session Report window**, and click **Generate Report**.


Your report displays as a document in HTML, PDF, or Microsoft Word format, depending upon the setting in **Preferences**. The report can be saved or printed using the **Print** and **Save** functions of the relevant software.

G.SEMG01 – BASELINE ASSESSMENT

This protocol measures the resting level of the muscle. The patient must be asked to totally relax the muscle.

The protocol script is divided into 3 steps:

1. **Read instructions:** Gives you an overview of what to expect; no data is recorded; press any key when you are done.
2. **Verify signal:** Take the time to make sure the sensor and cables are connected properly. No data is recorded. Pressing any key on your keyboard allows you to advance to the next activity.
3. **Baseline:** Relax muscles to determine new resting levels; duration is 15 seconds.

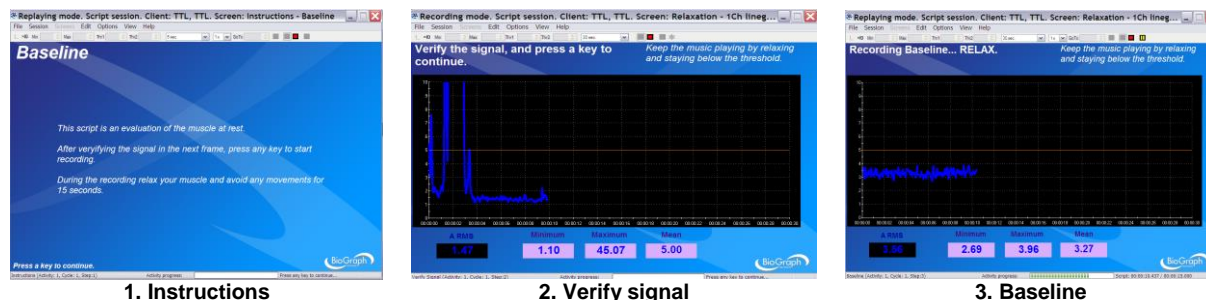


G.SEMG02 – MAXIMAL FORCE ASSESSMENT

This protocol measures the maximal force of the muscle. The maximal force is the highest level of voluntary contraction that a person can achieve without inducing unacceptable pain.

The protocol is divided into 3 steps:

1. **Read instructions:** Gives you an overview of what to expect; no data is recorded; press any key when you are done.
2. **Verify signal:** Take the time to make sure the sensor and cables are connected properly. No data is recorded. Pressing any key on your keyboard allows you to advance to the next activity.
3. **Rest-work cycle:** The patient relaxes and contracts muscles for 5 seconds, three times.

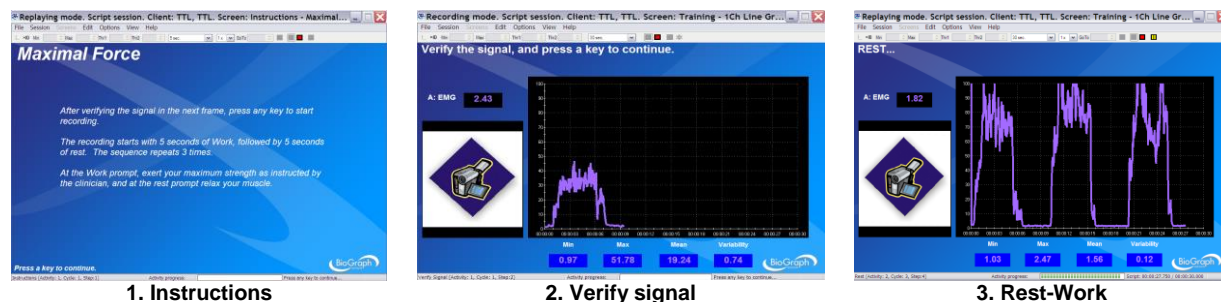


G.SEMG03 – ENDURANCE ASSESSMENT


This protocol assesses a sustained contraction. The patient contracts as strongly as they can during an extended period (about 20 seconds). This monitors the recruitment of the slow twitch fibers (muscle endurance). The contraction should be performed against static resistance (isometric contraction).

The protocol is divided into 5 steps:


1. **Read instructions:** Gives you an overview of what to expect; no data is recorded; press any key when you are done.
2. **Verify signal:** Take the time to make sure the sensor and cables are connected properly. No data is recorded. Pressing any key on your keyboard allows you to advance to the next activity.
3. **Pre-Baseline:** The patient must relax.
4. **Contract and Hold:** The patient contracts as strongly as they can during an extended period (about 20 seconds).
5. **Post-Baseline:** Comparison of resting levels (pre/post) assesses the ability to recover.

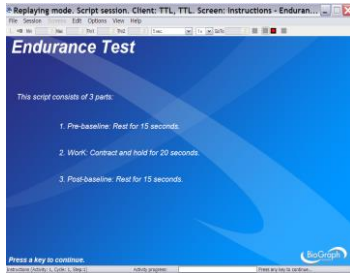


1. Instructions

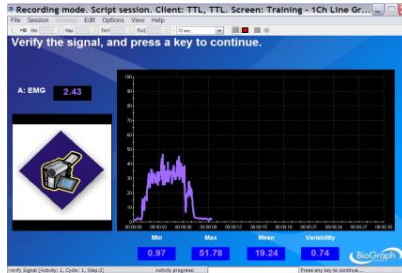


2. Verify signal

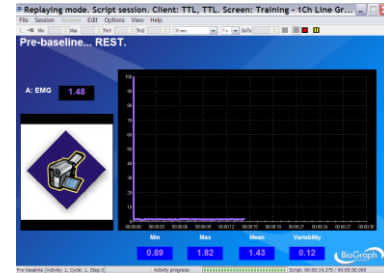

3. Pre baseline

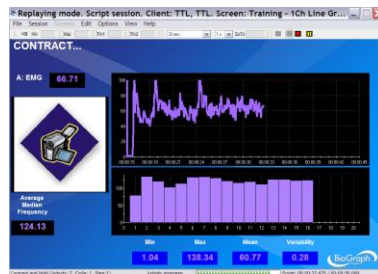

4. Contract and Hold

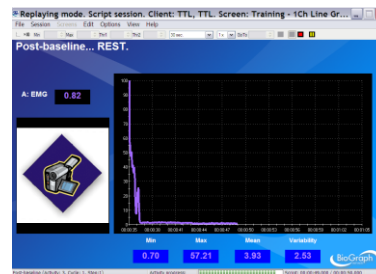


5. Post baseline

G.SEMG04 – FAST FLICK ASSESSMENT

This protocol measures the maximal force of the muscle. The maximal force is the highest level of voluntary contraction that a person can achieve without inducing unacceptable pain.

The protocol is divided into 5 steps:


1. **Read instructions:** Gives you an overview of what to expect; no data is recorded; press any key when you are done.
2. **Verify signal:** Take the time to make sure the sensor and cables are connected properly. No data is recorded. Pressing any key on your keyboard allows you to advance to the next activity.
3. **Pre-Baseline:** The patient must relax.
4. **Rapid Contractions:** the patient repetitively contracts as quickly and strongly as they can.
5. **Post-Baseline:** Comparison of resting levels (pre/post) assesses the ability to recover.

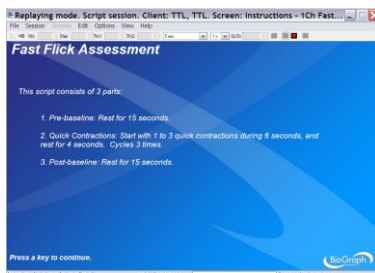


1. Instructions




2. Verify signal

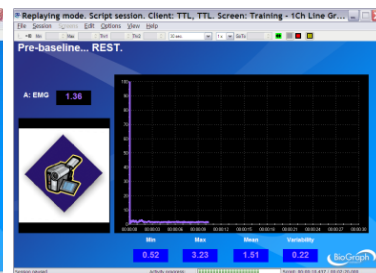

3. Pre baseline

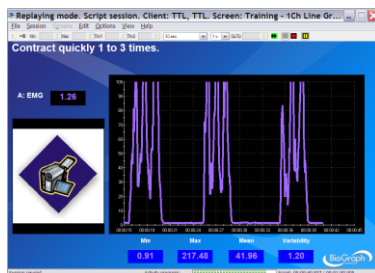


4. Rapid Contractions

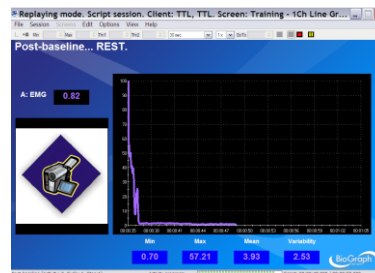


5. Post baseline

G.SEMG05 – GLOBAL SEMG ASSESSMENT 1 MUSCLE


This is a complete assessment of the muscle with 5 activities: pre baseline, fast flicks (rapid contractions), work/rest (maximal force), endurance and post baseline.

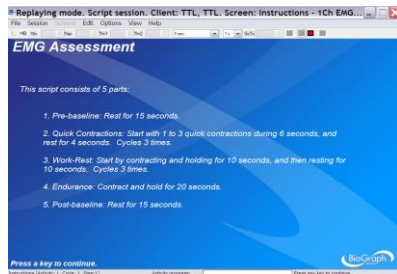


1. Instructions

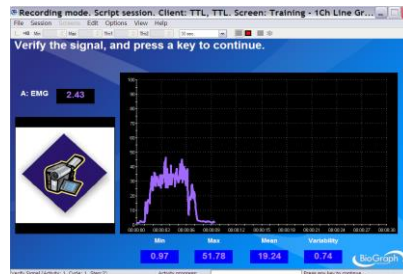


2. Signal Verification

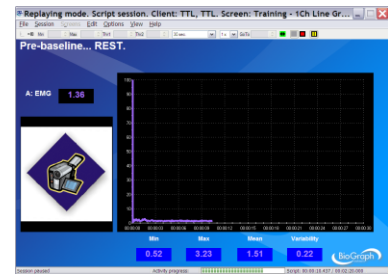


3. Pre-baseline

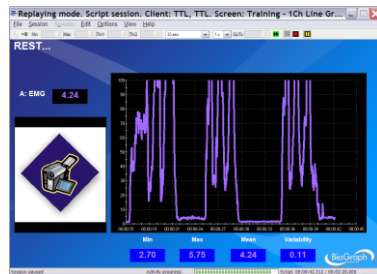


4. Fast flicks

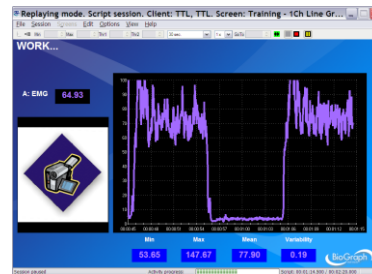

5. Work/Rest

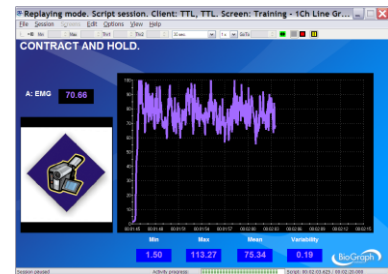

6. Endurance

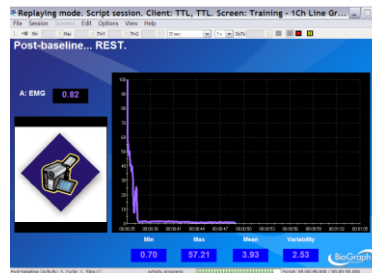

7. Post baseline

SEMG ASSESSMENT – GENERAL PURPOSE (G.SEMG06 and up)


This section regroups generic display screens designed for SEMG Assessment. As opposed to the protocols above, here there is no sequence of actions and limitation of time.

G.SEMG06 - SEMG Assessment with 2 EMG

General Purpose – 1Ch A General Purpose

1 2 3 4 5


One channel display of signal. The line graph is to display the RMS EMG. It displays the power spectrum and median frequency, as well as statistics.

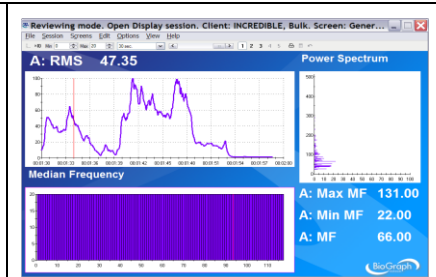

General Purpose – 1Ch B General Purpose

1 2 3 4 5


One channel display of signal. The line graph is to display the RMS EMG. It displays the power spectrum and median frequency, as well as statistics.

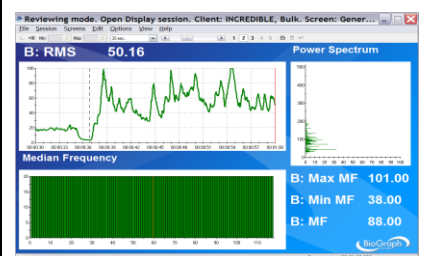

General Purpose – 2Ch A&B General Purpose

1 2 3 4 5


The two line graphs on the top display the raw EMG of A and B. The power spectrum displays A on the left and B on the right. The graph at the bottom displays the median frequency of both channels.

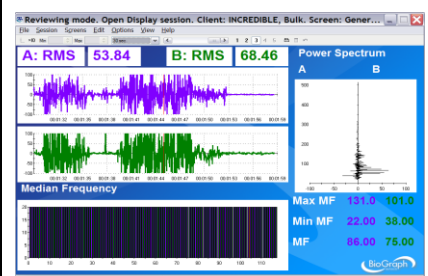

General Purpose – 2Ch Line-Bar Graphs

1 2 3 4 5

The two RMS EMG signals are displayed on line graphs and bar graphs. Statistics are also displayed.

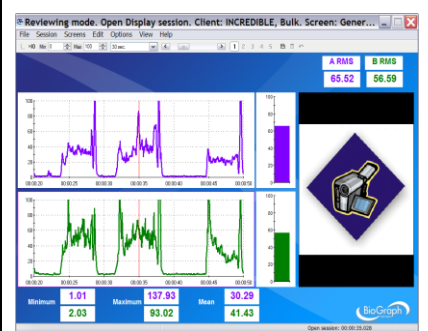


G.SEMG07 - SEMG Assessment with 4 EMG (FlexComp Infiniti only)

General Purpose – 4 EMG RMS with video

1 2 3 4 5


4 RMS EMG signals are displayed on 4 line graphs and bar graphs.

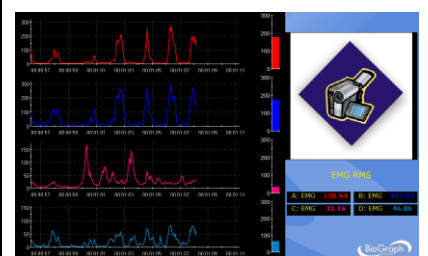

General Purpose – 4 raw EMG with video



1 2 3 4 5

4 raw EMG signals are displayed on 4 line graphs. The bar graphs and digital displays show RMS EMG.

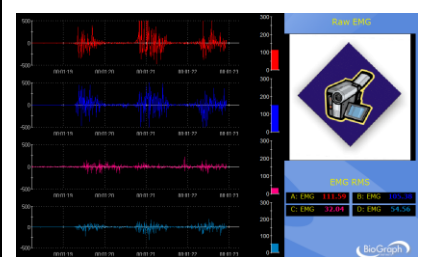



These screens are also used for review.

G.SEMG08 - SEMG Assessment with 6 EMG (FlexComp Infiniti only)
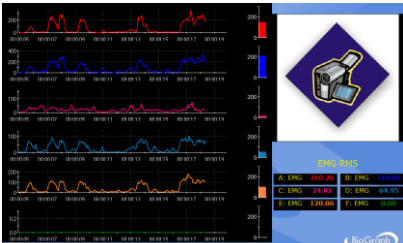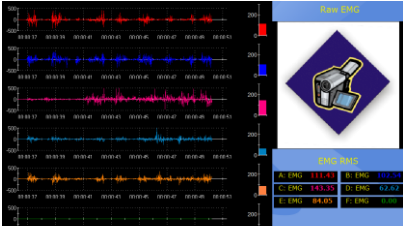
<p>General Purpose – 6 EMG RMS with video</p> <p>1 2 3 4 5</p> <p>6 RMS EMG signals are displayed on 6 line graphs and bar graphs.</p>	 <p>The screenshot displays six line graphs of RMS EMG signals. A summary table on the right shows the following values:</p> <table border="1"> <tr><td>A. EMG</td><td>200.78</td><td>B. EMG</td><td>111.39</td></tr> <tr><td>C. EMG</td><td>24.07</td><td>D. EMG</td><td>64.95</td></tr> <tr><td>E. EMG</td><td>130.06</td><td>F. EMG</td><td>68.00</td></tr> </table>	A. EMG	200.78	B. EMG	111.39	C. EMG	24.07	D. EMG	64.95	E. EMG	130.06	F. EMG	68.00
A. EMG	200.78	B. EMG	111.39										
C. EMG	24.07	D. EMG	64.95										
E. EMG	130.06	F. EMG	68.00										
<p>General Purpose – 6 raw EMG with video</p> <p>1 2 3 4 5</p> <p>6 raw EMG signals are displayed on 6 line graphs. The bar graphs and digital displays show RMS EMG.</p>	 <p>The screenshot displays six raw EMG signals. A summary table on the right shows the following values:</p> <table border="1"> <tr><td>A. EMG</td><td>111.43</td><td>B. EMG</td><td>110.44</td></tr> <tr><td>C. EMG</td><td>143.35</td><td>D. EMG</td><td>62.62</td></tr> <tr><td>E. EMG</td><td>84.07</td><td>F. EMG</td><td>67.00</td></tr> </table>	A. EMG	111.43	B. EMG	110.44	C. EMG	143.35	D. EMG	62.62	E. EMG	84.07	F. EMG	67.00
A. EMG	111.43	B. EMG	110.44										
C. EMG	143.35	D. EMG	62.62										
E. EMG	84.07	F. EMG	67.00										

These screens are also used for review.

G.SEMG09 - SEMG Assessment with 10 EMG (FlexComp Infiniti only)
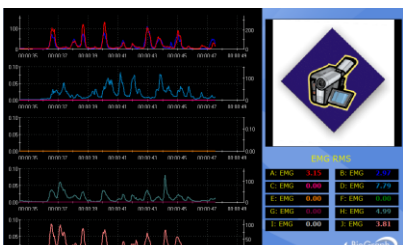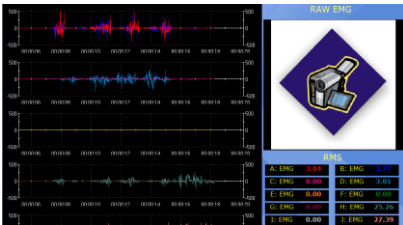
<p>General Purpose – 10 EMG RMS with video</p> <p>1 2 3 4 5</p> <p>10 RMS EMG signals are displayed on 5 line graphs.</p>	 <p>The screenshot displays five line graphs of RMS EMG signals. A summary table on the right shows the following values:</p> <table border="1"> <tr><td>A. EMG</td><td>3.15</td><td>B. EMG</td><td>2.92</td></tr> <tr><td>C. EMG</td><td>0.00</td><td>D. EMG</td><td>2.70</td></tr> <tr><td>E. EMG</td><td>0.00</td><td>F. EMG</td><td>0.00</td></tr> <tr><td>G. EMG</td><td>0.00</td><td>H. EMG</td><td>0.00</td></tr> <tr><td>I. EMG</td><td>0.00</td><td>J. EMG</td><td>3.81</td></tr> </table>	A. EMG	3.15	B. EMG	2.92	C. EMG	0.00	D. EMG	2.70	E. EMG	0.00	F. EMG	0.00	G. EMG	0.00	H. EMG	0.00	I. EMG	0.00	J. EMG	3.81
A. EMG	3.15	B. EMG	2.92																		
C. EMG	0.00	D. EMG	2.70																		
E. EMG	0.00	F. EMG	0.00																		
G. EMG	0.00	H. EMG	0.00																		
I. EMG	0.00	J. EMG	3.81																		
<p>General Purpose – 10 raw EMG with video</p> <p>1 2 3 4 5</p> <p>10 raw EMG signals are displayed on 10 line graphs. The digital displays show RMS EMG.</p>	 <p>The screenshot displays ten raw EMG signals. A summary table on the right shows the following values:</p> <table border="1"> <tr><td>A. EMG</td><td>0.00</td><td>B. EMG</td><td>0.00</td></tr> <tr><td>C. EMG</td><td>0.00</td><td>D. EMG</td><td>3.81</td></tr> <tr><td>E. EMG</td><td>0.00</td><td>F. EMG</td><td>0.00</td></tr> <tr><td>G. EMG</td><td>0.00</td><td>H. EMG</td><td>25.20</td></tr> <tr><td>I. EMG</td><td>0.00</td><td>J. EMG</td><td>27.39</td></tr> </table>	A. EMG	0.00	B. EMG	0.00	C. EMG	0.00	D. EMG	3.81	E. EMG	0.00	F. EMG	0.00	G. EMG	0.00	H. EMG	25.20	I. EMG	0.00	J. EMG	27.39
A. EMG	0.00	B. EMG	0.00																		
C. EMG	0.00	D. EMG	3.81																		
E. EMG	0.00	F. EMG	0.00																		
G. EMG	0.00	H. EMG	25.20																		
I. EMG	0.00	J. EMG	27.39																		


These screens are also used for review.

Static SEMG Assessment

OVERVIEW

Static SEMG Scanning assesses the resting level of the muscles surrounding the spine in static position. The protocol scans 12 sites x 2 (left/right) from C2 to L5.

CLINICAL GUIDE REFERENCE

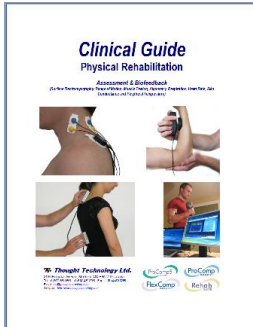

The clinical guide gives general guidelines and recommendations for the accurate use of the instrumentation. It also explains how to prepare the examinee and the procedure to follow.

Chapter: SEMG ASSESSMENT; **section:** STATIC SEMG ASSESSMENT

Also recommended:

- INTRODUCTION TO SURFACE ELECTROMYOGRAPHY


PROCEDURE


Starting the Session


- Connect the sensors to the encoder as follows:
 - MyoScan or MyoScan-Z sensors to channels A and B (left in A, right in B);
 - Foot pedal to channel C.

A	B	C	D	E	F	G	H	I	J
Myo (left)	Myo (right)	Pedal							

- Turn the encoder on.
 1. Click **Quick Start**.
 2. From **Categories**, select **Pro/Flex Rehab Suite - Static SEMG Assessment**.
 3. From **Clients**, select a name.
 4. From **Favorites**, select **S.EMG01 – Para-Spinal Scanning**.
 5. Click **OK**.

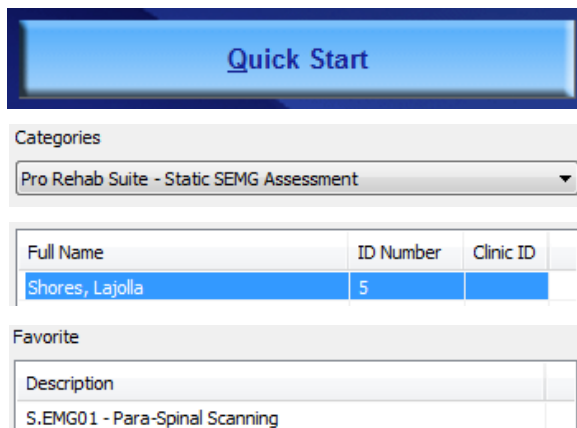


- Click the **Start** button  and read the instructions on the screen. **Do not press the foot pedal yet.**

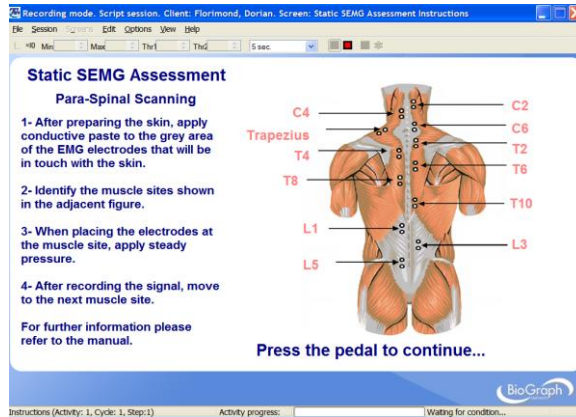

- Put conductive electrode paste or cream on the EMG electrodes (grey area only).
- Position yourself next to the examinee, the encoder attached to your belt, with the pedal near your foot.
- Press the foot pedal to go to the next screen.

Measuring

1. Place the active electrodes (positive and negative) in alignment with the spine (except for trapezius), at a distance of 3 cm out from the vertebral ridge, and at the site shown on the screen.

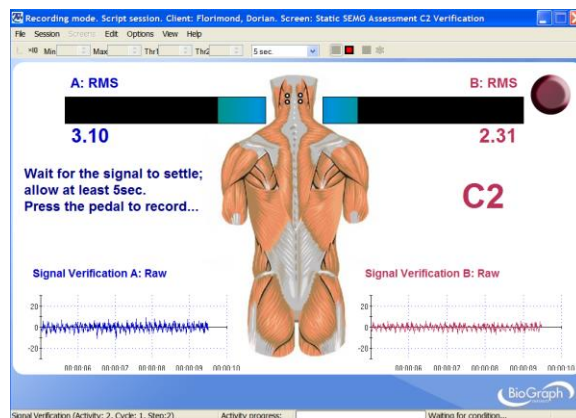


2. Position your hands so that the electrodes remain stable and do not move, as shown in the picture below. Apply gentle pressure on the sensor so that the 3 electrodes make good contact with the skin.




3. Ask the examinee to relax their muscles.

4. Wait for the signal to settle; allow at least 5 seconds before recording the data. But do not wait more than 30 seconds; the signal will not settle more and you risk creating movement artifacts by holding the electrodes too long.
5. Press the foot pedal to record.

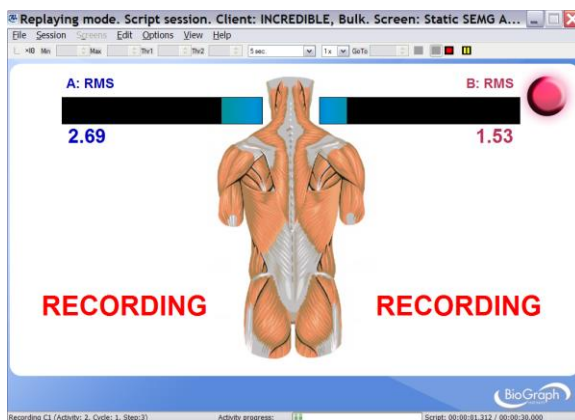


6. Then move to the next site.

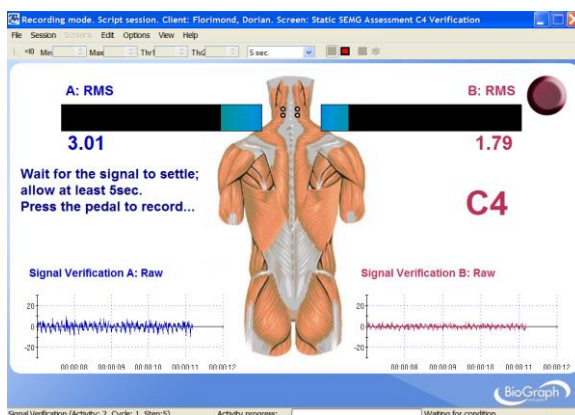


- Repeat steps 1 to 6 for sites from C2 to L5.
- When the session ends, a series of message prompts guides you through the process of saving data, adding notes to the session file, and switching to reviewing mode so that you can generate a session report.

Generating a report

- After the review screen opens, you can generate a SEMG Assessment report. This is an Excel report.

1. Click the **Session Report** icon  in the tool bar.
2. Select **Generate Excel Report**.

Microsoft Excel opens and displays your report as a workbook. You can print or save the report using Excel's **Print** and **Save** functions.

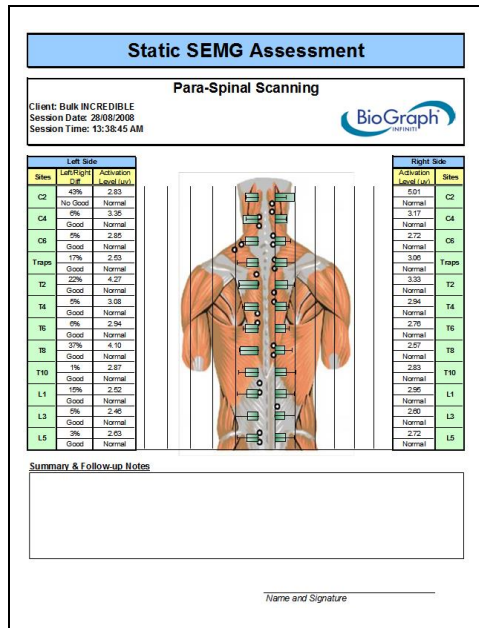

Normative Data are shown with a “T” on the graph:




The report shows the difference between left and right (in %) for each site. The formula is $(\text{Max}(\text{left}, \text{right}) - \text{Min}(\text{left}, \text{right})) / \text{Max}(\text{left}, \text{right})$.

The report rates the level of activation of each site:

Ranges	Level of activation	Color code
Between Mean – 1 StD. and Mean + 1 StD.	Normal	□ (green)
Between Mean + 1 StD. and Mean + 2 StD.	Mid Activation	□ (yellow)
Between Mean + 2 StD. and Mean + 3 StD.	Moderate Activation	□ (orange)
Greater than Mean + 3 StD.	Severe Activation	■ (red)
Less than Mean – 1 StD.	Mid Inhibition	□ (yellow)
Less than 1µV or greater than 25µV	Invalid Data	⊠ (red)

Summary and Follow-up notes

You can enter notes in the summary/follow-up notes boxes:

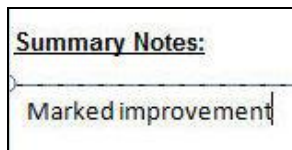

- In **Excel 2000** and **Excel 2003**, place your cursor in the text box and start typing.
- In **Excel 2007**, place your cursor in the text box.
 1. Right-click the mouse button.
 2. Select **Edit text**.
 3. Start typing.



Dynamic SEMG Assessment

OVERVIEW

Dynamic SEMG assesses muscle tension through various movements, such as flexion/extension, lateral flexion or rotation.

In addition to general protocols, the system provides three assessment protocols for the **cervical spine** (CP, SCM or Traps) and for the **lumbar spine**:

- Flexion/Extension
- Left/Right Lateral Flexion
- Left/Right Rotation


It also provides one assessment protocol for the **anterior knee** (VMO and VL).



This logo on a screen in a protocol indicates that you can use an optional webcam with the screen to record video of the session. You need only connect your webcam to a USB port of your computer and turn it on before you start the session.

Note: *Your computer system must be connected to a video capture device, such as a webcam, and the proper software installed, in order to use this function.*

CLINICAL GUIDE REFERENCE

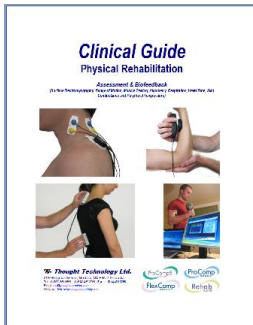

The clinical guide gives general guidelines and recommendations for the accurate use of the instrumentation. It also explains how to prepare the examinee and the procedure to follow.

Chapter: SEMG ASSESSMENT; **section:** DYNAMIC SEMG ASSESSMENT

Also recommended:

- INTRODUCTION TO SURFACE ELECTROMYOGRAPHY
- SKELETAL MUSCLE PROPERTIES

GENERAL PROCEDURE

Starting the Session


If you are going use a webcam to make a video recording of this session, connect your webcam to a USB port of your computer and turn it on.

- Connect the sensors to the encoder as follows:
 - MyoScan/MyoScan-Z sensors to channels A and B (left in A, right in B).
 - Optional MyoScan/MyoScan-Z sensors to channels C and D (left in C, right in D).

A	B	C	D	E	F	G	H	I	J
Myo	Myo	Myo (optional)	Myo (optional)						


- Turn the encoder on.

1. Click **Quick Start**.
2. From **Categories**, select **Pro/Flex Rehab Suite - Dynamic SEMG Assessment**.
3. From **Clients**, select a name.
4. From **Favorites**, select the desired **General Dynamic Assessment Protocol**.
5. Click **OK**.

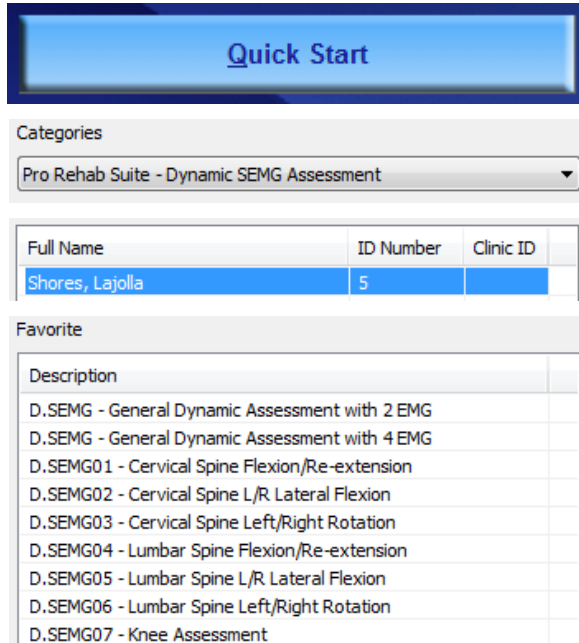


Measuring

- Click the **Start** button .

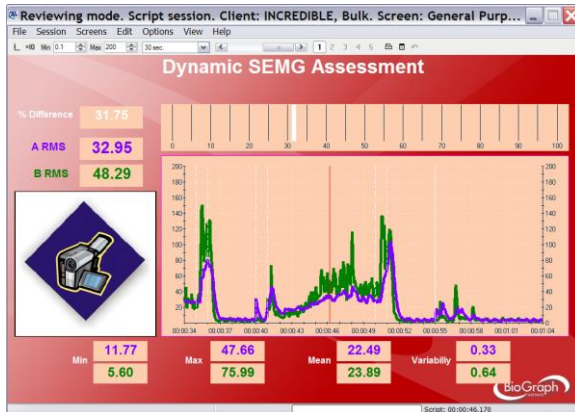

D.EMG – General Dynamic Assessment with 2 EMG

The Multi-line graph will display the EMG signal of 2 channels A and B. The Bar graph indicates the % Difference between the two signals. There are statistics available as well.

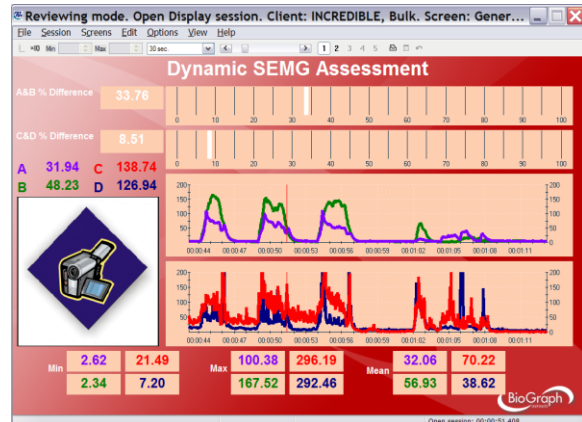

D.EMG – General Dynamic Assessment with 4 EMG


The two Multi-line graph will display the EMG signal of 2 channels simultaneously, respectively A and B, C and D. The Bar graphs indicates the % Difference between the signals (A and B), (C and D). There are statistics available as well.

Mark the different stages of the movement with event makers (events can be marked by hitting the space bar or a key of the keyboard that you would have preliminarily labeled with the name of the event).

Reviewing the session


When the session ends, a series of message prompts guides you through the process of saving data, adding notes to the session file, and switching to reviewing mode so that you can generate a session report.

In review mode, a screen similar to this will appear:

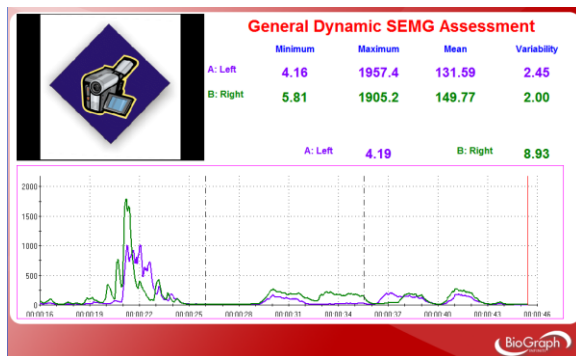

D.EMG – General Dynamic Assessment with 2 EMG

The Multi-line graph will display the EMG signal of 2 channels A and B. The Bar graph indicates the % Difference between the two signals. There are statistics available as well.

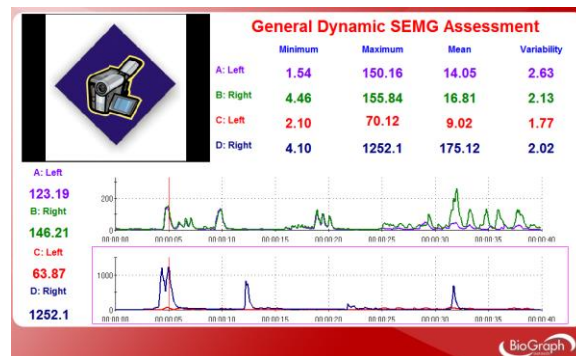


D.EMG – General Dynamic Assessment with 4 EMG

The two Multi-line graph will display the EMG signal of 2 channels simultaneously, respectively A and B, C and D. The Bar graphs indicates the % Difference between the signals (A and B), (C and D). There are statistics available as well.

Select the desired display time window on the screen by using the dropdown list.

It is recommended to select first **Size to session** for a global view of the session and then **Size to activity** to view the different phases of the motion.

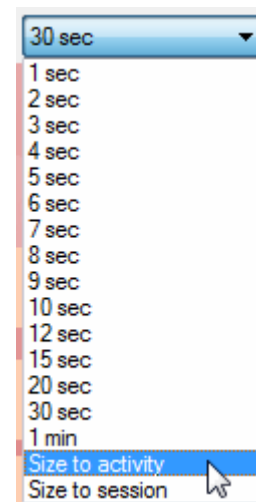


Scroll through the session by using the scroll bar.




You can read the value of a given point on the signal by moving the red Time Mark. The Time Mark is usually placed at the right end of the graph (see picture).

To move the Time Mark:

1. Place the cursor over the Time Mark.
2. Click and hold down the mouse button.
3. Drag the Time Mark to the desired position.

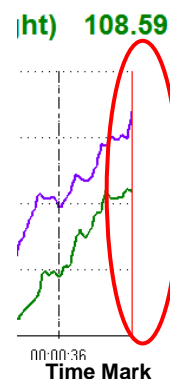

Move the mouse cursor over a marker (vertical dashed line) to display its label (such as flexion, re-extension etc.)

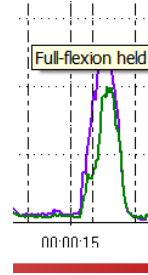


If you used a webcam to make a video recording of the session, during review the video playback is synchronized to the EMG signal as you move the Time Mark back and forth along the EMG signal line graph. This allows you to see lateral deviations in the movement and to observe any antalgic position.

The review allows you to verify that the motion has been properly performed by the examinee at the right time, when prompted. You can also check the consistency from trial to trial.

Generating a report

- After the review screen opens, you can generate a Dynamic SEMG Assessment report. This is a Text report.

- Click the **Session Report** icon  in the tool bar.
- Select **Generate Text Report**, select report components in the **Session Report window**, and click **Generate Report**.

Your report displays as a document in HTML, PDF, or Microsoft Word format, depending upon the setting in **Preferences**. The report can be saved or printed using the **Print** and **Save** functions of the relevant software.

PROCEDURE FOR LUMBAR SPINE ASSESSMENT


Starting the Session

If you are going use a webcam to make a video recording of this session, connect your webcam to a USB port of your computer and turn it on.

- Connect the **MyoScan/MyoScan-Z EMG sensors** to the encoder in **channels A and B** (left in A, right in B).


A	B	C	D	E	F	G	H	I	J
Myo (left)	Myo (right)								

- Turn the encoder on.
 - Click **Quick Start**.
 - From **Categories**, select **Pro/Flex Rehab Suite - Dynamic SEMG Assessment**.
 - From **Clients**, select a name.

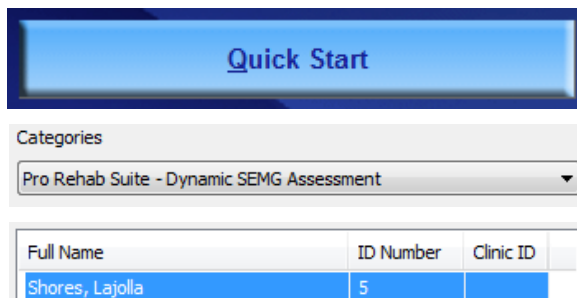


4. From **Favorites**, select the desired **Dynamic SEMG Protocol for Lumbar Spine**.
5. Click **OK**.

Favorite	
Description	
D.SEMG - General Dynamic Assessment with 2 EMG	
D.SEMG - General Dynamic Assessment with 4 EMG	
D.SEMG01 - Cervical Spine Flexion/Re-extension	
D.SEMG02 - Cervical Spine L/R Lateral Flexion	
D.SEMG03 - Cervical Spine Left/Right Rotation	
D.SEMG04 - Lumbar Spine Flexion/Re-extension	
D.SEMG05 - Lumbar Spine L/R Lateral Flexion	
D.SEMG06 - Lumbar Spine Left/Right Rotation	
D.SEMG07 - Knee Assessment	


- Click the **Start** button  and read the instructions on the screen.

First you will see an overview screen (Flexion/Re-extension, shown here):

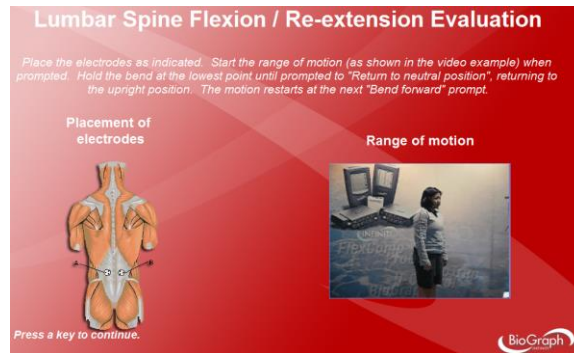

- Press a key to go to the next screen.

In the next screen, verify that the signal is free of artifacts and the sensors properly placed.

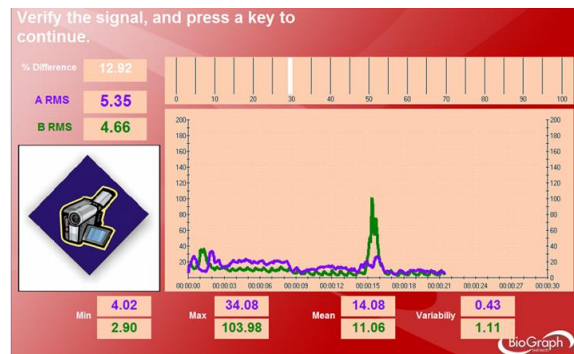

- Before going to the next screen and starting to record, instruct the examinee to perform the motion slowly, until they feel restriction, tightness or discomfort.

Measuring

1. Before performing the motion, the examinee stays in neutral position and relaxes their muscles during 10 seconds.



2. Then the voice prompt asks the examinee to perform the motion three times.




3. At the end of the exercise, the examinee is asked to stay in neutral position and relax their muscles during 10 seconds.



Reviewing the session

When the session ends, a series of message prompts guides you through the process of saving data, adding notes to the session file, and switching to reviewing mode so that you can generate a session report.


In review mode, a screen similar to this will appear:

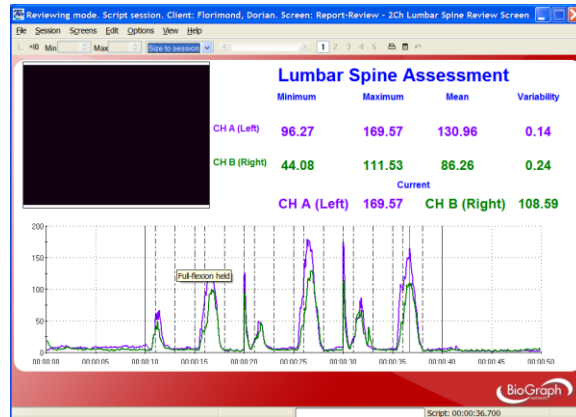

The review allows you to verify that the motion has been properly performed by the examinee at the right time, when prompted. You can also check the consistency from trial to trial.


Generating a report

- After the review screen opens, you can generate a Dynamic SEMG Assessment report. This is an Excel report.

1. Click the **Session Report** icon  in the tool bar.
2. Select **Generate Excel Report**.

Microsoft Excel opens and displays your report as a workbook. You can print or save the report using Excel's **Print** and **Save** functions.

Section 1: Graphs

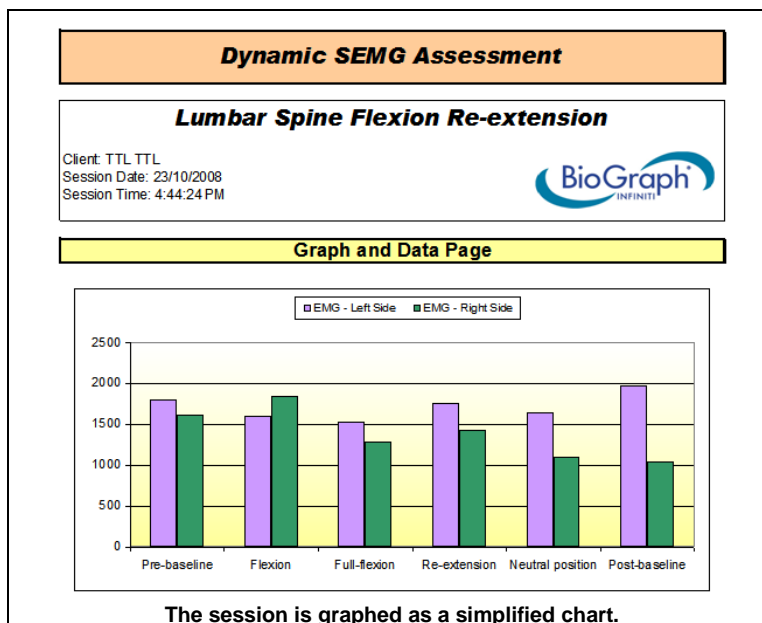

For relaxation phases, such as baselines, neutral position and completed range of motion (full flexion, full rotation etc.), the EMG (RMS) mean is plotted. For motion phases, such as flexion, re-extension or rotation, the EMG (RMS) peak is plotted.

Section 2: Data

BASELINE ANALYSIS	Left side (µV)	Right side (µV)	Difference (Right - Left) (µV)	Average Difference (%)
Pre-baseline average	8.0	5.3	-2.7	33.84%
Flexion peak	71.5	56.6	-15.9	22.26%
Full-Flexion average	5.8	4.3	-1.5	25.48%
Re-extension peak	161.5	114.4	-47.1	29.16%
Neutral position average	4.4	3.4	-1.1	23.94%
Post-baseline average	2.9	3.3	0.4	11.74%

WITHIN-MOTION ANALYSIS	Left side	Right side	Difference
Flexion/re-extension peaks ratios	0.44	0.49	8.88%
Pre/post baselines difference (µV)	5.03	1.94	-3.09

TEMPORAL ANALYSIS	Left side	Right side	Difference (Right - Left)
Time to flexion peak (seconds)	1.35	1.33	-0.03
Time to re-extension peak (seconds)	1.24	1.39	0.16
Pre-baseline variability	0.16	0.15	-0.01
Post-baseline variability	0.57	0.33	-0.23

The data are grouped in order to facilitate the analysis.

The report computes the difference between left and right either in µV or in %.

The formula for µV difference is (right - left).

The formula for percentage difference is (Max(left,right)- Min(left, right))/Max(left,right).

Section 3: Summary and Follow-up notes

You can enter notes in the summary/follow-up notes boxes:

Summary Notes:

Marked improvement

- In **Excel 2000** and **Excel 2003**, place your cursor in the text box and start typing.
- In **Excel 2007**, place your cursor in the text box.
 1. Right-click the mouse button.
 2. Select **Edit text**.
 3. Start typing.



PROCEDURE FOR CERVICAL SPINE ASSESSMENT


Starting the Session

If you are going use a webcam to make a video recording of this session, connect your webcam to a USB port of your computer and turn it on.

- Connect the sensors to the encoder as follows:
 - **MyoScan/MyoScan-Z sensors** positioned on the CPs to channels **A** and **B** (left in A, right in B).
 - **MyoScan/MyoScan-Z sensors** positioned on the SCMs or Traps to channels **C** and **D** (left in C, right in D).

A	B	C	D	E	F	G	H	I	J
Myo (left CP)	Myo (right CP)	Myo (left SCM or Traps)	Myo (right SCM or Traps)						

- Turn the encoder on.
 1. Click **Quick Start**.

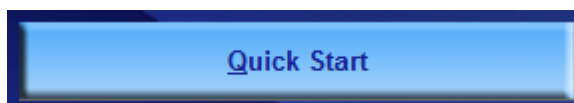

2. From **Categories**, select **Pro/Flex Rehab Suite - Dynamic SEMG Assessment**.

Categories
 Pro Rehab Suite - Dynamic SEMG Assessment

3. From **Clients**, select a name.


Full Name	ID Number	Clinic ID
Shores, Lajolla	5	

4. From **Favorites**, select the desired **Dynamic SEMG Protocol for Cervical Spine**.


Favorite

Description
D.SEMG - General Dynamic Assessment with 2 EMG
D.SEMG - General Dynamic Assessment with 4 EMG
D.SEMG01 - Cervical Spine Flexion/Re-extension
D.SEMG02 - Cervical Spine L/R Lateral Flexion
D.SEMG03 - Cervical Spine Left/Right Rotation
D.SEMG04 - Lumbar Spine Flexion/Re-extension
D.SEMG05 - Lumbar Spine L/R Lateral Flexion
D.SEMG06 - Lumbar Spine Left/Right Rotation
D.SEMG07 - Knee Assessment

5. Click **OK**.


- Click the **Start** button  and read the instructions on the screen.

First you will see an overview screen (Flexion/Re-extension, shown here):

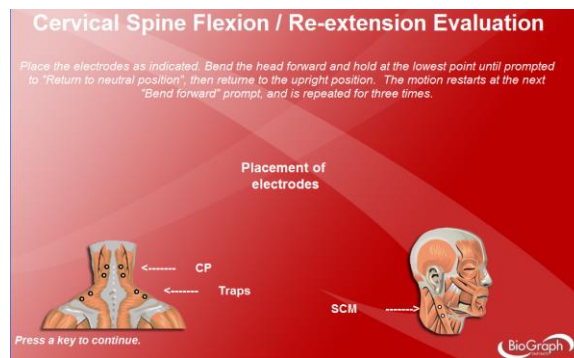

- Press a key to go to the next screen.

In the next screen, verify that the signal is free of artifacts and the sensors properly placed.

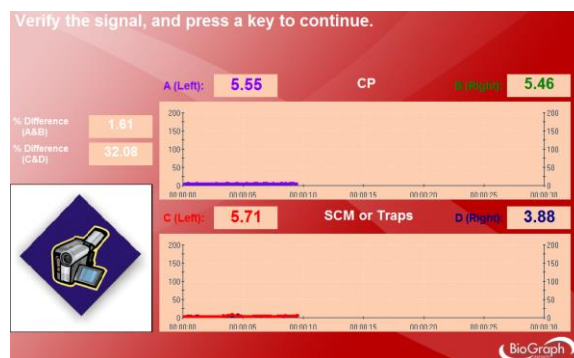


- Before going to the next screen and start recording, instruct the examinee to perform the motion slowly, until they feel restriction, tightness or discomfort.

Measuring


- Before performing the motion, the examinee stays in neutral position and relaxes their muscles during 10 seconds.



- Then the voice prompt ask the examinee to perform the motion three times.

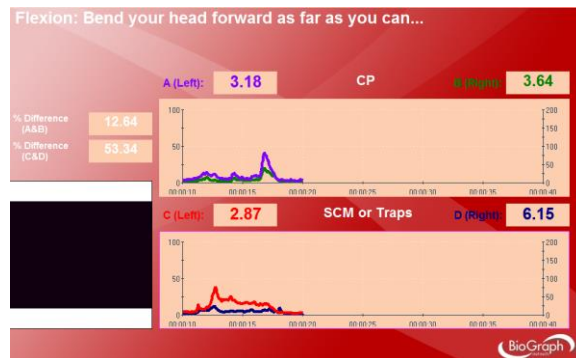


- At the end of the exercise, the examinee is asked to stay in neutral position and relax their muscles during 10 seconds.

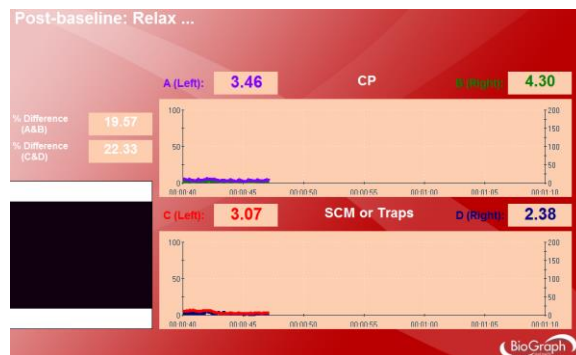

Reviewing the session

- When the session ends, a series of message prompts guides you through the process of saving data, adding notes to the session file, and switching to reviewing mode so that you can generate a session report.


In review mode, a screen similar to this will appear:

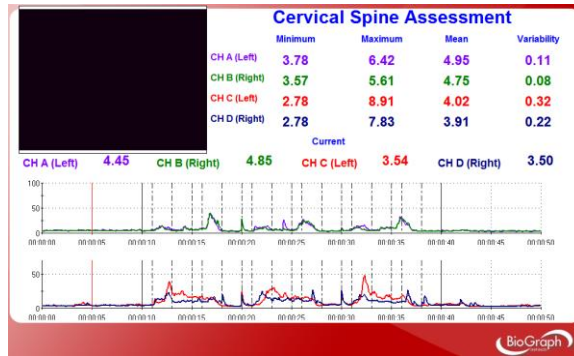

The review allows you to verify that the motion has been properly performed by the examinee at the right time, when prompted. You can also check the consistency from trial to trial.


Generating a report

- After the review screen opens, you can generate a Dynamic SEMG Assessment report. This is an Excel report.

- Click the **Session Report** icon  in the tool bar.
- Select **Generate Excel Report**.

Microsoft Excel opens and displays your report as a workbook. You can print or save the report using Excel's **Print** and **Save** functions.


Section 1: Graphs

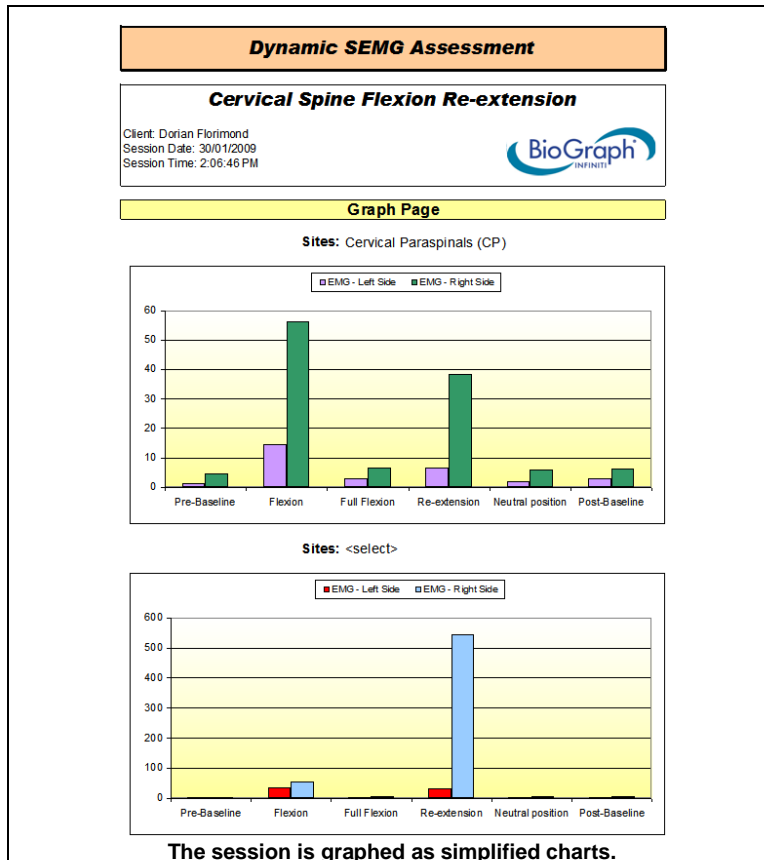

For relaxation phases, such as baselines, neutral position and completed range of motion (full flexion, full rotation etc.), the EMG (RMS) mean is plotted. For motion phases, such as flexion, re-extension or rotation, the EMG (RMS) peak is plotted.

The first graph shows CP. The second graph shows the second pair of monitored sites. They can be either SCM or traps.

In order to select the sites, you have to click on "<select>".

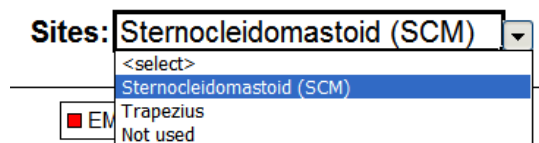

The legend on the next pages will be automatically updated.

Section 2: Data

Data Page				
Cervical Paraspinals (CP)				
PARALLEL SYMMETRY ANALYSIS	Left side (µV)	Right side (µV)	Difference (Right - Left) (µV)	Average Difference (%)
Pre-baseline average	5.2	5.2	0.0	0.08%
Flexion peak	16.9	15.5	-1.4	8.06%
Full-Flexion average	8.5	8.0	-0.5	6.00%
Re-extension peak	18.5	23.0	4.5	19.59%
Neutral position average	4.4	4.5	0.1	2.85%
Post-baseline average	4.0	3.9	-0.1	2.93%

WITHIN-MOTION ANALYSIS	Left side	Right side	Difference
Flexion/re-extension peaks ratios	1.31	1.24	5.69%
Prepost baselines difference (µV)	0.02	-1.57	-1.58

TEMPORAL ANALYSIS	Left side	Right side	Difference (Right - Left)
Time to flexion peak (seconds)	2.10	2.24	0.13
Time to re-extension peak (seconds)	0.90	0.82	-0.08
Pre-baseline variability	0.15	0.10	-0.05
Post-baseline variability	0.15	0.08	-0.07

Sternocleidomastoid (SCM)				
PARALLEL SYMMETRY ANALYSIS	Left side (µV)	Right side (µV)	Difference (Right - Left) (µV)	Average Difference (%)
Pre-baseline average	1.9	2.1	0.2	10.46%
Flexion peak	7.1	13.9	6.8	49.10%
Full-Flexion average	3.4	9.8	6.4	65.24%
Re-extension peak	5.4	11.3	5.9	52.00%
Neutral position average	3.1	6.8	3.7	54.65%
Post-baseline average	1.9	3.7	1.8	48.81%

The data are grouped in order to facilitate the analysis.

The report computes the difference between left and right either in µV or in %.

The formula for µV difference is (right - left) .

The formula for percentage difference is (Max(left,right)- Min(left, right))/Max(left,right).

Section 3: Summary and Follow-up notes

You can enter notes in the summary/follow-up notes boxes:

Summary Notes:

Marked improvement

- In **Excel 2000** and **Excel 2003**, place your cursor in the text box and start typing.
- In **Excel 2007**, place your cursor in the text box.
 1. Right-click the mouse button.
 2. Select **Edit text**.
 3. Start typing.



PROCEDURE FOR ANTERIOR KNEE ASSESSMENT

Starting the Session

If you are going use a webcam to make a video recording of this session, connect your webcam to a USB port of your computer and turn it on.

- Connect the **MyoScan/MyoScan-Z EMG sensors** to the encoder in **channels A and B** (A for VMO, B for VL).


A	B	C	D	E	F	G	H	I	J
Myo (left)	Myo (right)								

- Turn the encoder on.


1. Click **Quick Start**.




2. From **Categories**, select **Pro/Flex Rehab Suite - Dynamic SEMG Assessment**.

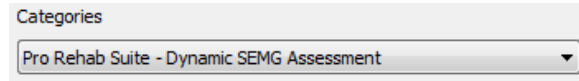


3. From **Clients**, select a name.

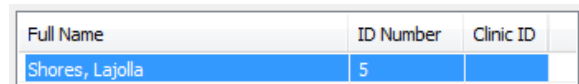


4. From **Favorites**, select the desired **Dynamic SEMG Protocol for Knee Assessment**.

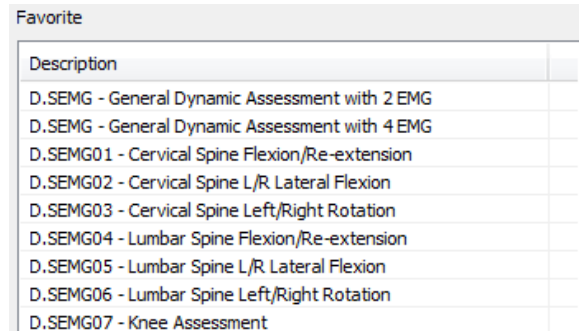

5. Click **OK**.


- Click the **Start** button  and read the instructions on the screen.

First you will see an overview screen:

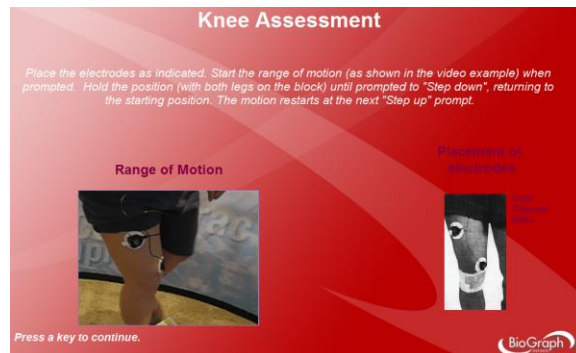

- Press a key to go to the next screen.

In the next screen, verify that the signal is free of artifacts and the sensors properly placed.

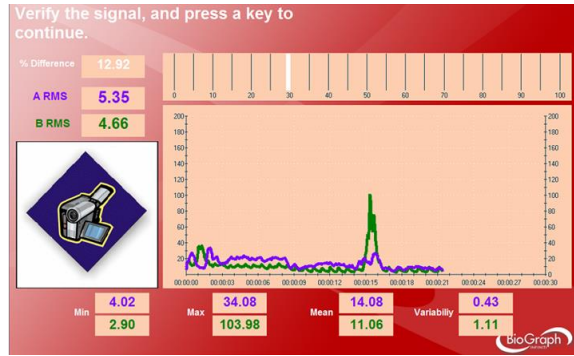


- Before going to the next screen and starting to record, instruct the examinee to perform the motion slowly, until they feel restriction, tightness or discomfort.

Measuring


1. Before performing the motion, the examinee stays in neutral position and relaxes their muscles during 10 seconds.



2. Then the voice prompt asks the examinee to perform the motion three times.

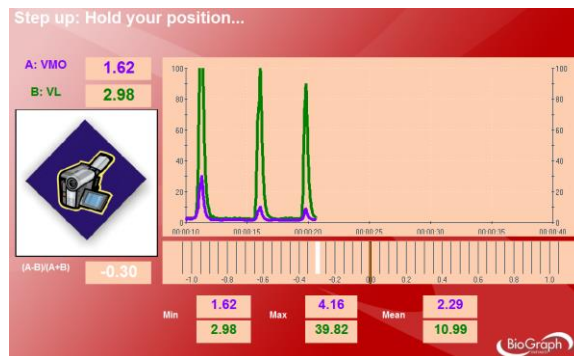


3. At the end of the exercise, the examinee is asked to stay in neutral position and relax their muscles during 10 seconds.

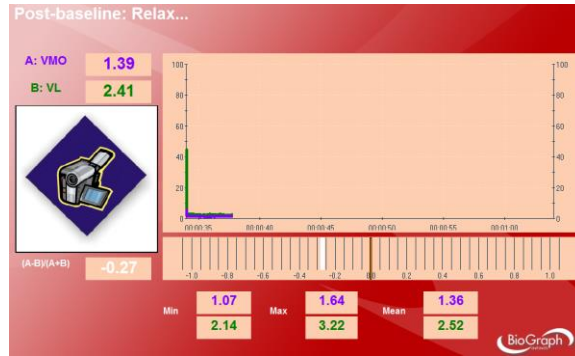

When the session ends, a series of message prompts guides you through the process of saving data, adding notes to the session file, and switching to reviewing mode so that you can generate a session report.

Reviewing the session


In review mode, a screen similar to this will appear:

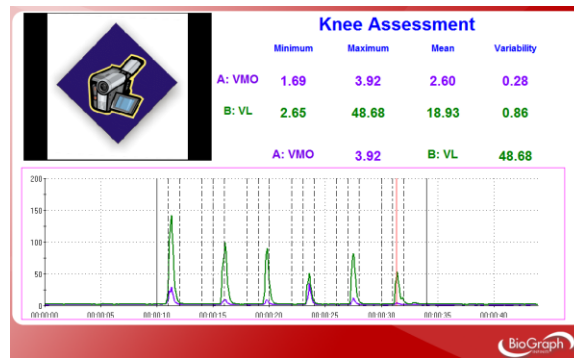

The review allows you to verify that the motion has been properly performed by the examinee at the right time, when prompted. You can also check the consistency from trial to trial.


Generating a report

- After the review screen opens, you can generate a Dynamic SEMG Assessment report. This is an Excel report.

- Click the **Session Report** icon  in the tool bar.
- Select **Generate Excel Report**.

Microsoft Excel opens and displays your report as a workbook. You can print or save the report using Excel's **Print** and **Save** functions.

Section 1: Graphs

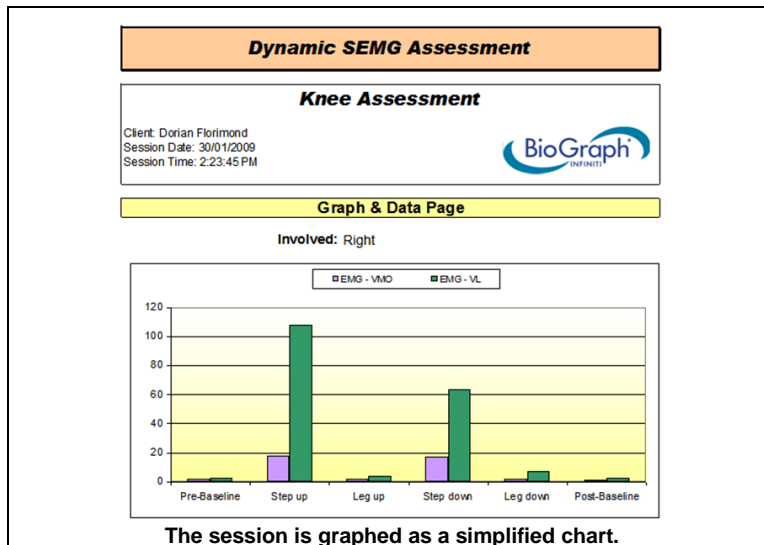

For relaxation phases, such as baselines and neutral position (leg up and leg down), the EMG (RMS) mean is plotted. For motion phases, (step up and step down), the EMG (RMS) peak is plotted.

In order to select the involved side, you have to click on "Involved".



Section 2: Data

PARALLEL SYMMETRY ANALYSIS	VMO (µV)	VL (µV)
Pre-baseline average	1.6	2.5
Step up	17.9	107.8
Leg up	1.7	3.8
Step down	17.1	63.4
Leg down	2.0	7.3
Post-baseline average	1.4	2.5
Pre/post baselines difference (µV)	0.26	0.06

PARALLEL SYMMETRY ANALYSIS	VMO/VL
During Step-up	0.2
During Step-down	0.3

The data are grouped in order to facilitate the analysis.

The report computes the difference between pre and post baselines and the ratio VMO/VL EMG peaks during the motion phases.

Section 3: Summary and Follow-up notes

You can enter notes in the summary/follow-up notes boxes:

Summary Notes:

Marked improvement

- In **Excel 2000** and **Excel 2003**, place your cursor in the text box and start typing.
- In **Excel 2007**, place your cursor in the text box.
 1. Right-click the mouse button.
 2. Select **Edit text**.
 3. Start typing.



Muscle Fatigue Monitoring

OVERVIEW


SEMG can be used as an indicator of muscle fatigue. The following screens were designed for this purpose.



This logo on a screen in a protocol indicates that you can use an optional webcam with the screen to record video of the session. You need only connect your webcam to a USB port of your computer and turn it on before you start the session.

Note: Your computer system must be connected to a video capture device, such as a webcam, and the proper software installed, in order to use this function.

CLINICAL GUIDE REFERENCE

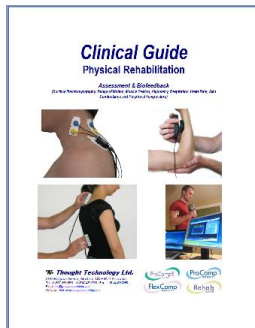

The clinical guide gives general guidelines and recommendations for the accurate use of the instrumentation. It also explains how to prepare the examinee and the procedure to follow.

Chapter: SEMG ASSESSMENT; **section:** MUSCLE FATIGUE MONITORING

Also recommended:

- INTRODUCTION TO SURFACE ELECTROMYOGRAPHY
- SKELETAL MUSCLE PROPERTIES

GENERAL PROCEDURE


Starting the Session

If you are going use a webcam to make a video recording of this session, connect your webcam to a USB port of your computer and turn it on.

- Connect the sensors to the encoder as follows:
 - MyoScan/MyoScan-Z sensor to channels A.
 - Optional MyoScan/MyoScan-Z sensor to channel B.

A	B	C	D	E	F	G	H	I	J
Myo	Myo (optional)								

- Turn the encoder on.
 1. Click **Quick Start**.
 2. From **Categories**, select **Pro/Flex Rehab Suite – Muscle Fatigue**.

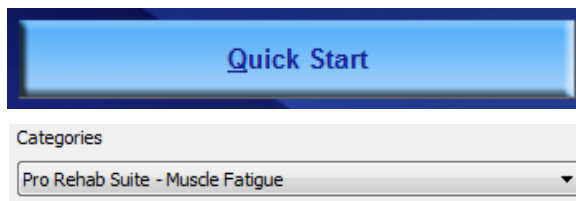

3. From **Clients**, select a name.

Full Name	ID Number	Clinic ID
Shores, Lajolla	5	

4. From **Favorites**, select **Muscle Fatigue Monitoring**.

Favorite	
Description	
Muscle Fatigue Monitoring with 2 EMG	


5. Click **OK**.




Measuring

- Click the **Start** button .

You can view each screen by clicking on the numerical buttons on the toolbar at the top.



- To stop the session, click the **Stop** button: .
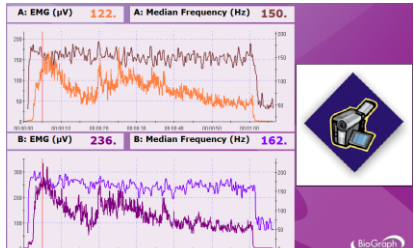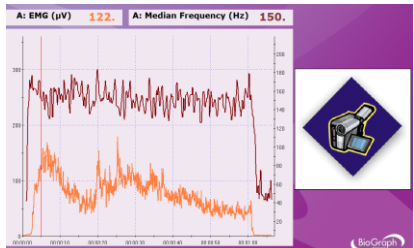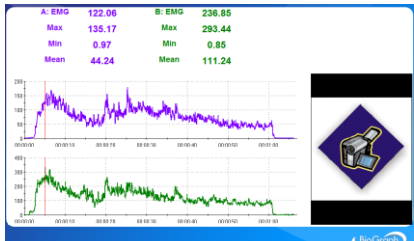
<p>Muscle Fatigue Monitoring on A & B</p> <p>1 2 3 4 5</p> <p>For each signal, RMS EMG and median frequency are plotted on the same line graph. Video can also be recorded.</p>													
<p>Muscle Fatigue Monitoring on A</p> <p>1 2 3 4 5</p> <p>This screen is similar to the one above, but only displays channel A.</p>													
<p>Report - Review – 2Ch 2 Line Graphs</p> <p>1 2 3 4 5</p> <p>The two RMS EMG signals are displayed on line graphs. Statistics are also available.</p>	 <table border="1" data-bbox="1008 1281 1419 1354"> <thead> <tr> <th colspan="2">A: EMG (µV)</th> <th colspan="2">B: EMG (µV)</th> </tr> </thead> <tbody> <tr> <td>122.06</td> <td>135.17</td> <td>236.85</td> <td>293.44</td> </tr> <tr> <td>0.97</td> <td>44.24</td> <td>0.85</td> <td>111.24</td> </tr> </tbody> </table>	A: EMG (µV)		B: EMG (µV)		122.06	135.17	236.85	293.44	0.97	44.24	0.85	111.24
A: EMG (µV)		B: EMG (µV)											
122.06	135.17	236.85	293.44										
0.97	44.24	0.85	111.24										

Review screen

The same screens are used for review.

All these screens are accessible from **Start Open Display Session** in the channel set **“SEMG Assessment with 2 EMG”**.

SEMG Biofeedback

OVERVIEW


This section suggests several training screens that will enhance and speed up rehabilitation thanks to SEMG biofeedback.



This logo on a screen in a protocol indicates that you can use an optional webcam with the screen to record video of the session. You need only connect your webcam to a USB port of your computer and turn it on before you start the session.

Note: Your computer system must be connected to a video capture device, such as a webcam, and the proper software installed, in order to use this function.

CLINICAL GUIDE REFERENCE

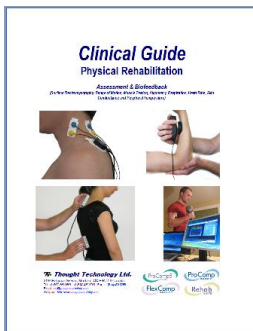

The clinical guide gives general guidelines and recommendations for the accurate use of the instrumentation. It also explains several biofeedback training techniques.

Chapter: SEMG BIOFEEDBACK TRAINING

Also recommended:

- INTRODUCTION TO SURFACE ELECTROMYOGRAPHY
- SKELETAL MUSCLE PROPERTIES

GENERAL PROCEDURE


Starting the Session

If you are going use a webcam to make a video recording of this session, connect your webcam to a USB port of your computer and turn it on.

- Connect the sensors to the encoder as follows:
 - MyoScan/MyoScan-Z sensor to channel A.
 - Optional MyoScan/MyoScan-Z sensors to channel B to D, and channels E to J for FlexComp Infinity.

A	B	C	D	E	F	G	H	I	J
Myo	Myo (optional)	Myo (optional)	Myo (optional)	Myo (optional for FlexComp)	Myo (optional for FlexComp)	Myo (optional for FlexComp)	Myo (optional for FlexComp)	Myo (optional for FlexComp)	Myo (optional for FlexComp)

- Turn the encoder on.
 1. Click **Quick Start**.
 2. From **Categories**, select one of the categories starting with “**Pro/Flex Rehab Suite – SEMG Biofeedback –**”.

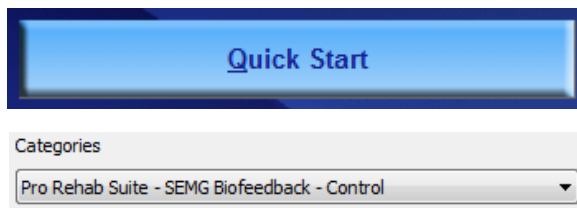

3. From **Clients**, select a name.

Full Name	ID Number	Clinic ID
Shores, Lajolla	5	


4. From **Favorites**, select the desired **SEMG Biofeedback Protocol**.

Favorite	
Description	
RELAX01 - Relaxation with 1 EMG	
RELAX02 - Relaxation with 1 EMG	
RELAX03 - Relaxation with 2 EMG	

5. Click **OK**.

• Click the **Start** button .

You can view each screen by clicking on the numerical buttons on the toolbar at the top.

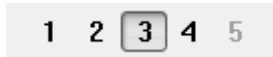

• If the feedback is **scale dependent**, you can adjust the sensitivity of the animation or graph as follow:




Min 0 Max 180

Select the animation by clicking on it. A thin red frame will appear around it.


Type the desired EMG scale in **Min** and **Max**. Then click anywhere on the screen.

• If the feedback is **threshold-dependent**: The threshold line on a bar graph, line graph and animation can be adjusted in the same way as the vertical scale. The text box labeled **Thr1** is for single threshold graphs, while **Thr2** is for double threshold graphs like the multi-line graph that can have up to two guidelines.

Max 1.1 Thr1 0 Thr2

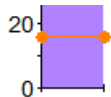

You can also directly move the threshold by placing the mouse cursor on the threshold line, pressing and holding the mouse left button and moving the cursor up and down.

• To stop the session, click the **Stop** button: .

CATEGORY: PRO/FLEX SUITE – SEMG BIOFEEDBACK – RELAXATION

These screens are designed for muscle deactivation training and, ultimately, total relaxation.


RELAX01 - Relaxation with 1 EMG

Relaxation - 1Ch Line Graph



- Scale-dependent
- Threshold-dependent
- Other

Music is played when the channel A signal stays below the threshold.

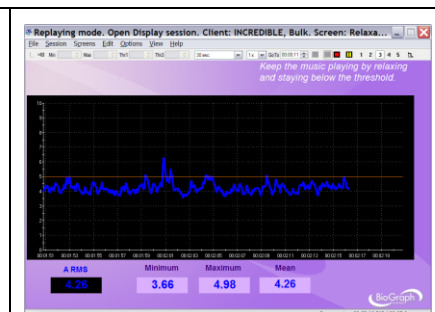


Relaxation - 1Ch Relaxation Bar Graph

1 2 3 4 5

- Scale-dependent
- Threshold-dependent
- Other

Music is played when the channel A signal stays below the threshold.

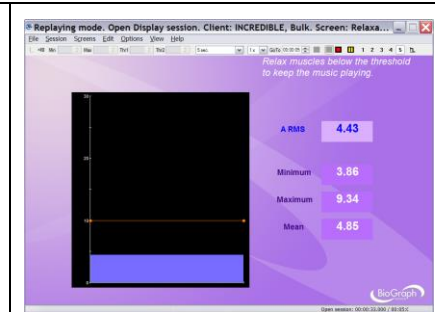


Relaxation - 1Ch Smiley

1 2 3 4 5

- Scale-dependent
- Threshold-dependent
- Other

The face will smile when the channel A signal is below the threshold.

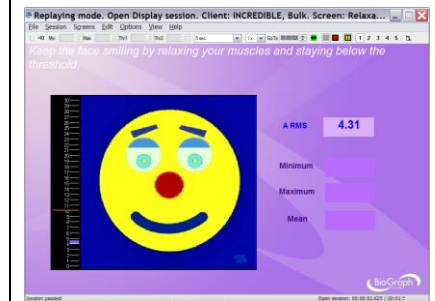

Relaxation - 1Ch Knee Flexion

1 2 3 4 5

- Scale-dependent
- Threshold-dependent
- Other

When the signal is below the threshold, the leg relaxes; when above, it straightens.




Relaxation - 1Ch Wrist Flexion

1 2 3 4 5

- Scale-dependent
- Threshold-dependent
- Other

When the signal is below the threshold, the wrist relaxes; when above, it straightens.

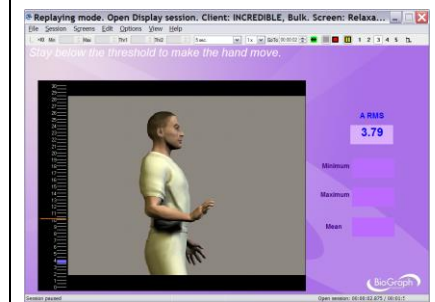

RELAX02 - Relaxation with 1 EMG


Relaxation - 1Ch Filled Line-Bar Graphs

1 2 3 4 5

- Scale-dependent
- Threshold-dependent
- Other

Music is played when the channel A signal stays below the line graph threshold.

The signal is displayed in two different views: filled line graph and bar graph.

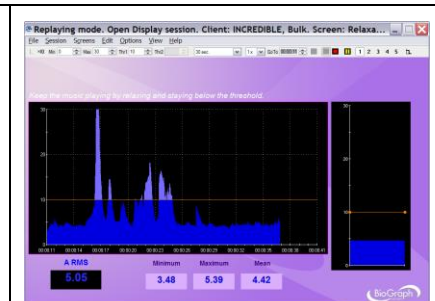

Relaxation - 1 Ch Parrot Puzzle

1 2 3 4 5

- Scale-dependent
- Threshold-dependent
- Other

If the EMG reading is below the threshold for 10 seconds then the puzzle starts to fill in. If the signal goes above, pieces will disappear.




Relaxation - 1Ch Growing Fractal

1 2 3 4 5

- Scale-dependent
- Threshold-dependent
- Other

This display assists a patient to differentiate between contracting and relaxing their muscles. Set the animation scale to a maximum value that is appropriate for a low sub-maximal contraction. Set the animation threshold in the middle of this scale. As the patient sustains a sub-maximal contraction and the EMG activity goes above the threshold, the fractal will fill in. As the patient releases the contraction and the EMG activity falls below its threshold the fractal will slowly open and a relaxing song is heard. The complete animation cycle takes approximately 20 seconds, 10 on each side of the threshold.

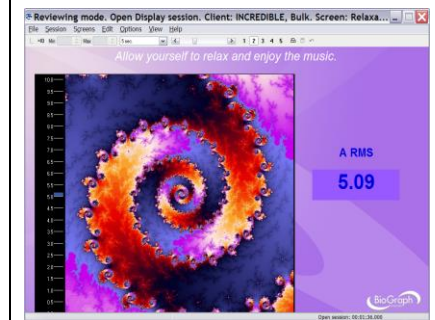


Relaxation - 1Ch Space Hoops

1 2 3 4 5

- Scale-dependent
- Threshold-dependent
- Other

The animation moves when the channel A signal is below the threshold.

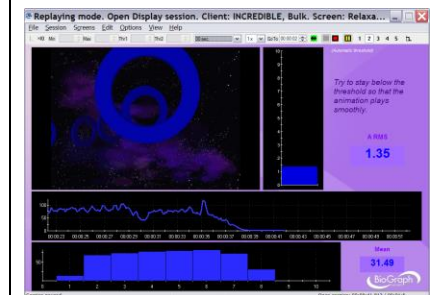


Relaxation - 1Ch DVD

1 2 3 4 5

- Scale-dependent
- Threshold-dependent
- Other

The channel A signal must stay below the bar graph's threshold to keep the DVD screen size constant.

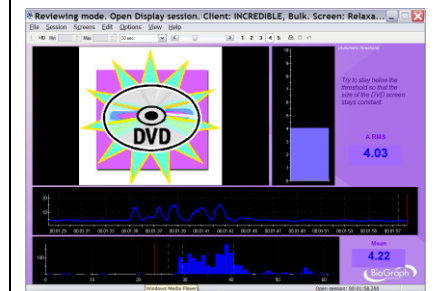

RELAX03 - Relaxation with 2 EMG


Relaxation - 2Ch Shrinking Heads

1 2 3 4 5

- Scale-dependent
- Threshold-dependent
- Other

Music is played when both channels A and B (bar graphs on the right) are below the threshold. The two heads must have the same size, which means both signals must be at the same level.

Channel B could be connected to the healthy site, in order to use it as a model for the unhealthy site.

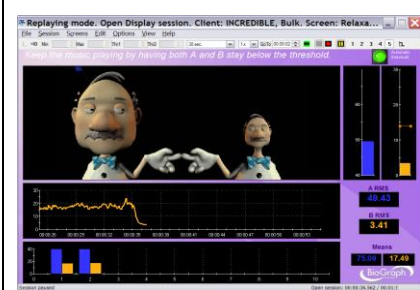


Relaxation - 2Ch Relaxation with DVD

1 2 3 4 5

- Scale-dependent
- Threshold-dependent
- Other

The DVD will resume playing when both channels A and B are below the threshold.

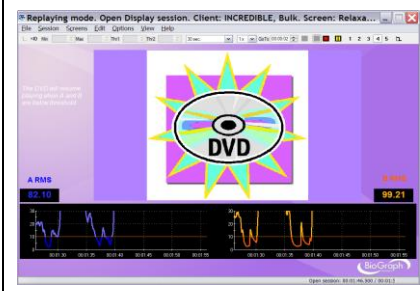

CATEGORY: PRO/FLEX SUITE – SEMG BIOFEEDBACK – STRENGTHENING

These screens are designed for muscle activation training and ultimately strengthening. The scale should be adjusted according to the maximal force and the threshold to the training goal.

STRN01 - Strengthening with 1 EMG

The three screens show a classic view of the signal with bar graphs and line graphs.


Strengthening - 1Ch Bar Graph

1 2 3 4 5

- Scale-dependent
- Threshold-dependent
- Other

This screen graphs the channel A signal on a bar graph and also displays statistics.

A sound plays when the signal goes above the threshold.

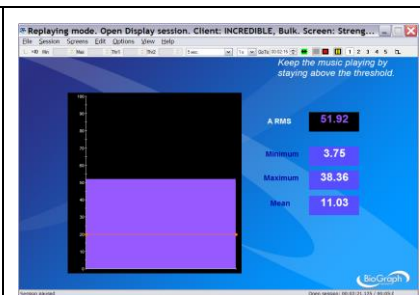

Strengthening - 1Ch Filled Line Graph

1 2 3 4 5

- Scale-dependent
- Threshold-dependent
- Other

The signal changes color, and bolero music plays, when channel A goes above the threshold.




Strengthening - 1Ch Filled Line-Bar Graphs

1 2 3 4 5

- Scale-dependent
- Threshold-dependent
- Other

The signal changes color, and music plays, when channel A goes above the line graph threshold.

The bar graph also displays the EMG levels in real time.

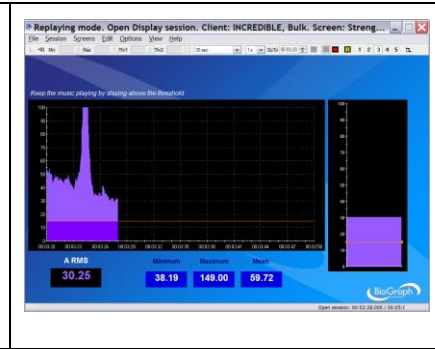

STRN02 – Contract and Hold (with 1 EMG)


The four next screens provide the patient with more interesting feedback. Each screen requires the patient to hold the contraction for a longer period of time.

Strengthening - 1Ch Smiley

1 2 3 4 5

- Scale-dependent
- Threshold-dependent
- Other

The face will continue smiling as long as the contraction on channel A is being held above threshold.

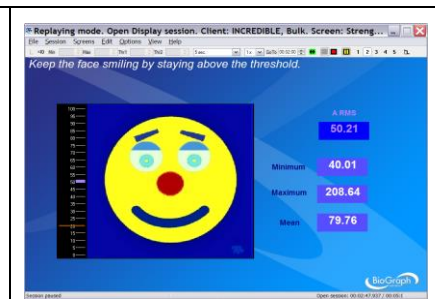

Strengthening - 1Ch Rooster Puzzle

1 2 3 4 5

- Scale-dependent
- Threshold-dependent
- Other

The puzzle will fill when the contraction on channel A has been held above the threshold for more than 3 seconds. If the contraction dips below the threshold, then the timer will reset.

The threshold is also indicated by the Tarantella tune and can be set on the bar graph instrument.



Strengthening - 1Ch Flower Puzzle

1 2 3 4 5

- Scale-dependent
- Threshold-dependent
- Other

The puzzle will fill when the contraction on channel A has been held above the threshold for more than 5 seconds. If the contraction dips below the threshold, then the timer will reset.

The threshold is also indicated by a jazz tune and set on the bar graph.




Strengthening - 1Ch Dolphin Puzzle

1 2 3 4 5

- Scale-dependent
- Threshold-dependent
- Other

The puzzle will fill when the contraction on channel A has been held above the threshold for more than 10 seconds. If the contraction dips below the threshold, then the timer will reset.

The threshold is also indicated by a harpsichord sound and can be set on the bar graph instrument.

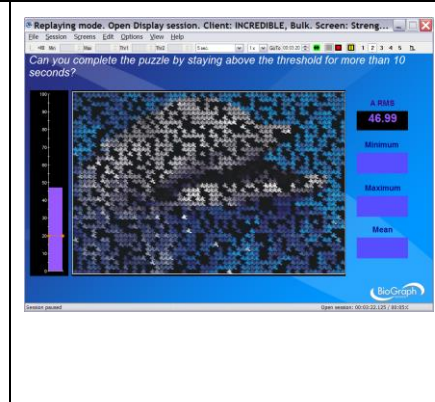

STRN03 – Uprain A / Downtrain B (with 2 EMG)

The four next screens are more challenging, involving two muscles. Channel A is used for the muscle that must be activated, while channel B is used for the muscle that must not be activated.


Strengthening - 2Ch Tomato Puzzle

1 2 3 4 5

- Scale-dependent
- Threshold-dependent
- Other

The signal displayed on channel A must exceed the threshold and channel B must be below the threshold for 5 seconds in order for the puzzle to be revealed.

Basic statistics are also displayed to give a more detailed picture.

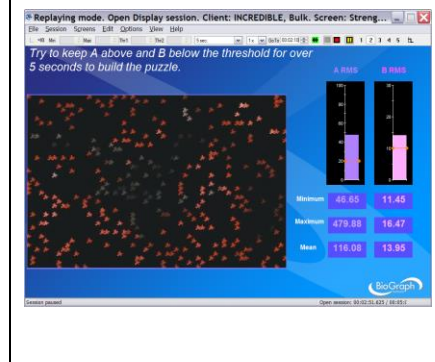

Strengthening - 2Ch Hero Morph-Slow

1 2 3 4 5

- Scale-dependent
- Threshold-dependent
- Other

The animation is controlled by both channels A and B, and is threshold dependent. The signal A goes above its threshold, and the signal B stays below its threshold, the boy slowly morphs into a superman. The complete animation cycle is about 14 seconds. An audio tone is heard when the signal A is above the threshold. If one of the conditions is not satisfied, the animation resets to the beginning. The animation can also be reset manually.




Strengthening - 2Ch Car Race

1 2 3 4 5

- Scale-dependent
- Threshold-dependent
- Other

The aim is to make the blue car (channel A) pass the finish line (the white bar on the far right of each track) before the yellow car (channel B). To meet this goal, A must stay above its threshold while B stays below. The points for A increment when both channels are in condition, and the points for B increment when B is above threshold.

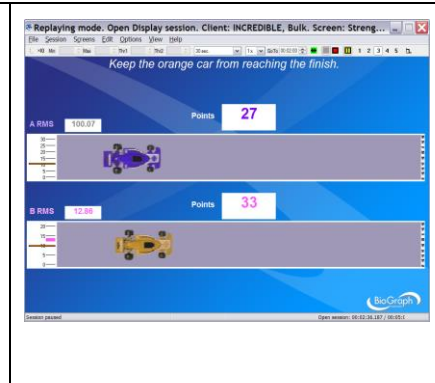


Strengthening - 2Ch Conditional DVD

1 2 3 4 5

- Scale-dependent
- Threshold-dependent
- Other

The DVD stays on when the channel A signal is above its threshold and channel B signal stays below. If either condition is not met the DVD stops playing.

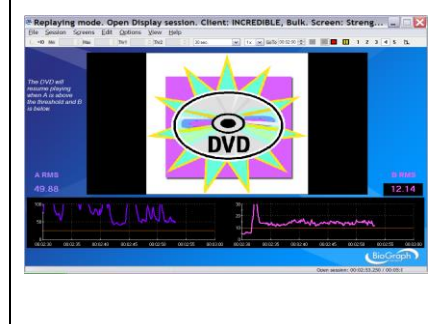

STRN04 – Uprain A / Downtrain B Specific (with 2 EMG)


The two next screens are specific to a given joint.

Strengthening - 2Ch Knee Flexion

1 2 3 4 5

- Scale-dependent
- Threshold-dependent
- Other

Control over both channels during the movement is required to move the animation. The threshold can be set on the bar graphs to make it progressively easier or harder to trigger the animation.

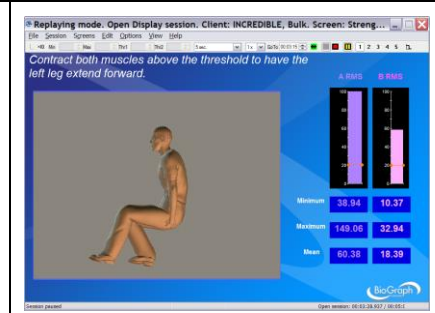


Strengthening - 2Ch Wrist Flexion

1 2 3 4 5

- Scale-dependent
- Threshold-dependent
- Other

Control over both channels during the movement is required to move the animation. The threshold can be set on the bar graphs to make it progressively easier or harder to trigger the animation.

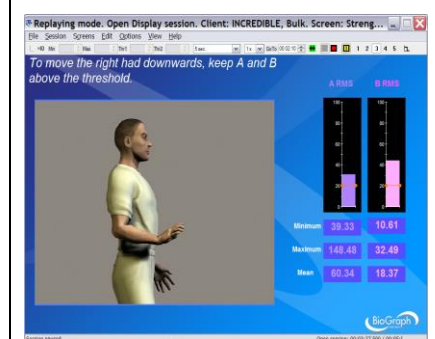

CATEGORY: PRO/FLEX SUITE – SEMG BIOFEEDBACK – CONTROL

These screens are designed for muscle control training. The scale should be adjusted according to the maximal force. Channel B is used to train the patient not to activate a second muscle while activating the first one. The threshold of channel B should be set at a small value above the resting baseline.


CTRL01 - Control with 1 EMG

Control - 1Ch Tubes

1 2 3 4 5

- Scale-dependent
- Threshold-dependent
- Other

The animation represents channel A and is dependent on the scale on the left. The ball climbs the tubes when the signal goes up. Instruct your patient to move the ball to a given tube color.

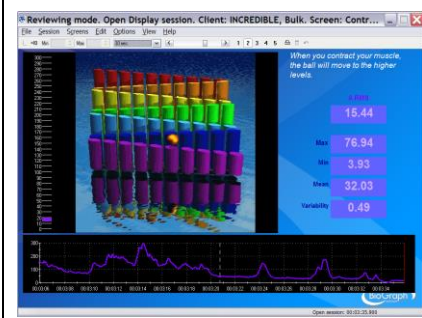

Control - 1Ch Tension Discrimination Training Level 1

1 2 3 4 5

Control - 1Ch Tension Discrimination Training Level 2


1 2 3 4 5

Control - 1Ch Tension Discrimination Training Level 3

1 2 3 4 5

- Scale-dependent
- Threshold-dependent
- Other

Each screen contains a template to follow. Three levels of difficulty are available. You can also adjust the level of difficulty by adjusting the graph scale. These screens are designed for tension discrimination training and for muscle contraction control.

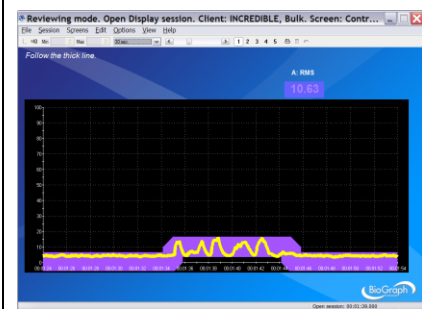


CTRL02 - Control with 2 EMG

Control - 2Ch Hero Morph-Fast

1 2 3 4 5

- Scale-dependent
- Threshold-dependent
- Other

The animation represents channel A and is dependent on the scale on the left. The boy morphs into a superman when the channel A signal is toward the upper range of the scale. As the signal comes down the scale, the superman returns to a boy.

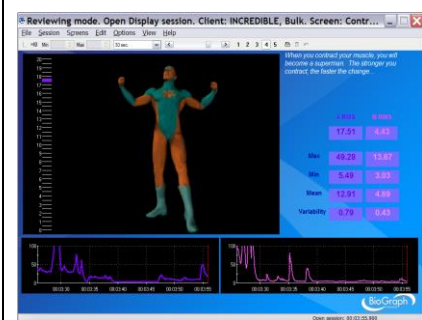


Control - 2Ch Animal Game

1 2 3 4 5

- Scale-dependent
- Threshold-dependent
- Other

An exercise to control muscle contraction by lining up the cartoon man with the animal in the blue square while the line-up of animals constantly changes. Channel A is connected to the animation. The stronger the contraction, the further the man moves to the right. To keep the man moving, the signal from channel B should remain below its threshold.

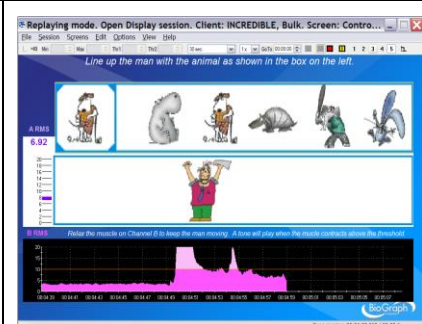

CATEGORY: PRO/FLEX SUITE – SEMG BIOFEEDBACK – EQUILIBRATION

These screens are designed for equilibration training. Equilibration refers to bringing muscles into equilibrium.

EQUL01 - Equilibration with 2 EMG


Equilibration - 2Ch Balance Ratio

1 2 3 4 5

- Scale-dependent
- Threshold-dependent
- Other

This two-channel ratio screen easily communicates the interplay of two muscles. When the muscles are in equilibrium, the weight is in the middle of the balance. When the weight is off to one end or the other, the muscles are progressively more out of balance.

The bar graphs are connected to channels A, B.

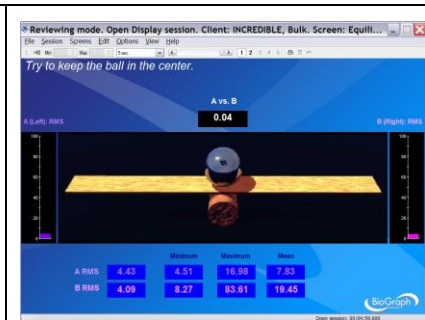


Equilibration - 2Ch Gorilla Ratio

1 2 3 4 5

- Scale-dependent
- Threshold-dependent
- Other

This two-channel ratio screen easily communicates the interplay of two muscles. When the muscles are in equilibrium, the ball is balanced on the gorilla's shoulders. When the ball is off to one end or the other, the muscles are progressively more out of balance.

The bar graphs are connected to channels A, B.

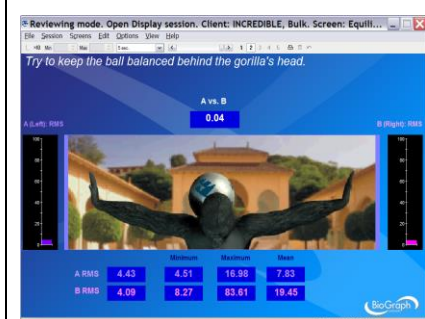


Equilibration - 2Ch Bi-Lateral Bar-Video

1 2 3 4 5

- Scale-dependent
- Threshold-dependent
- Other

The light turns red when the difference is greater than 35%. Both signals are also displayed in the same line graph and mirrored bars for comparison.

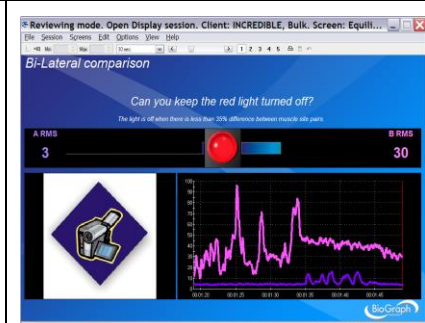


Equilibration - 2Ch Bi-Lateral Bar-Video 2

1 2 3 4 5

- Scale-dependent
- Threshold-dependent
- Other

The light turns red when the difference is greater than 35%. Both signals are also displayed in the same line graph and bars for comparison.

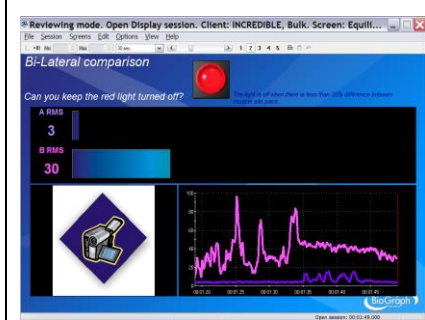


EQU02 - Equilibration with 4 EMG

Equilibration - 4Ch Balance Ratio

1 2 3 4 5

- Scale-dependent
- Threshold-dependent
- Other

This four-channel ratio screen easily communicates the interplay of four muscles. When the muscles are in equilibrium, the weight is in the middle of the balance. When the weight is off to one end or the other, the muscles are progressively more out of balance. The bar graphs will be connected to channels A, B, C, D.

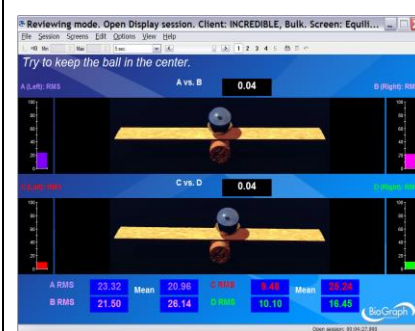

Equilibration - 4Ch Gorilla Ratio

1 2 3 4 5

- Scale-dependent
- Threshold-dependent
- Other

This four-channel ratio screen easily communicates the interplay of four muscles. When the muscles are in equilibrium, the ball is balanced on the gorilla's shoulders. When the ball is off to one end or the other, the muscles are progressively more out of balance. The bar graphs will be connected to channels A, B, C, D.




Equilibration - 4Ch Bi-Lateral Bar-Video

1 2 3 4 5

- Scale-dependent
- Threshold-dependent
- Other

The light turns red when the difference is greater than 35%. Both signals are also displayed in the same line graph and mirrored bars for comparison.

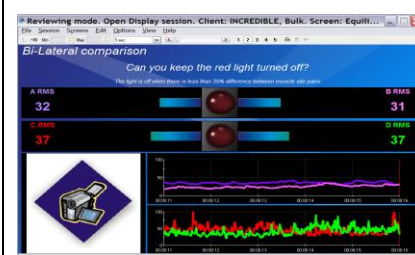

Equilibration - 4Ch Bi-Lateral Bar-Video 2

1 2 3 4 5

- Scale-dependent
- Threshold-dependent
- Other

The light turns red when the difference is greater than 35%. Both signals are also displayed in the same line graph and bars for comparison.



EQU03 – Postural Training with 4 EMG


The next three screens are specified for postural training. They will reinforce good posture with visual assistance.

Equilibration – 4Ch Postural Training – Cervical & Thoracic Spine

1 2 3 4 5

- Scale-dependent
- Threshold-dependent
- Other

The light will turn red when the difference between the two muscles at the site is greater than 35%. The mirrored bar graphs assist the comparison at the site.

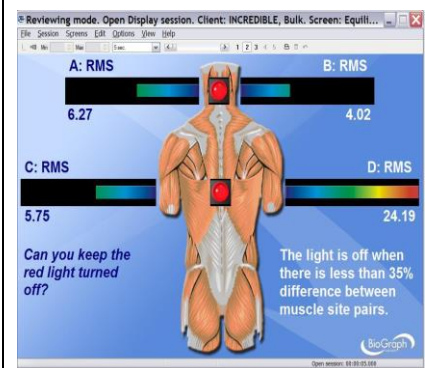


Equilibration – 4Ch Postural Training – Cervical & Lumbar Spine

1 2 3 4 5

- Scale-dependent
- Threshold-dependent
- Other

The light will turn red when the difference between the two muscles at the site is greater than 35%. The mirrored bar graphs assist the comparison at the site.

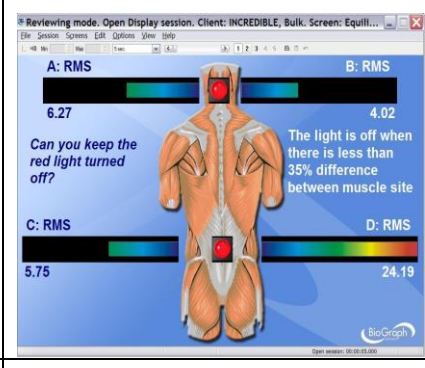


Equilibration – 4Ch Postural Training – Thoracic & Lumbar Spine

1 2 3 4 5

- Scale-dependent
- Threshold-dependent
- Other

The light turns red when the difference is greater than 35%. Both signals are also displayed in the same line graph and bars for comparison.

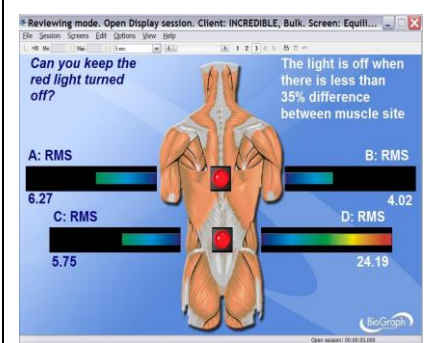

EQU04 – Postural Training with 6 EMG (FlexComp Infinity only)

Equilibration – 6Ch Bi-Lateral – Video

1 2 3 4 5

- Scale-dependent
- Threshold-dependent
- Other

Comparing 6 muscle sites, the difference between each two-muscle site should be less than 35%. The red light will be on to indicate the difference is greater than 35%.




Equilibration – 6Ch Postural Training – Cervical & Thoracic & Lumbar Spine

1 2 3 4 5

- Scale-dependent
- Threshold-dependent
- Other

The 6 muscle sites are as indicated (Cervical, thoracic, and lumbar). The difference between each pair of muscles at the site should be less than 35%. The red light will be on to indicate that the difference is greater than 35%.

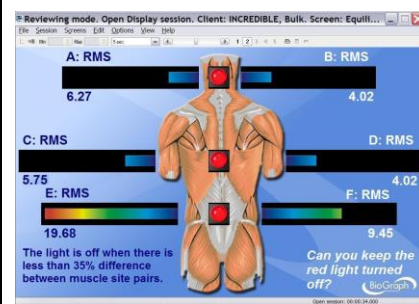

EQU05 – Postural Training with 10 EMG (FlexComp Infiniti only)

Equilibration – 10Ch Bi-Lateral - Video

1 2 3 4 5

- Scale-dependent
- Threshold-dependent
- Other

Comparing 10 muscle sites, the difference between each two-muscle site should be less than 35%. The red light will be on to indicate the difference is greater than 35%.




Equilibration – 10Ch Postural Training – Cervical & Thoracic & Lumbar Spine

1 2 3 4 5

- Scale-dependent
- Threshold-dependent
- Other

The 10 muscle sites are as indicated (Cervical, thoracic, and lumbar). The difference between each pair of muscles at the site should be less than 35%. The red light will be on to indicate that the difference is greater than 35%.

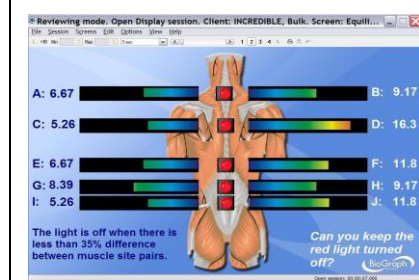


Equilibration – 10Ch Postural Training – Cervical & Thoracic & Lumbar Spine 2

1 2 3 4 5

- Scale-dependent
- Threshold-dependent
- Other

The 10 muscle sites are as indicated (Cervical, thoracic, and lumbar). The difference between each pair of muscles at the site should be less than 35%. The red light will be on to indicate that the difference is greater than 35%.

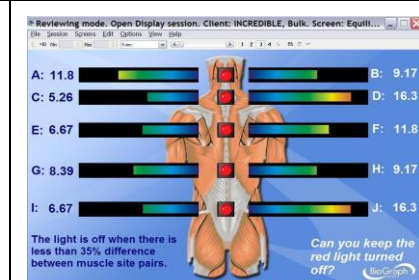


Equilibration – 10Ch Postural Training –Thoracic & Lumbar Spine

1 2 3 **4** 5

- Scale-dependent
- Threshold-dependent
- Other

The 10 muscle sites are as indicated (Thoracic, and lumbar). The difference between each pair of muscles at the site should be less than 35%. The red light will be on to indicate that the difference is greater than 35%.

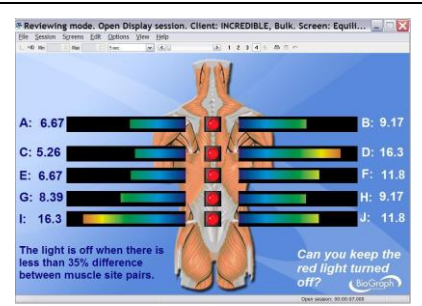

CATEGORY: PRO/FLEX SUITE – SEMG BIOFEEDBACK – TRAINING

These screens are for general training, involving more complex or various exercises.


TRNG01 - Training with 2 EMG

Training - 2Ch Line Graph – Grow Box

1 2 3 4 **5**

- Scale-dependent
- Threshold-dependent
- Other

A proportional sound is played when the square expands past the limit (red line). The left square is for channel A and the right square for channel B. The signals are also displayed on a line graph.

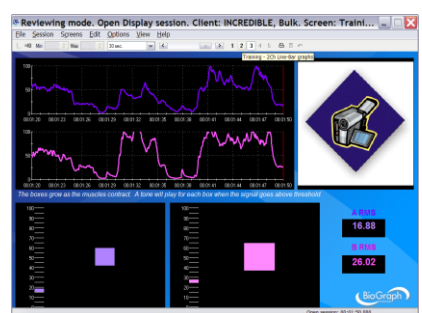


Training - 2Ch Bar Graphs

1 **2** 3 4 5

- Scale-dependent
- Threshold-dependent
- Other

A simple and easy-to-understand display for two channels of EMG.

A song is played in two parts. If both channels are below the threshold then the music stops. If one or both channels are above the threshold then progressively more layers of music are added.

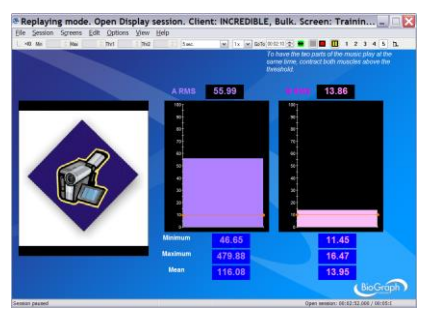


Training - 2Ch Filled Line Graphs

1 2 **3** 4 5

- Scale-dependent
- Threshold-dependent
- Other

Two filled line graphs display the two channels, with a color change at the threshold. Each channel controls a different part of the same song.

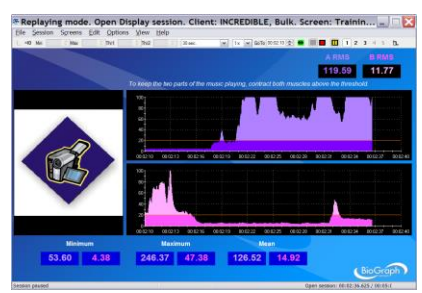

Training - 2Ch Line-Bar Graphs

1 2 3 **4** 5

- Scale-dependent
- Threshold-dependent
- Other

Both channels are displayed on a line graph and a bar graph.




Training - 1Ch Motor Copy Training

1 2 3 4 **5**

- Scale-dependent
- Threshold-dependent
- Other

This screen is used for the motor copy training technique. It can also be used to train the patient to consistently repeat the same contraction without a template.

Do the first repetition, and then wait for the new signal to appear in the top graph below before doing the next repetition. The movement detection threshold is set to 20µV. If you want to modify this value, from the **Edit** menu select **Edit VC Settings**, select V210 and edit the **Input 2 Constant Value**.

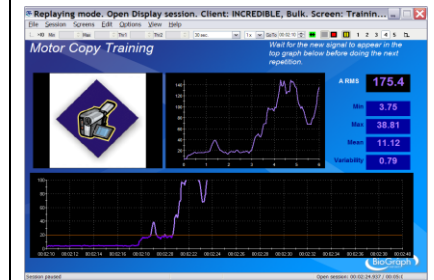

TRNG02 - Training with 4 EMG

Training - 4Ch Bar Graphs

1 2 3 4 5

- Scale-dependent
- Threshold-dependent
- Other

Four bar-graphs representing 4 EMG channels, with a color change above threshold.




Training - 4Ch Filled Line Graphs

1 **2** 3 4 5

- Scale-dependent
- Threshold-dependent
- Other

Four filled line graphs display the four channels, with a color change at the threshold.

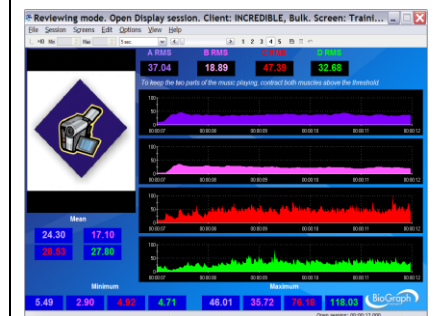


Training - 4Ch Line-Bar Graphs

1 2 **3** 4 5

- Scale-dependent
- Threshold-dependent
- Other

The four channels are displayed on a line graph and a bar graph.

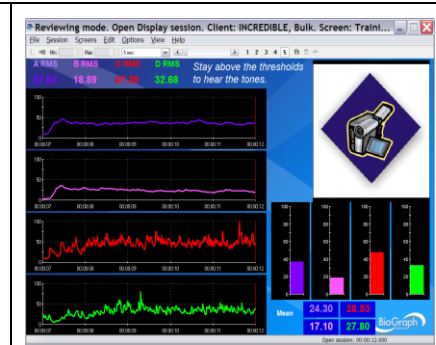

UNSTABLE SHOULDER


If you are going use a webcam to make a video recording of this session, connect your webcam to a USB port of your computer and turn it on.


- Connect the sensors to the encoder as follows:
 - **MyoScan/MyoScan-Z sensor to channel A** (Infraspinatus) and **B** (Lower Trapezius).

A	B	C	D	E	F	G	H	I	J
Myo	Myo								

- Turn the encoder on.

1. Click **Quick Start**.
2. From **Categories**, select one of the categories starting with **Pro/Flex Rehab Suite – SEMG Biofeedback - Unstable Shoulder**.
3. From **Clients**, select a name.
4. From **Favorites**, select the **Unstable Shoulder Protocol**.
5. Click **OK**.

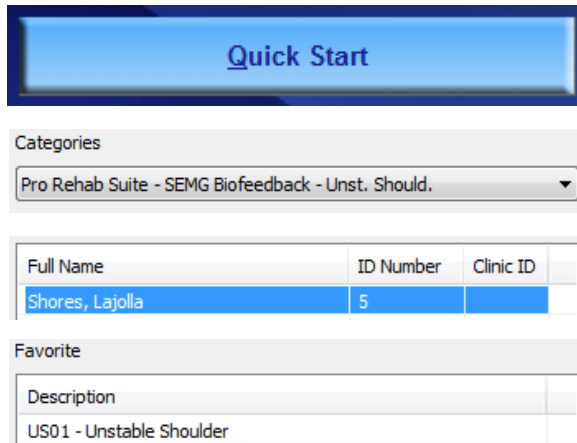

- Click the **Start** button  and read the instructions on the screen.
You can view each screen by clicking on the numerical buttons on the toolbar at the top.

1 2 **3** 4 5

Unstable Shoulder – 2Ch Unstable Shoulder – Overview Screen

1 2 3 4 5

- Scale-dependent
- Threshold-dependent
- Other

This screen will demonstrate videos of required ranges of motion that the patient is required to perform.




Unstable Shoulder – 2Ch Unstable Shoulder – Training Screen

1 2 3 4 5

- Scale-dependent
- Threshold-dependent

Two filled line graphs represent Infraspinatus muscle (Ch A) and Lower Trapezius (Ch B). Both channels have to be above threshold. Two different alarms will play when the signals (A&B) are above or below threshold.

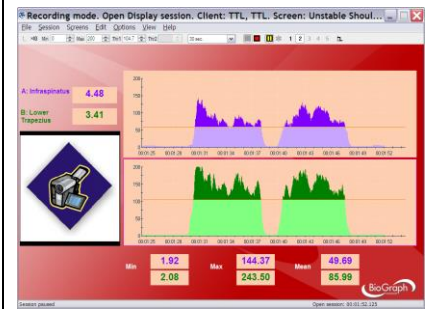


MOTOR FUNCTION RESTORATION

- Connect the sensors to the encoder as follows:
 - MyoScan/MyoScan-Z sensor to channels A to D.
 - Optional MyoScan/MyoScan-Z sensors to channels E to J for FlexComp Ininiti.

A	B	C	D	E	F	G	H	I	J
Myo	Myo	Myo	Myo	Myo (optional for FlexComp)	Myo (optional for FlexComp)	Myo (optional for FlexComp)	Myo (optional for FlexComp)	Myo (optional for FlexComp)	Myo (optional for FlexComp)

- Turn the encoder on.
 1. Click **Quick Start**.
 2. From **Categories**, select one of the categories starting with **Pro/Flex Rehab Suite – Motor Function Restoration**.
 3. From **Clients**, select a name.
 4. From **Favorites**, select the desired Motor Function Restoration Protocol.
 5. Click **OK**.

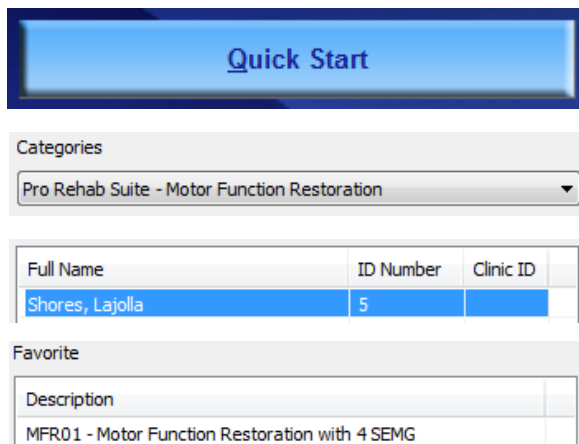

- Click the **Start** button .

You can view each screen by clicking on the numerical buttons on the toolbar at the top.

All the screens under this category are based on the same principle. Only the number of channels and their assignment to a group (uptrain/downtrain) vary.

The feedback is given on the ensemble of the signals. The patient gets a reward only if all the muscles simultaneously follow their correct activation pattern. The muscles and their corresponding signals are sorted into two groups: the up-trained and the down-trained. The up-trained muscles are the ones that will be encouraged to fire. The down-trained muscles are the ones that will be discouraged to fire.

When the EMG of the up-trained muscles goes above a threshold and the EMG of the down-trained muscles goes below a threshold, the DVD resumes.

All the uptrain signals are on the left, while all the downtrain signals are on the right.

When a signal is not in the right state (above threshold for uptrain, below for downtrain), the corresponding bar is red (and green otherwise).

When they are all green, the DVD resumes.



Respiration Training

OVERVIEW

Respiration training should be considered as part of the rehabilitation program:

- Proper breathing during effort enhances the efficiency of the contraction and therefore the efficiency of the training.
- Proper breathing is necessary for overall health and speeds up healing.
- Respiration training may also help the examinee to relax before therapy (reduces anxiety).


The following biofeedback training screens were designed for this purpose.



This logo on a screen in a protocol indicates that you can use an optional webcam with the screen to record video of the session. You need only connect your webcam to a USB port of your computer and turn it on before you start the session.

Note: *Your computer system must be connected to a video capture device, such as a webcam, and the proper software installed, in order to use this function.*

CLINICAL GUIDE REFERENCE

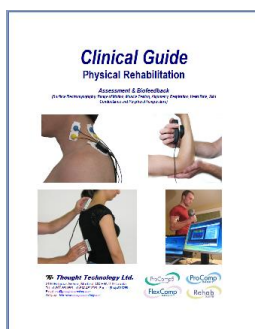

The clinical guide gives general guidelines and recommendations.


Chapter: RESPIRATION TRAINING WITH BIOFEEDBACK

PROCEDURE

- Connect the sensors to the encoder as follows:
 - Respiration sensor to channel D.
 - MyoScan/MyoScan-Z to channel A and B (optional).

A	B	C	D	E	F	G	H	I	J
Myo (optional)	Myo (optional)		Resp						

- Turn the encoder on.
 1. Click **Quick Start**.

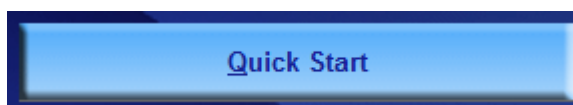

2. From **Categories**, select **Pro/Flex Rehab Suite – Respiration Training**.
3. From **Clients**, select a name.
4. From **Favorites**, select the training protocol.
5. Click **OK**.

Categories
 Pro Rehab Suite - Respiration Training


Full Name	ID Number	Clinic ID
Shores, Lajolla	5	


Favorite

Description
Respiration Training

- Click the **Start** button .

You can view each screen by clicking on the numerical buttons on the toolbar at the top.

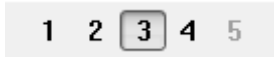

- To stop the session, click the **Stop** button: .

<p>Training - Abdominal Respiration Training</p> <p>1 2 3 4 5</p> <p>This screen was designed to train abdominal breathing.</p> <p>The line graph at the bottom displays the respiration amplitude.</p> <p>The two line graphs on the right display the EMG of the associated muscles. If the EMG of one channel is above its threshold, music will be heard.</p>	
<p>Training - Abdominal Respiration Training Windmill</p> <p>1 2 3 4 5</p> <p>This screen was designed to train abdominal breathing. The greater is the abdominal respiration amplitude, the faster the windmill turns.</p> <p>The line graph at the bottom displays the respiration amplitude.</p> <p>The two line graphs on the right display the EMG of the associated muscles. If the EMG of one channel is above its threshold, the windmill will stop turning and music will be heard.</p>	
<p>Training - Relaxation Training</p> <p>1 2 3 4 5</p> <p>This screen was designed for relaxation.</p> <p>The line graph displays the respiration amplitude.</p> <p>The examinee must follow the pink dot. This will lead them to a slow breathing rate (around 6 breath/min).</p>	

Training – Respiration During Exercise

1 2 3 **4** 5

The screen was designed to help the examinee to breath properly during an exercise. It has a line graph displaying the respiration amplitude and a video camera.


A tone proportional to the respiration amplitude can be heard.



Review screen: Report Review – Respiration and EMG Review Screen

The screen was designed for the review of the session.

It shows the respiration amplitude and the EMG activity along with the video.

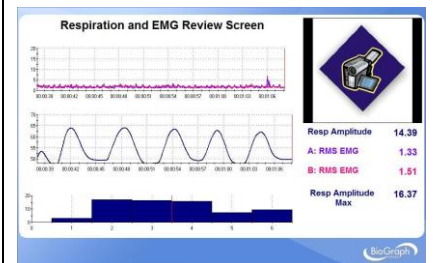

All these screens are accessible from **Start Open Display Session** in the channel set “**Respiration Training**”.

Heart Rate Monitoring and HRV

OVERVIEW

The system allows the therapist to monitor rapid changes in the heart rate. It gives the choice of gathering the heart rate from **EKG (electrocardiogram)** or **BVP (Blood Volume Pulse)**.


The following screens allow the monitoring of heart rate (from BVP or EKG), respiration and EMG (2 sites).



This logo on a screen in a protocol indicates that you can use an optional webcam with the screen to record video of the session. You need only connect your webcam to a USB port of your computer and turn it on before you start the session.

Note: *Your computer system must be connected to a video capture device, such as a webcam, and the proper software installed, in order to use this function.*

CLINICAL GUIDE REFERENCE

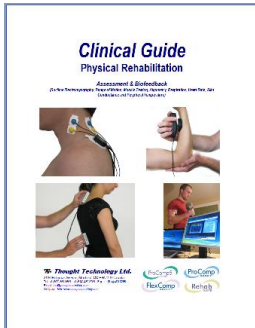

The clinical guide gives general guidelines and recommendations.

Chapter: HEART RATE MONITORING & HRV

Also recommended:


- INTRODUCTION TO SURFACE ELECTROMYOGRAPHY
- RESPIRATION TRAINING WITH BIOFEEDBACK

PROCEDURE

- Connect the sensors to the encoder as follows:
 - BVP or EKG sensor to channel C.
 - Respiration sensor to channel D.
 - MyoScan/MyoScan-Z to channel A and B (optional).

A	B	C	D	E	F	G	H	I	J
Myo (optional)	Myo (optional)	EKG or BVP	Resp						

- Turn the encoder on.
 1. Click **Quick Start**.
 2. From **Categories**, select **Pro/Flex Rehab Suite – Heart Rate Monitoring**.

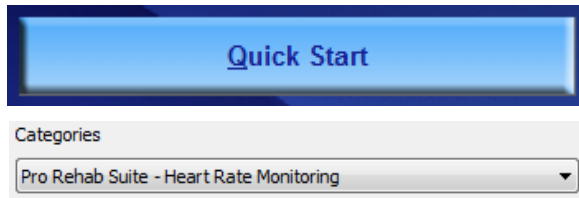

3. From **Clients**, select a name.

Full Name	ID Number	Clinic ID
Shores, Lajolla	5	


4. From **Favorites**, select the training protocol (with EKG or BVP).


Favorite	
Description	
Heart Rate Monitoring with BVP	
Heart Rate Monitoring with EKG	





5. Click **OK**.

- Click the **Start** button .

You can view each screen by clicking on the numerical buttons on the toolbar at the top.

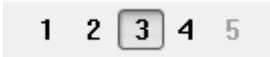


- To stop the session, click the **Stop** button: .
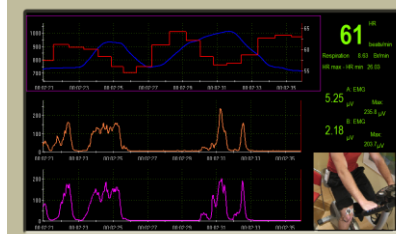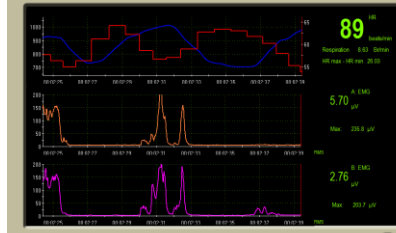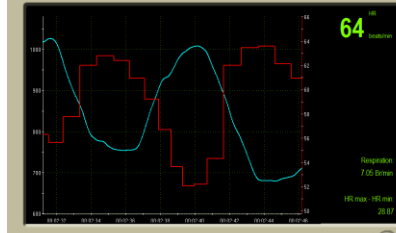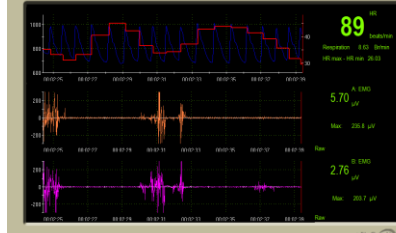
<p>Heart Rate & EMG Monitoring with webcam</p> <p>1 2 3 4 5</p> <p>This screen displays the respiration amplitude and the heart rate on the same line graph. It plots the 2 channels of EMG. The HRV measure HR Max - HR Min is also displayed. Video can be recorded.</p>	
<p>Heart Rate & EMG Monitoring</p> <p>1 2 3 4 5</p> <p>This screen is similar to the one above, but without webcam.</p>	
<p>Heart Rate & Monitoring</p> <p>1 2 3 4 5</p> <p>This screen is similar to the ones above, but shows only the heart rate and respiration.</p>	
<p>Verification Screen</p> <p>1 2 3 4 5</p> <p>This is a signal verification screen. It shows raw BVP (or EKG) with heart rate, as well as raw EMG.</p>	

Review screen

The screen was designed for the review of the session.

It shows the respiration amplitude and rate, the raw BVP (or EKG), heart rate, and Heart Rate Variability (HRV), and the EMG activity along with the video.

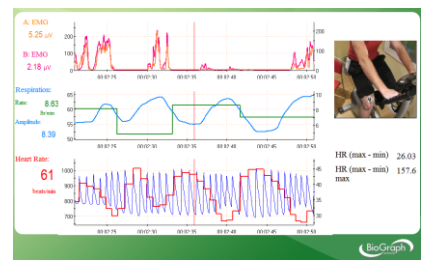

All these screens are accessible from Start Open Display Session in the channel set “Heart Rate Monitoring with BVP and EMG” or “Heart Rate Monitoring with EKG and EMG”.

Skin Conductance and Temperature Biofeedback

OVERVIEW

Skin conductance and peripheral temperature biofeedback is useful when training of overall physiology is desired, as it incorporates two modalities with simple correlations to relaxation: skin conductance (lower) and temperature (raise). When these physiological measures meet the biofeedback conditions, the subject is in a general state of relaxation. These physiological measures can also be used to assess the level of stress of the examinee during the examination.


The following biofeedback screens were designed for that purpose.



This logo on a screen in a protocol indicates that you can use an optional webcam with the screen to record video of the session. You need only connect your webcam to a USB port of your computer and turn it on before you start the session.

Note: *Your computer system must be connected to a video capture device, such as a webcam, and the proper software installed, in order to use this function.*

CLINICAL GUIDE REFERENCE

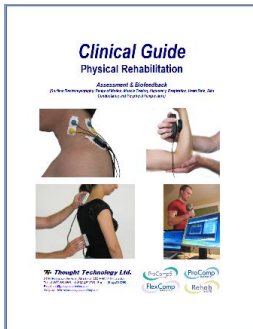

The clinical guide gives general guidelines and recommendations.

Chapter: SKIN CONDUCTANCE AND PERIPHERAL TEMPERATURE BIOFEEDBACK

Also recommended:


- RESPIRATION TRAINING WITH BIOFEEDBACK
- HEART RATE MONITORING & HRV

PROCEDURE

- Connect the sensors to the encoder as follows:
 - Temperature sensor to channel C.
 - Skin Conductance sensor to channel E.

A	B	C	D	E	F	G	H	I	J
		Temp		SC					

- Turn the encoder on.
 1. Click **Quick Start**.
 2. From **Categories**, select **Pro/Flex Rehab Suite – Skin Conductance and Temperature**.

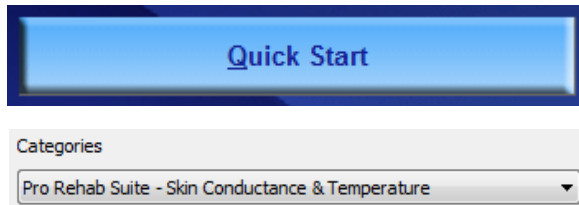

3. From **Clients**, select a name.

Full Name	ID Number	Clinic ID
Shores, Lajolla	5	


4. From **Favorites**, select **Biofeedback with Skin Conductance and Temperature**.


Favorite	
Description	
Biofeedback with Skin Conductance and Temperature	




5. Click **OK**.

- Click the **Start** button .

You can view each screen by clicking on the numerical buttons on the toolbar at the top.

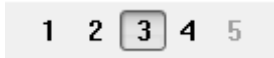

- To stop the session, click the **Stop** button: .

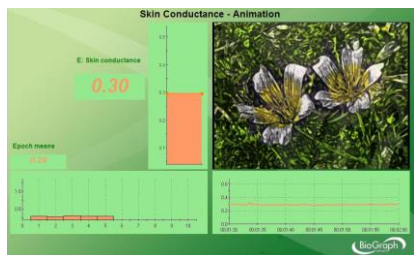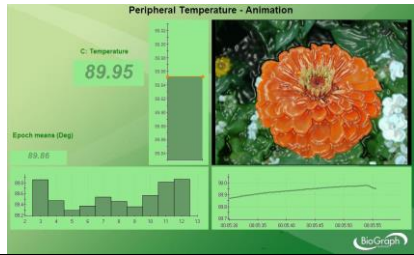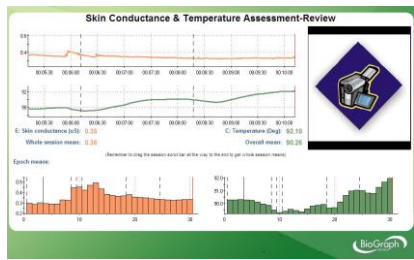
<p>Training - Biofeedback on Skin Conductance</p> <p>1 2 3 4 5</p> <p>The screen shows a bar graph of the skin conductance and plays a song and an animation when the SC value dips below the threshold. The threshold is set to automatically follow the signal to allow for instantaneous feedback of any change of direction.</p>	
<p>Training - Biofeedback on Temperature</p> <p>1 2 3 4 5</p> <p>The screen shows a bar graph of the temperature signal and plays a song and an animation when the signal value moves over the threshold. The threshold is set to automatically follow the signal to allow for instantaneous feedback of any change of direction.</p>	
<p>Assessment & Review – Skin Conductance & Temperature</p> <p>1 2 3 4 5</p> <p>The screen was designed for assessment.</p> <p>The screen shows a line graph of the raw signals of SC and Temp and a trend graph of epoch means. It also has a video/audio instrument, so you can record the examination.</p> <p>Events can be marked by hitting the space bar or a key of the keyboard that you would have preliminarily labeled with the name of the event).</p>	

Review/replay screen

The screen was designed for the review of the session.

The screen shows a line graph of the raw signals of SC and Temp and a trend graph of epoch means.

Drag the time marker (vertical red line) over the raw signal and see the value for any data point as a number in the center of the screen. To see the whole session mean, drag the time marker completely to the end of the session.

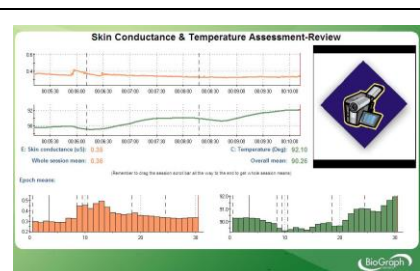

All these screens are accessible from **Start Open Display Session** in the channel set "Biofeedback with Skin Conductance and Temperature".

Static Range of Motion Assessment

OVERVIEW

Range of Motion (ROM) assessment evaluates the ability of the examinee to achieve the full range of movement for a given part of the body.

A dual inclinometer is used for measuring the angle between the neutral position and the maximum range of motion position that the examinee can achieve until they feel restriction, tightness or discomfort.


The protocols are based on AMA guides (6th edition, 2007).

Protocols use the world standard Neutral Zero Reference method. This method defines the 0° angle as referring to the neutral position of the joint.

The measurement is repeated 6 times as a validity check.

The measures are compared to AMA normative data.

CLINICAL GUIDE REFERENCE

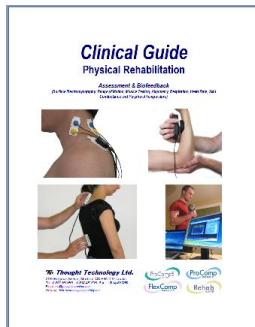

The clinical guide gives general guidelines and recommendations for the accurate use of the instrumentation. It also explains how to prepare the examinee and the procedure to follow.

Chapter: RANGE OF MOTION ASSESSMENT; **section:** STATIC RANGE OF MOTION ASSESSMENT


GENERAL PROCEDURE

Preparing the instrumentation

- Connect the sensors to the encoder as follows:
 - Dual/single inclinometer to channel E;
 - Foot pedal to channel C.


A	B	C	D	E	F	G	H	I	J
		Pedal		INC					


- Turn the encoder on.
 1. Click **Quick Start**.
 2. From **Categories**, select **Pro/Flex Static ROM Assessment**.
 3. From **Clients**, select a name.

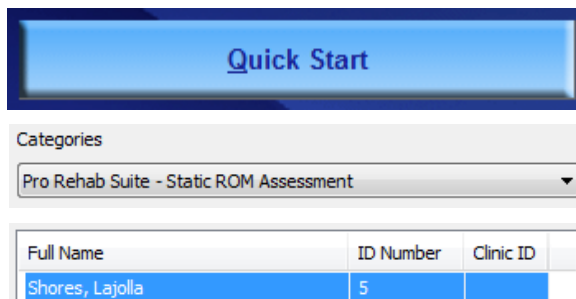


4. From **Favorites**, select the desired Static ROM Protocol.
5. Click **OK**.

Favorite	
Description	
S.ROM 01 - Cervical Spine - Flexion/Extension	
S.ROM 02 - Cervical Spine - L/R Lateral Flexion	
S.ROM 03 - Cervical Spine - Left/Right Rotation	
S.ROM 04 - Thoracic Spine - Flexion/Extension	
S.ROM 05 - Thoracic Spine - Left/Right Rotation	
S.ROM 06 - Lumbar Spine - Flexion/Extension	
S.ROM 07 - Lumbar Spine - L/R Lateral Flexion	
S.ROM 08 - Shoulder - Flexion/Extension	
S.ROM 09 - Shoulder - Abduction/Adduction in S30°	
S.ROM 10 - Shoulder - External/Internal Rotation	
S.ROM 11 - Elbow - Flexion/Extension	
S.ROM 12 - Forearm - Supination/Pronation	
S.ROM 13 - Wrist - Flexion/Extension	
S.ROM 14 - Wrist - Radial/Ulnar Deviation	
S.ROM 15 - Hip - Flexion/Extension	
S.ROM 16 - Hip - Abduction/Adduction	
S.ROM 17 - Hip - External/Internal Rotation	
S.ROM 18 - Knee - Flexion/Hyper-Extension	
S.ROM 19 - Ankle - Flexion/Extension	


- Click the **Start** button  and read the instructions on the screen. **Do not press the foot pedal yet.**

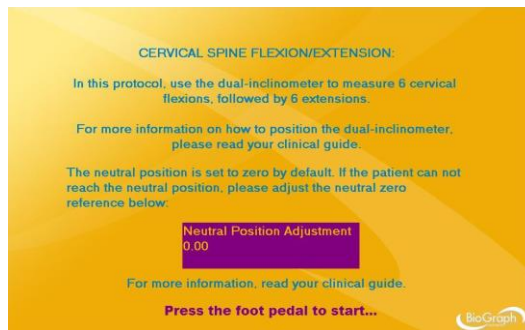

- Position yourself next to the examinee, with the encoder attached to your belt and the pedal near your foot.
- Press the foot pedal to go to the next screen.
- Position and stabilize the examinee in neutral position.
- If the patient cannot reach the neutral position, right-click on the purple box on the screen, enter the neutral zero reference (angle displayed when zero button is pressed) and click **Apply**:

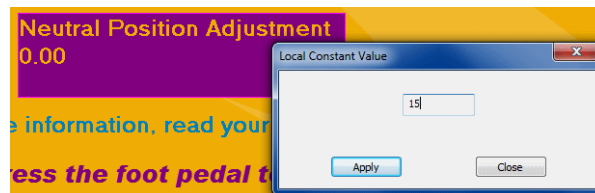


- Position the dual inclinometer on the examinee and press the Zero button (on the primary or secondary, whichever is more convenient for you).

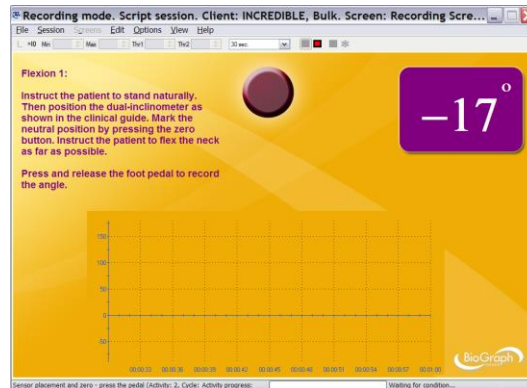


Measuring

1. Instruct the examinee to perform the motion slowly, until they feel restriction, tightness or discomfort. Make sure the inclinometer does not move against the body part during the motion.
2. When the position is stabilized, press the pedal to record the measure.

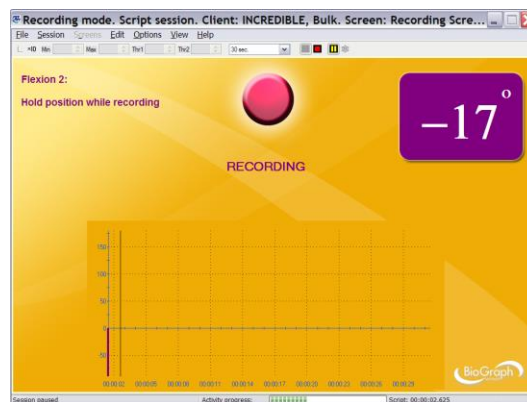


3. Instruct the examinee to go back to neutral position. **Do not press the Zero button again.**
- Repeat the exercise **6 times**.
 - Then repeat it another **6 times** for the antagonistic movement.

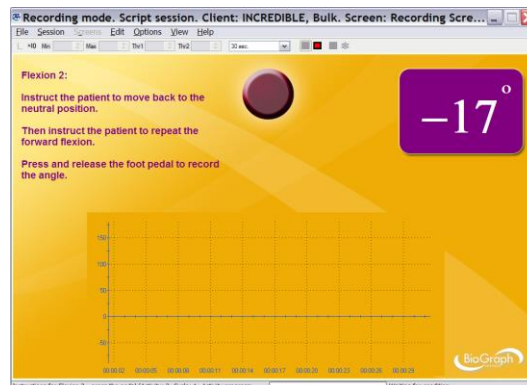

- When the session ends, a series of message prompts guides you through the process of saving data, adding notes to the session file, and switching to reviewing mode so that you can generate a session report.

Generating a report

- After the review screen opens, you can generate a ROM Assessment report. This is an Excel report.


1. Click the **Session Report** icon  in the tool bar.
2. Select **Generate Excel Report**.

Microsoft Excel opens and displays your report as a workbook. You can print or save the report using Excel's **Print** and **Save** functions.

Static Range of Motion - Inclinometry


Cervical Spine Flexion/Extension

Client: Bulk INCREDIBLE
 Session Date: 15/08/2008
 Session Time: 3:30:38 PM



Position: <type if here or select>
Movement type: <select>

Flexion	Result	Valid	Used
Measure 1	42.9	YES	<input type="checkbox"/>
Measure 2	46.0	YES	<input type="checkbox"/>
Measure 3	45.4	YES	<input type="checkbox"/>
Measure 4	44.7	YES	<input checked="" type="checkbox"/>
Measure 5	45.7	YES	<input checked="" type="checkbox"/>
Measure 6	42.8	YES	<input checked="" type="checkbox"/>

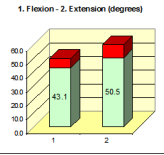

1. Flexion - 2. Extension (degrees)

Extension	Result	Valid	Used
Measure 1	53.8	YES	<input type="checkbox"/>
Measure 2	53.2	YES	<input type="checkbox"/>
Measure 3	53.4	YES	<input type="checkbox"/>
Measure 4	52.8	YES	<input checked="" type="checkbox"/>
Measure 5	50.1	YES	<input checked="" type="checkbox"/>
Measure 6	48.6	YES	<input checked="" type="checkbox"/>

ROM	Norm	Result	Max	% Norm	Dev	% Dev
Flexion	50	43.1	44.7	86.2	-6.9	-13.8
Extension	60	50.5	52.8	84.2	-9.5	-15.8

Summary Notes

Follow-up Notes


Validity Check of the Trials

Left Rotation	Result	Valid	Used
Measure 1	22.3	YES	<input type="checkbox"/>
Measure 2	24.7	YES	<input type="checkbox"/>
Measure 3	23.8	YES	<input type="checkbox"/>
Measure 4	24.9	YES	<input checked="" type="checkbox"/>
Measure 5	23.8	YES	<input checked="" type="checkbox"/>
Measure 6	24.7	YES	<input checked="" type="checkbox"/>

Right Rotation	Result	Valid	Used
Measure 1	36.8	YES	<input type="checkbox"/>
Measure 2	32.9	YES	<input type="checkbox"/>
Measure 3	33.8	YES	<input type="checkbox"/>
Measure 4	33.9	YES	<input checked="" type="checkbox"/>
Measure 5	34.0	YES	<input checked="" type="checkbox"/>
Measure 6	34.1	YES	<input checked="" type="checkbox"/>

The report shows the 6 measures and which ones are selected. Measures are considered as valid if they are within +/-5° or +/-10% of the average of the three. It takes the three last consecutive valid measures.

Final Result

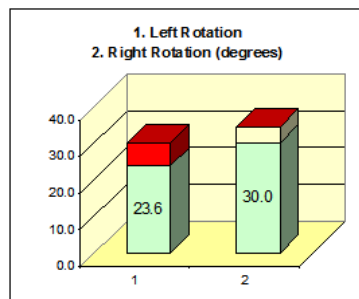

ROM	Norm	Result	Max	% Norm	Dev	% Dev
Left Rotation	30	23.6	24.7	78.7	-6.4	-21.3
Right Rotation	30	34.5	36.8	114.9	4.5	14.9

It then compares these to AMA Norms and shows the % of norm and the deviation from the norm.

If the result is less than the norm, you see the result in green and the deviation from the norm in red.

If the result is greater than the norm, you see the norm in green and the deviation from the norm in yellow (so the result is green+yellow).

User's fields

Position

You can select the examinee's position (there is a different list for each protocol):

Position:

Left Rotation	Result	Value	Used
---------------	--------	-------	------

Or you can enter a different one:

Involved Side (for the extremities only):

You can select the involved side:

Side:

Used	Left	Right
------	------	-------

Movement Type:

You can select the movement type:

Movement type:

1. Left Rotation	2. Right Rotation (degrees)
------------------	-----------------------------

Summary and Follow-up notes

You can enter notes in the summary/follow-up notes boxes:

Summary Notes:


Marked improvement

- In **Excel 2000** and **Excel 2003**, place your cursor in the text box and start typing.
- In **Excel 2007**, place your cursor in the text box.
 1. Right-click the mouse button.
 2. Select **Edit text**.
 3. Start typing.



Comparing to uninvolved side

Some reports allow you to compare the results with the uninvolved side:

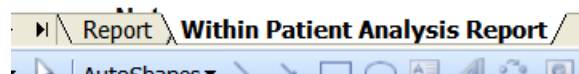

Select the sheet "Within Patient Analysis Report".

Type the results of the uninvolved side in the corresponding column:

Uninvolved	Res
153	12
0	2

Dynamic Range of Motion Assessment

OVERVIEW

Similarly to Dynamic SEMG, Dynamic ROM assesses muscle tension through various movements, by combining EMG with inclinometry.

In addition to general protocols, the system provides three assessment protocols for the **cervical spine** (CP, SCM or Traps) and two for the **lumbar spine**.


It also provides one assessment protocol for the **anterior knee** (VMO and VL).



This logo on a screen in a protocol indicates that you can use an optional webcam with the screen to record video of the session. You need only connect your webcam to a USB port of your computer and turn it on before you start the session.

Note: Your computer system must be connected to a video capture device, such as a webcam, and the proper software installed, in order to use this function.

CLINICAL GUIDE REFERENCE

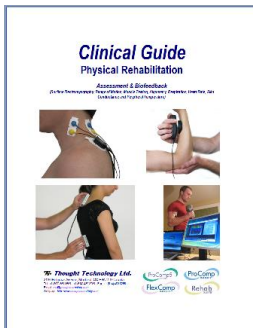

The clinical guide gives general guidelines and recommendations for the accurate use of the instrumentation. It also explains how to prepare the examinee and the procedure to follow.

Chapter: RANGE OF MOTION ASSESSMENT; **section:** DYNAMIC RANGE OF MOTION ASSESSMENT

Also recommended:

- INTRODUCTION TO SURFACE ELECTROMYOGRAPHY
- SKELETAL MUSCLE PROPERTIES
- SEMG ASSESSMENT; **section:** DYNAMIC SEMG ASSESSMENT

GENERAL PROCEDURE

Starting the Session


If you are going use a webcam to make a video recording of this session, connect your webcam to a USB port of your computer and turn it on.

- Connect the sensors to the encoder as follows:
 - MyoScan/MyoScan-Z sensors to channels A and B (left in A, right in B).
 - Optional MyoScan/MyoScan-Z sensors to channels C and D (left in C, right in D).
 - Dual/single inclinometer to channel E.

A	B	C	D	E	F	G	H	I	J
Myo	Myo	Myo (optional)	Myo (optional)	INC					


- Turn the encoder on.

1. Click **Quick Start**.
2. From **Categories**, select **Pro/Flex Rehab Suite - Dynamic ROM Assessment**.
3. From **Clients**, select a name.
4. From **Favorites**, select the desired General Dynamic ROM Assessment Protocol.
5. Click **OK**.

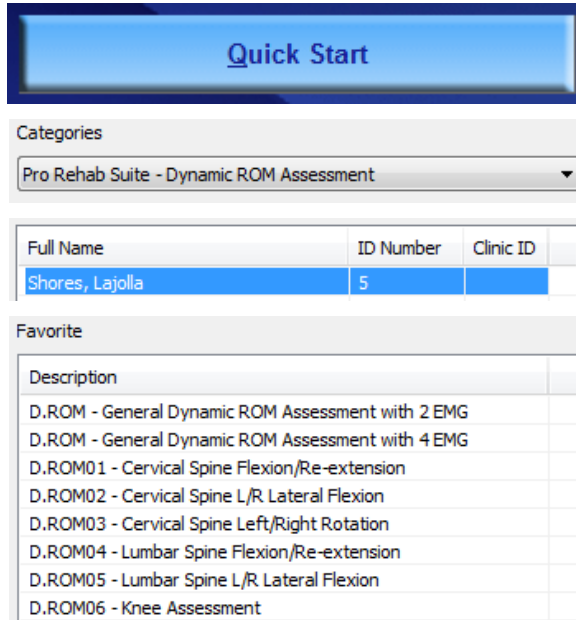


Measuring

- Click the **Start** button .

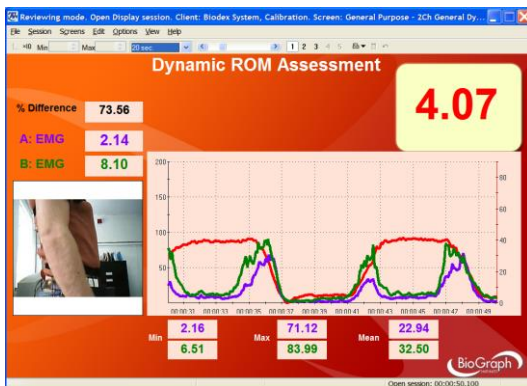

D.ROM – General Dynamic ROM Assessment with 2 EMG

The Multi-line graph will display the EMG signal of 2 channels A and B along with the angle (in red). There are statistics available as well.

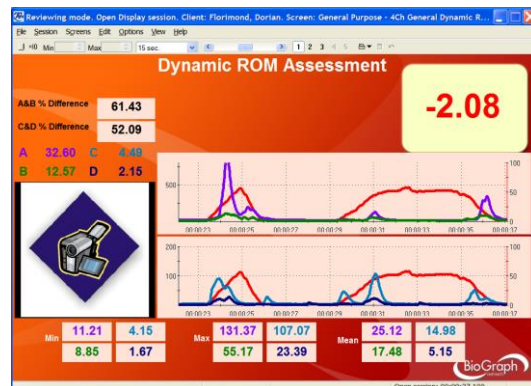

D.ROM – General Dynamic ROM Assessment with 4 EMG


The two Multi-line graph will display the EMG signal of 2 channels simultaneously, respectively A and B, C and D, along with the angle (in red). There are statistics available as well.

- Press the zero button of the inclinometer once, to mark the neutral position.
- Mark the different stages of the movement with event makers (events can be marked by hitting the space bar or a keyboard key that was previously assigned the name of the event).

Reviewing the session


When the session ends, a series of message prompts guides you through the process of saving data, adding notes to the session file, and switching to reviewing mode so that you can generate a session report.

In review mode, a screen similar to this will appear:

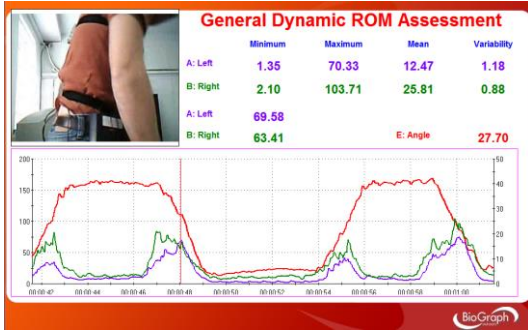

D.ROM – General Dynamic ROM Assessment with 2 EMG

The Multi-line graph will display the EMG signal of 2 channels A and B along with the angle (in red). There are statistics available as well.

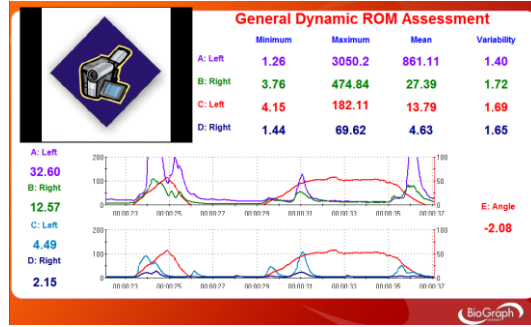


D.ROM – General Dynamic ROM Assessment with 4 EMG

The two Multi-line graph will display the EMG signal of 2 channels simultaneously, respectively A and B, C and D, along with the angle (in red). There are statistics available as well.

Select the desired display time window on the screen by using the dropdown list.

It is recommended to select first **Size to session** for a global view of the session and then **Size to activity** to view the different phases of the motion.

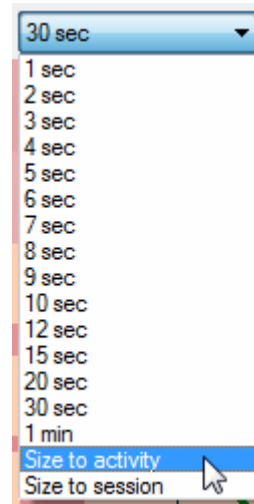


Scroll through the session by using the scroll bar.




You can read the value of a given point on the signal by moving the red Time Mark. The Time Mark is usually placed at the right end of the graph (see picture).

To move the Time Mark:

1. Place the cursor over the Time Mark.
2. Click and hold down the mouse button.
3. Drag the Time Mark to the desired position.

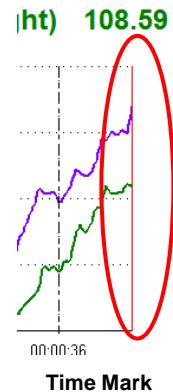

Move the mouse cursor over a marker (vertical dashed line) to display its label (such as flexion, re-extension etc.)

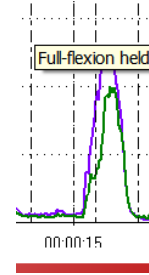


If you used a webcam to make a video recording of the session, during review the video playback is synchronized to the EMG signal as you move the Time Mark back and forth along the EMG signal line graph. This allows you to see lateral deviations in the movement and to observe any antalgic position.

The review allows you to verify that the motion has been properly performed by the examinee at the right time, when prompted. You can also check the consistency from trial to trial.

Generating a report

- After the review screen opens, you can generate a Dynamic SEMG Assessment report. This is a Text report.

1. Click the **Session Report** icon  in the tool bar.
2. Select **Generate Text Report**, select report components in the **Session Report window**, and click **Generate Report**.

Your report displays as a document in HTML, PDF, or Microsoft Word format, depending upon the setting in **Preferences**. The report can be saved or printed using the **Print** and **Save** functions of the relevant software.

PROCEDURE FOR LUMBAR SPINE ASSESSMENT


Starting the Session

If you are going use a webcam to make a video recording of this session, connect your webcam to a USB port of your computer and turn it on.

- Connect the **MyoScan/MyoScan-Z EMG sensors** to the encoder in **channels A and B** (left in A, right in B).
- Dual inclinometer to channel E.


A	B	C	D	E	F	G	H	I	J
Myo (left)	Myo (right)			INC					

- Turn the encoder on.
 1. Click **Quick Start**.
 2. From **Categories**, select **Pro/Flex Rehab Suite - Dynamic ROM Assessment**.
 3. From **Clients**, select a name.

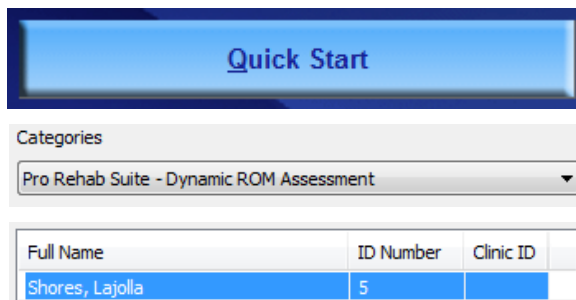

4. From **Favorites**, select the desired **Dynamic ROM Protocol for Lumbar Spine**.
5. Click **OK**.

Favorite	
Description	
D.ROM - General Dynamic ROM Assessment with 2 EMG	
D.ROM - General Dynamic ROM Assessment with 4 EMG	
D.ROM01 - Cervical Spine Flexion/Re-extension	
D.ROM02 - Cervical Spine L/R Lateral Flexion	
D.ROM03 - Cervical Spine Left/Right Rotation	
D.ROM04 - Lumbar Spine Flexion/Re-extension	
D.ROM05 - Lumbar Spine L/R Lateral Flexion	
D.ROM06 - Knee Assessment	


- Click the **Start** button  and read the instructions on the screen.

First you will see an overview screen (Flexion/Re-extension, shown here):



- Press a key to go to the next screen.

In the next screen, verify that the signal is free of artifacts and the sensors properly placed.

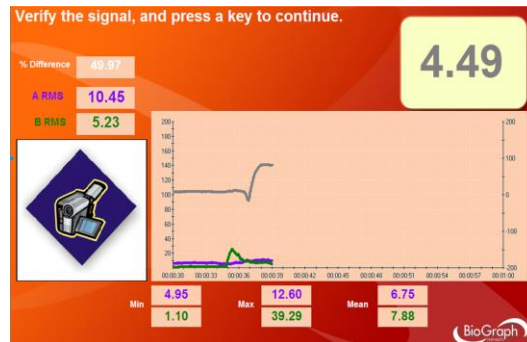

- Press the zero button of the inclinometer once, to mark the neutral position.
- Before going to the next screen and starting to record, instruct the examinee to perform the motion slowly, until they feel restriction, tightness or discomfort.

Measuring

1. Before performing the motion, the examinee stays in neutral position and relaxes their muscles during 10 seconds.



2. Then the voice prompt asks the examinee to perform the motion three times.




3. At the end of the exercise, the examinee is asked to stay in neutral position and relax their muscles during 10 seconds.



Reviewing the session

When the session ends, a series of message prompts guides you through the process of saving data, adding notes to the session file, and switching to reviewing mode so that you can generate a session report.


In review mode, a screen similar to this will appear:

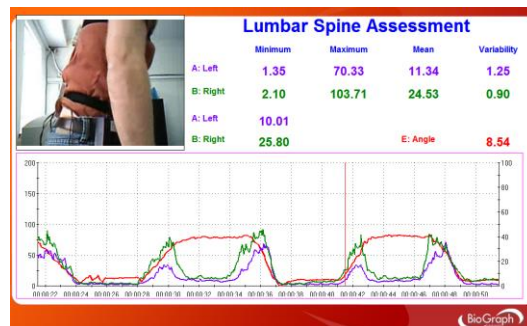

The review allows you to verify that the motion has been properly performed by the examinee at the right time, when prompted. You can also check the consistency from trial to trial.

Generating a report

- After the review screen opens, you can generate a Dynamic ROM Assessment report. This is a Text report.

- Click the **Session Report** icon  in the tool bar.
- Select **Generate Text Report**, select report components in the **Session Report window**, and click **Generate Report**.

Your report displays as a document in HTML, PDF, or Microsoft Word format, depending upon the setting in **Preferences**. The report can be saved or printed using the **Print** and **Save** functions of the relevant software.

PROCEDURE FOR CERVICAL SPINE ASSESSMENT


Starting the Session


If you are going use a webcam to make a video recording of this session, connect your webcam to a USB port of your computer and turn it on.

- Connect the sensors to the encoder as follows:
 - MyoScan/MyoScan-Z** sensors positioned on the CPs to channels **A** and **B** (left in A, right in B).
 - MyoScan/MyoScan-Z** sensors positioned on the SCMs or Traps to channels **C** and **D** (left in C, right in D).
 - Dual inclinometer** to channel **E**.

A	B	C	D	E	F	G	H	I	J
Myo (left CP)	Myo (right CP)	Myo (left SCM or Traps)	Myo (right SCM or Traps)	INC					

- Turn the encoder on.
 - Click **Quick Start**.
 - From **Categories**, select **Pro/Flex Rehab Suite - Dynamic ROM Assessment**.
 - From **Clients**, select a name.
 - From **Favorites**, select the desired **Dynamic ROM Protocol for Cervical Spine**.
 - Click **OK**.

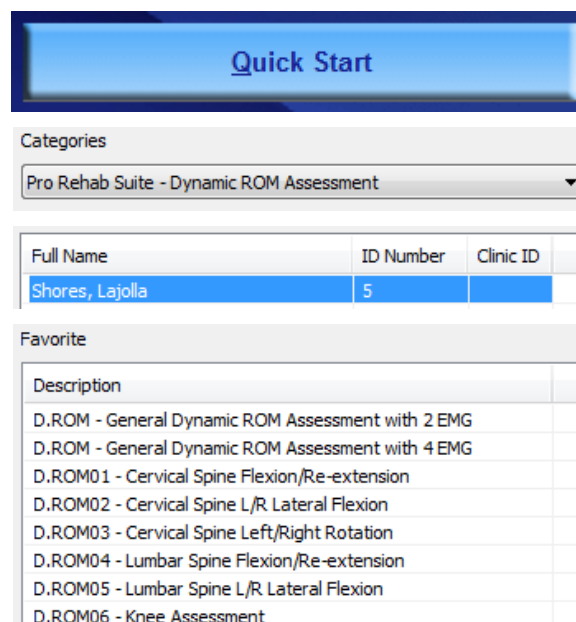


- Click the **Start** button  and read the instructions on the screen. First you will see an overview screen (**Flexion/Re-extension**, shown here):



- Press a key to go to the next screen.


In the next screen, verify that the signal is free of artifacts and the sensors properly placed.

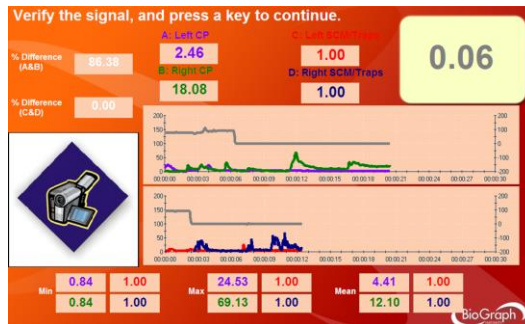

- Press the zero button of the inclinometer once, to mark the neutral position.
- Before going to the next screen and start recording, instruct the examinee to perform the motion slowly, until they feel restriction, tightness or discomfort.

Measuring

1. Before performing the motion, the examinee stays in neutral position and relaxes their muscles during 10 seconds.

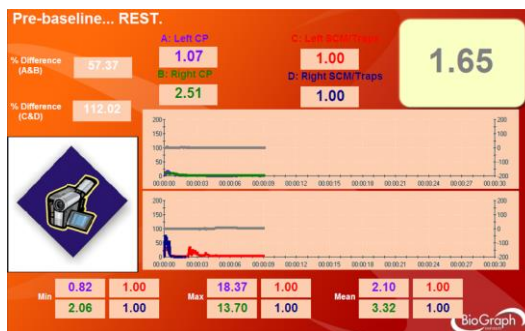

2. Then the voice prompt ask the examinee to perform the motion three times.



- At the end of the exercise, the examinee is asked to stay in neutral position and relax their muscles during 10 seconds.



Reviewing the session


- When the session ends, a series of message prompts guides you through the process of saving data, adding notes to the session file, and switching to reviewing mode so that you can generate a session report.

In review mode, a screen similar to this will appear:



The review allows you to verify that the motion has been properly performed by the examinee at the right time, when prompted. You can also check the consistency from trial to trial.

Generating a report

- After the review screen opens, you can generate a Dynamic ROM Assessment report. This is a Text report.
 - Click the **Session Report** icon  in the tool bar.
 - Select **Generate Text Report**, select report components in the **Session Report window**, and click **Generate Report**.

Your report displays as a document in HTML, PDF, or Microsoft Word format, depending upon the setting in **Preferences**. The report can be saved or printed using the **Print** and **Save** functions of the relevant software.

PROCEDURE FOR ANTERIOR KNEE ASSESSMENT

Starting the Session

If you are going use a webcam to make a video recording of this session, connect your webcam to a USB port of your computer and turn it on.

- Connect the **MyoScan/MyoScan-Z EMG sensors** to the encoder in **channels A and B** (A for VMO, B for VL).
- Dual inclinometer** to channel **E**.


A	B	C	D	E	F	G	H	I	J
Myo (left)	Myo (right)			INC					

- Turn the encoder on.

1. Click **Quick Start**.



2. From **Categories**, select **Pro/Flex Rehab Suite - Dynamic ROM Assessment**.

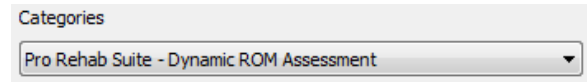


3. From **Clients**, select a name.

Full Name	ID Number	Clinic ID
Shores, Lajolla	5	

4. From **Favorites**, select the **Dynamic ROM Protocol for Knee Assessment**.

Description
D.ROM - General Dynamic ROM Assessment with 2 EMG
D.ROM - General Dynamic ROM Assessment with 4 EMG
D.ROM01 - Cervical Spine Flexion/Re-extension
D.ROM02 - Cervical Spine L/R Lateral Flexion
D.ROM03 - Cervical Spine Left/Right Rotation
D.ROM04 - Lumbar Spine Flexion/Re-extension
D.ROM05 - Lumbar Spine L/R Lateral Flexion
D.ROM06 - Knee Assessment

5. Click **OK**.

- Click the **Start** button  and read the instructions on the screen.

First you will see an overview screen:



- Press a key to go to the next screen.

In the next screen, verify that the signal is free of artifacts and the sensors properly placed.



- Press the zero button of the inclinometer once, to mark the neutral position.
- Before going to the next screen and starting to record, instruct the examinee to perform the motion slowly, until they feel restriction, tightness or discomfort.

Measuring

1. Before performing the motion, the examinee stays in neutral position and relaxes their muscles during 10 seconds.



2. Then the voice prompt asks the examinee to perform the motion three times.




3. At the end of the exercise, the examinee is asked to stay in neutral position and relax their muscles during 10 seconds.



When the session ends, a series of message prompts guides you through the process of saving data, adding notes to the session file, and switching to reviewing mode so that you can generate a session report.

Reviewing the session


In review mode, a screen similar to this will appear:

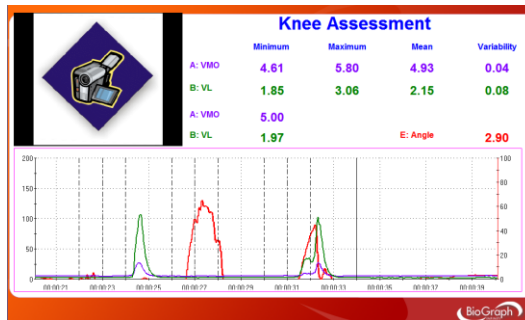

The review allows you to verify that the motion has been properly performed by the examinee at the right time, when prompted. You can also check the consistency from trial to trial.

Generating a report

- After the review screen opens, you can generate a Dynamic ROM Assessment report. This is a Text report.

1. Click the **Session Report** icon  in the tool bar.
2. Select **Generate Text Report**, select report components in the **Session Report window**, and click **Generate Report**.

Your report displays as a document in HTML, PDF, or Microsoft Word format, depending upon the setting in **Preferences**. The report can be saved or printed using the **Print** and **Save** functions of the relevant software.

Range of Motion Training

OVERVIEW

Range-of-motion therapy usually consists of simple exercises to increase the range of motion, flexibility, strength, endurance and control over the movement.


The following biofeedback training screens were designed for this purpose.



This logo on a screen in a protocol indicates that you can use an optional webcam with the screen to record video of the session. You need only connect your webcam to a USB port of your computer and turn it on before you start the session.

Note: Your computer system must be connected to a video capture device, such as a webcam, and the proper software installed, in order to use this function.

CLINICAL GUIDE REFERENCE

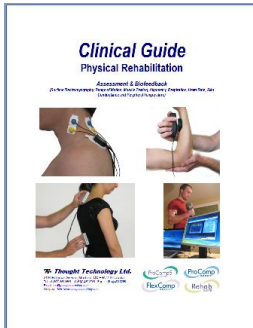

The clinical guide gives general guidelines and recommendations for the use of biofeedback for ROM therapy. It also explains how to prepare the examinee and the procedure to follow.


Chapter: ROM THERAPY WITH BIOFEEDBACK

PROCEDURE

- Connect the sensors to the encoder as follows:
 - A **single inclinometer** unit (InclinoTrac) to channel **C**.
 - Optionally, the second InclinoTrac unit can be used as a **single inclinometer** and connected to **channel E**.

A	B	C	D	E	F	G	H	I	J
		INC		INC (optional)					

- Turn the encoder on.
 1. Click **Quick Start**.
 2. From **Categories**, select **Pro/Flex Rehab Suite – ROM Training**.
 3. From **Clients**, select a name.

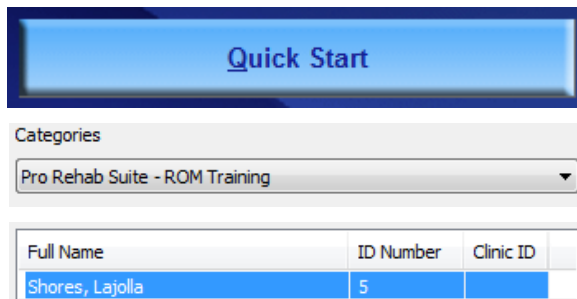



4. From **Favorites**, select the desired training protocol.
5. Click **OK**.

Favorite	
Description	
ROMT01 - ROM Training with 1 inclinometer	
ROMT02 - ROM Training with 2 inclinometers	





- Click the **Start** button .

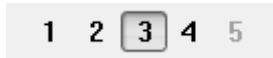You can view each screen by clicking on the numerical buttons on the toolbar at the top.



- To stop the session, click the **Stop** button: .


ROMT01 – ROM Training with 1 inclinometer
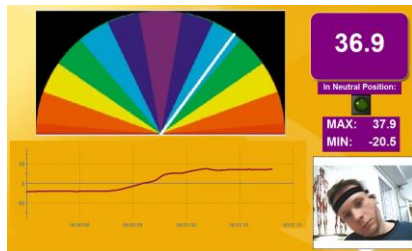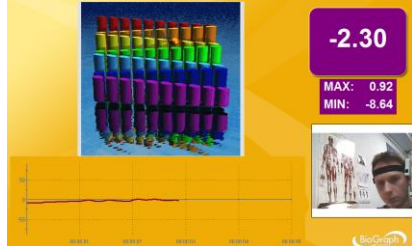
<p>Neutral Position Training – Gorilla Animation</p> <p>1 2 3 4 5</p> <p>In this screen, the examinee is being trained to keep his joint in neutral position, by keeping the ball centered behind the monkey's neck. When he is successful, the green light turns on.</p>	
<p>Threshold-based Training – Bowling Animation</p> <p>1 2 3 4 5</p> <p>This is a threshold-based exercise. The examinee must perform the motion until a threshold is crossed. When this happens, the animation starts, and the bowl is thrown. The examinee must hold the position during 5 seconds to do a strike and get a point. They must then go back to neutral position to restart.</p>	
<p>Neutral Position and ROM Training – Dial Animation</p> <p>1 2 3 4 5</p> <p>This is a stretching exercise. The examinee is told to reach a given color on the rainbow. When they return to neutral position, the light turns on.</p>	
<p>ROM Training – Tube Game</p> <p>1 2 3 4 5</p> <p>This screen is similar to the Dial Animation. This time, the examinee has to move the ball up and down.</p>	

ROM Training – Animal Game

1 2 3 4 **5**

Here, the examinee has to align the character with the animal appearing in the left box, by moving his limb.

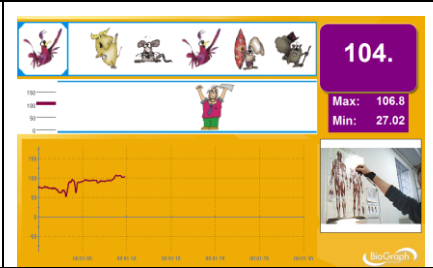

Review screen: ROM Training - Review

The screen was designed for reviewing the session. It shows the ROM amplitude along with the video.



All these screens are accessible from **Start Open Display Session** in the channel set “ROM Training with 1 inclinometer”.


ROMT02 – ROM Training with 2 inclinometers

The second unit of the dual-InclinoTrac can also be used as a single unit. You can use both units together, either for working on the three dimensions or bilateral training.

ROM Training – Balance Animation

1 2 3 4 5


For this screen, the inclinometers are positioned at the back and the left of the head or the trunk. The examinee can train to keep his head/trunk straight (line in the center of the lissajous graph, the balls in the middle of the balance and the green lights turned on). He can also roll his head around his neck and see where his ROM is limited.

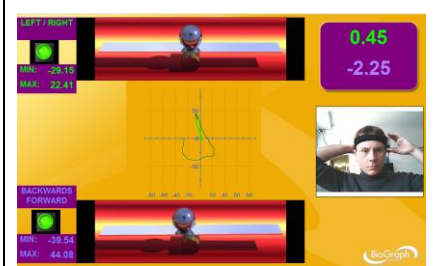

ROM Training – Motor Copy

1 **2** 3 4 5

Here the inclinometers are attached to the arms or legs. The examinee has to elevate both limbs in the same time to keep the ball centered on the balance. The line on the rainbow gives the averaged angle between the two sides.

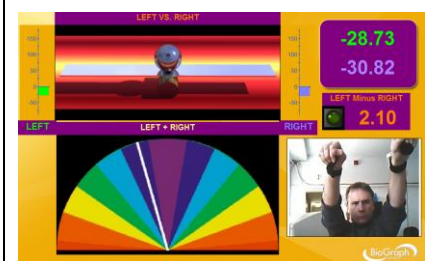


ROM Training - Review

1 2 **3** 4 5

The screen was designed for reviewing the session but can be used to monitor the motion.

It shows the ROM amplitudes along with the video.

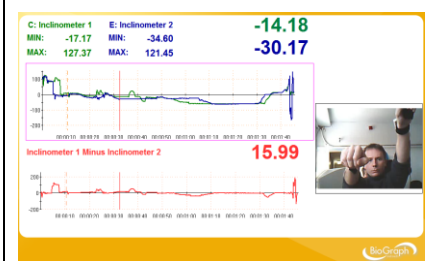

Review Screen: ROM Training - Review

The screen was designed for reviewing the session.
It shows the ROM amplitudes along with the video.



All these screens are accessible from **Start Open Display Session** in the channel set **“ROM Training with 2 inclinometers”**.

Manual Muscle Testing

OVERVIEW


Manual muscle testing is the assessment of muscles and tendons and their ability to generate force. Computerized manual muscle testing improves and enriches a method that has existed and evolved for almost a century, by replacing subjective factors with objective data. SEMG allows the monitoring of potential muscle substitution.

Note that muscle testing with ForceTrac is allowed only for grades 4 and 5.

The examinee should be monitored closely and tests should be terminated immediately if there is any evidence of pain.

The system is intended only to capture an objective record of force applied during testing, rather than to reduce the risk of injury. **Therefore the ultimate responsibility for risk of injury rests with the examiner.**

CLINICAL GUIDE REFERENCE

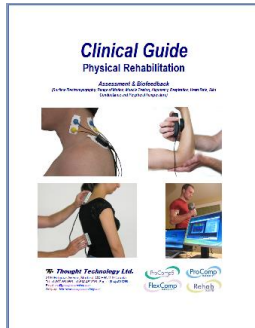

The clinical guide gives general guidelines and recommendations for the accurate use of computerized instruments along with traditional muscle testing. It also explains how to prepare the examinee and the procedure to follow.

Chapter: MANUAL MUSCLE TESTING

Also recommended: INTRODUCTION TO SURFACE ELECTROMYOGRAPHY


PROCEDURE

Starting the Session

- Connect the sensors to the encoder as follows:
 - **ForceTrac** to channel **D**;
 - **Foot pedal** to channel **C**.
 - **MyoScan/MyoScan-Z** to channel **A** and **B** (optional).

A	B	C	D	E	F	G	H	I	J
Myo (optional)	Myo (optional)	Pedal	FTrac						

- Turn the encoder on.
 1. Click **Quick Start**.
 2. From **Categories**, select **Pro/Flex Rehab Suite - Manual Muscle Testing**.
 3. From **Clients**, select a name.

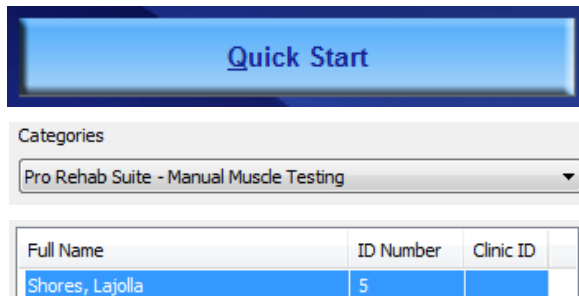

4. From **Favorites**, select the desired **Muscle Testing Protocol**.
5. Click **OK**.


Favorite	
Description	
MMT01 - Manual Muscle Testing Training	
MMT02 - Manual Muscle Testing Training with EMG	
MMT03 - Unilateral Muscle Testing	
MMT04 - Bilateral Muscle Testing	
MMT05 - Unilateral Muscle Testing with EMG	
MMT06 - Bilateral Muscle Testing with EMG	


Muscle Testing Training (MMT01 & MMT02)

These screens were designed for training purposes and for becoming familiar with the procedure and the instrumentation. They allow the recording of video, so the session can be replayed.


- If you use EMG sensors (MMT02), position them on the muscles of interest.

Note: Make sure your ForceTrac is properly calibrated before starting.

- Click the **Start** button .
- Ask the examinee to complete the range of motion and stabilize their position.
- Position the ForceTrac on the examinee.
- When ready, press the pedal and perform the test at full load.
- Resistance should always be applied gradually (not suddenly) and in the direction of the motion (“line of pull” of the muscle). The break usually happens within 2 or 3 seconds. A shorter break time may lead to re-injury. A longer break time may lead to fatigue.
- The trace of the applied force should appear after few seconds.
- You can repeat the exercise.

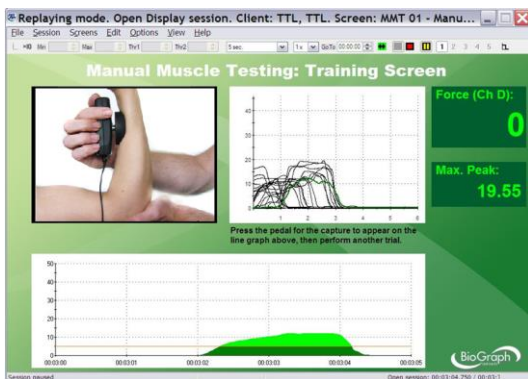

Training Screen without SEMG

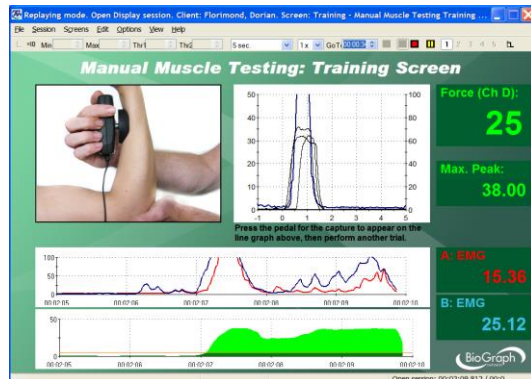


Training Screen with SEMG


Unilateral/Bilateral Muscle Testing (MMT03 & MMT04)

Note: Make sure your ForceTrac is properly calibrated before starting.

- Click the **Start** button  and read the instructions on the screen. **Do not press the foot pedal yet.**

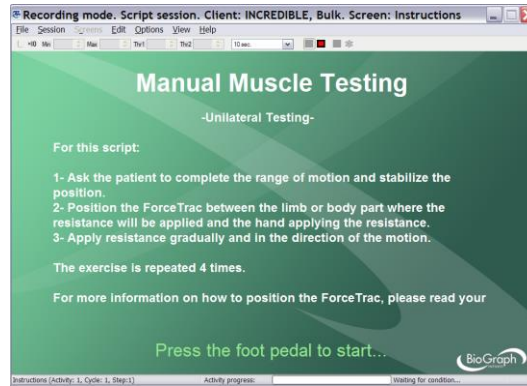

- Position yourself next to the examinee, with the encoder attached to your belt, the pedal near your foot and the ForceTrac in the palm of your hand.
- Press the foot pedal to go to the next screen.
- Position and stabilize the examinee in neutral position.
- Ask the examinee to complete the range of motion and stabilize their position.
- Position the ForceTrac on the examinee and perform a trial at 50% load (do not press the pedal).

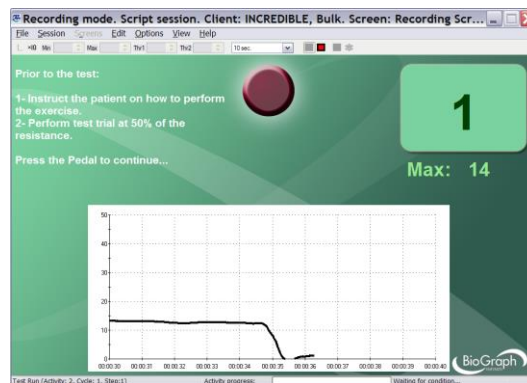


- Press the foot pedal to go to the next screen.


The examinee should be monitored closely and tests should be terminated immediately if there is any evidence of pain.

Measuring

1. Ask the examinee to complete the range of motion and stabilize their position.

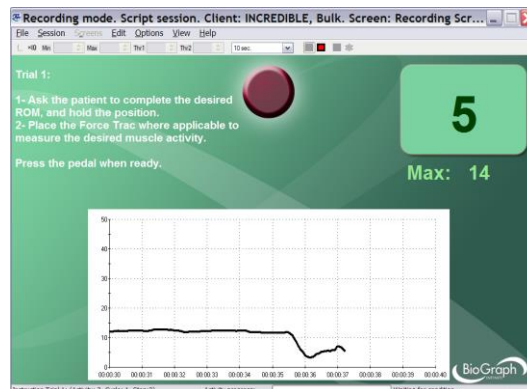

2. When ready, press the pedal and perform the test at full load.

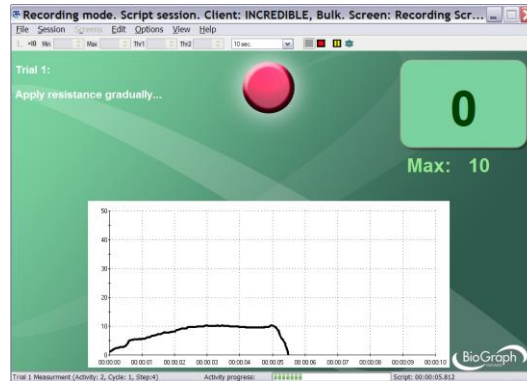


Resistance should always be applied gradually (not suddenly) and in the direction of the motion (“line of pull” of the muscle). The break usually happens within 2 or 3 seconds. A shorter break time may lead to re-injury. A longer break time may lead to fatigue.

- Repeat the exercise **4 times**.
- Then repeat it **4 times** for the opposite side, if you are performing bilateral testing.
- When the session ends, a series of message prompts guides you through the process of saving data, adding notes to the session file, and switching to reviewing mode so that you can generate a session report.

Unilateral/Bilateral Muscle Testing with EMG (MMT05 & MMT06)


The procedure is very similar to the one described above. The only difference is that the EMG sensors must be positioned on the muscles of interest and that EMG activity will be recorded.

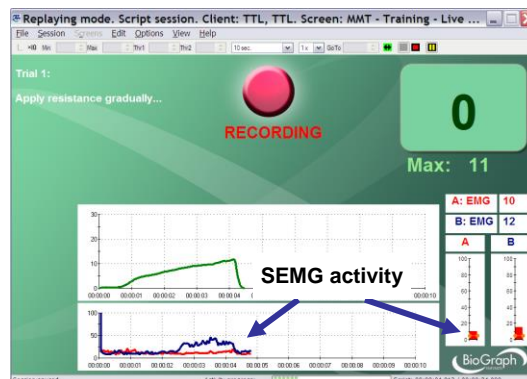

Potential substitute muscles being monitored with SEMG during the test.
A tone can be heard when the SEMG goes above a threshold.

Generating a report

- After the review screen opens, you can generate an MMT report. This is an Excel report.

1. Click the **Session Report** icon  in the tool bar.
2. Select **Generate Excel Report**.

Microsoft Excel opens and displays your report as a workbook. You can print or save the report using Excel's **Print** and **Save** functions.

Manual Muscle Testing

Unilateral Testing

Client Bulk INCREDIBLE
Session Date: 15/08/2008
Session Time: 3:18:49 PM

MUSCLE TEST: <Type it here or select>

Trial	Result (Max)	Used
Trial 1	47.254	<input type="checkbox"/>
Trial 2	51.998	<input type="checkbox"/>
Trial 3	55.043	<input type="checkbox"/>
Trial 4	54.07	<input type="checkbox"/>

Mean	SD
52.09	3.47

Trials

Final Result

Summary Notes GRADE: <select>

Follow-up Notes

Name and Signature

Unilateral Testing

Manual Muscle Testing

Bilateral Testing

Client Bulk INCREDIBLE
Session Date: 15/08/2008
Session Time: 4:35:53 PM

MUSCLE TEST: Scapular Abduction and Upward Rotation INVOLVED: <Select>

Trials	Uninvolved Side		Involved Side	
	Result (Max)	Used	Result (Max)	Used
Trial 1	30.329	<input type="checkbox"/>	-0.227	<input type="checkbox"/>
Trial 2	32.815	<input type="checkbox"/>	19.743	<input type="checkbox"/>
Trial 3	35.777	<input type="checkbox"/>	26.372	<input type="checkbox"/>
Trial 4	35.218	<input type="checkbox"/>	26.022	<input type="checkbox"/>

Mean	Uninvolved	Involved
33.53	Invalid	Invalid

SD	Uninvolved	Involved
2.49	Invalid	Invalid

% Deficit: Invalid

Trials

Final Result

Summary Notes GRADE: <Select>

Follow-up Notes

Name and Signature

Bilateral Testing

Validity Check of the Trials

The important data is the **peak load** before the break (maximum force). Trials are selected when if there is less than 15% between two consecutive trials and if the overall variation (coefficient of variation) between the selected trials is also less than 15%. The test requires at least 3 selected trials to be valid.

Only the selected trials are plotted on the graph.

Trial	Result (Max)	Used
Trial 1	47.254	<input checked="" type="checkbox"/>
Trial 2	51.998	<input checked="" type="checkbox"/>
Trial 3	55.043	<input checked="" type="checkbox"/>
Trial 4	54.07	<input checked="" type="checkbox"/>

Trials


Unilateral testing with all trials selected.

Trials	Uninvolved Side		Involved Side	
	Result (Max)	Used	Result (Max)	Used
Trial 1	30.329	<input checked="" type="checkbox"/>	-0.227	<input type="checkbox"/>
Trial 2	32.815	<input checked="" type="checkbox"/>	19.743	<input checked="" type="checkbox"/>
Trial 3	35.777	<input checked="" type="checkbox"/>	26.372	<input checked="" type="checkbox"/>
Trial 4	35.218	<input checked="" type="checkbox"/>	26.022	<input checked="" type="checkbox"/>

Trials

Bilateral testing with only 3 trials selected for the involved side. Trial 1 is discarded.

Final Result

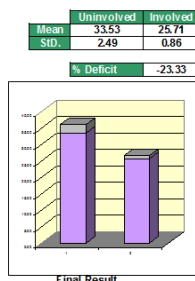

The final result computes the mean and standard deviation of the selected trials.

For the bilateral test, the percentage of deficit is also computed. This is the difference in percent between the final result of the uninvolved side (reference) and the involved side.

For the protocols with SEMG, the EMG peak intensity as well as the EMG-to-Force ratios are computed.

Uninvolved Side				Involved Side			
EMG (Side A)		EMG (Side B)		EMG (Side A)		EMG (Side B)	
EMG (Max)	EMG /F	EMG (Max)	EMG /F	EMG (Max)	EMG/F	EMG (Max)	EMG/F
140.411	1.051	55.85	1.057	39.366	0.753	107.305	0.88
55.418	0.889	130.164	0.854	42.201	0.723	49.438	0.976
94.885	0.909	239.211	1.137	93.068	1.186	41.895	0.951
47.728	0.812	64.256	0.969	65.424	0.673	119.005	1.511

User's fields

Muscle Test

You can select the muscle test type (different list for unilateral/bilateral test):

MUSCLE TEST:		<Type it here or select>
		<Type it here or select>
		Scapular Abduction and Upward Rotation
		Scapular Elevation
		Scapular Adduction
		Scapular Depression and Adduction
		Scapular Adduction and Downward Rotation
		Shoulder Flexion
		Shoulder Extension

Trial	Uninvolved Result (Max)
Trial 1	30.329
Trial 2	32.815
Trial 3	35.777

Or you can enter your own test: **MUSCLE TEST:** Cerv

Involved (bilateral testing only):

You can select the involved side (right/left):

INVOLVED:	<Select>
	<Select>
	LEFT
	RIGHT

Uninvolved	<Select>
	<Select>
	LEFT
	RIGHT

Grade

You can select the grade:

GRADE:	<Select>
	<Select>
	5
	4
	3

Summary and Follow-up notes

You can enter notes in the summary/follow-up notes boxes:

Summary Notes:
Marked improvement

- In **Excel 2000** and **Excel 2003**, place your cursor in the text box and start typing.
- In **Excel 2007**, place your cursor in the text box.
 - Right-click the mouse button.
 - Select **Edit text**.
 - Start typing.




Algometry

OVERVIEW

Algometry measures Pressure Threshold (PTM). It quantifies the sensitivity of para-spinal tissues and their tenderness.

CLINICAL GUIDE REFERENCE

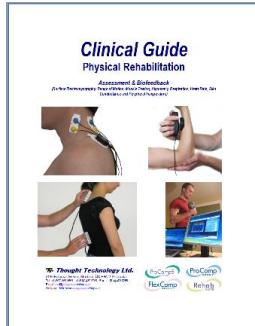

The clinical guide gives general guidelines and recommendations for the accurate use of the instrument. It also explains how to prepare the examinee and the procedure to follow.

Chapter: ALGOMETRY


PROCEDURE

Starting the Session

- Connect the sensors to the encoder as follows:
 - **ForceTrac** with rod attachment to channel **D**;
 - **Foot pedal** to channel **C**.


A	B	C	D	E	F	G	H	I	J
		Pedal	FTrac						


- Turn the encoder on.
 1. Click **Quick Start**.
 2. From **Categories**, select **Pro/Flex Rehab Suite - Algometry**.
 3. From **Clients**, select a name.
 4. From **Favorites**, select the Algometry Protocol.
 5. Click **OK**.

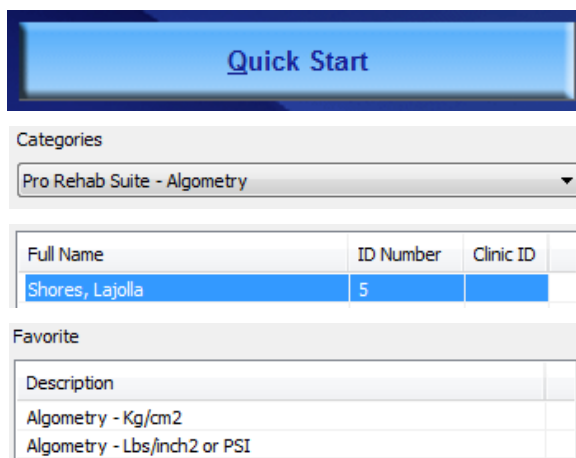

Note: The force unit (kgs/cm² or lbs/in²) depends on whether you have selected Pounds or Kilogram units in the **General** tab of **Preferences** (accessed from the Main Menu screen).

Note: Make sure your ForceTrac is properly calibrated before starting.

- Click the **Start** button  and read the instructions on the screen. **Do not press the foot pedal yet.**

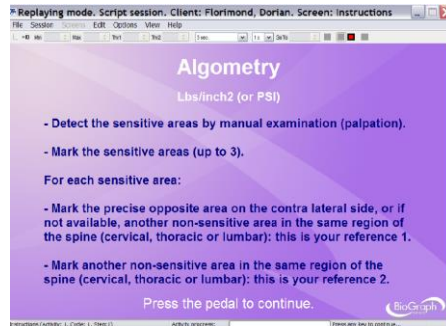

- Press the foot pedal to go to the next screen.
- Hold the ForceTrac in your hand.


Force Unit Verification screen

Before you can proceed, you must verify the Force units.

If you use imperial units, press the ForceTrac algometer against a hard surface until it reaches at least 100 Lbs/inch² (PSI). If you are unable to reach this value, it means the Force unit is not properly set.

Note: 1 Lbs/inch² = 1 PSI.

- If you can't reach Lbs/inch² (PSI), change the force unit to Pounds.
 - To do this, first end the session and return to the Main Menu screen of BioGraph Ininiti.
 - Open the **Tools** menu, click on **Preferences** and in the **General** tab, select **Pounds**.
 - Click **OK** and then start a new session.

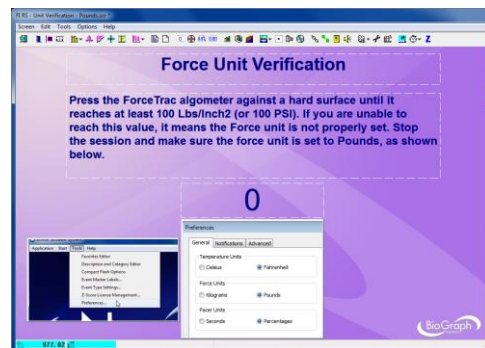

If you use metric units, press the ForceTrac algometer against a hard surface. If you are able to reach at least 14 Kg/cm², it means the Force unit is not properly set, and the red light will turn on.

- If the red light turns on, change the force unit to Kilograms.
 - To do this, first end the session and return to the Main Menu screen of BioGraph Ininiti.
 - Open the **Tools** menu, click on **Preferences** and in the **General** tab, select **Kilograms**.
 - Click **OK** and then start a new session.
- If the red light does NOT turn on, press the pedal to continue.



Note: Pressing the pedal allows you to go to the next screen only if the maximum applied pressure is greater than 3 Kg/cm² and less than 14 Kg/cm².



- Position yourself next to the examinee, with the encoder attached to your belt, the pedal near your foot and the ForceTrac in the palm of your hand.



Measuring

- Repeat the following sequence three times, once for each pressure threshold measure:

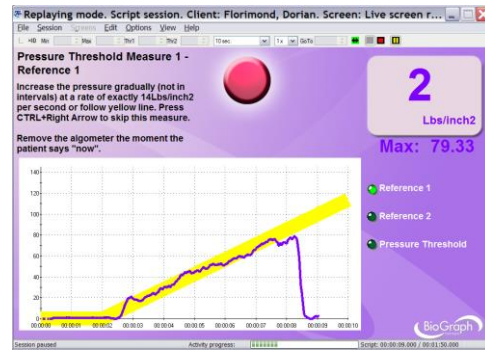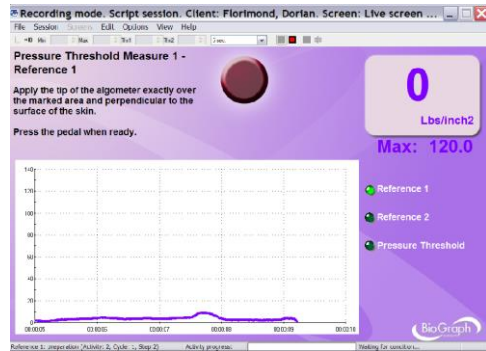



- Measure of reference 1:** Apply the tip of the algometer over the marked area and perpendicular to the surface of the skin. When ready to measure, press the foot pedal.

- Increase the pressure gradually at a rate of exactly 14Lbs/inch² or 10Kg/cm² or follow the yellow line. Remove the algometer at the moment the patient says "now".

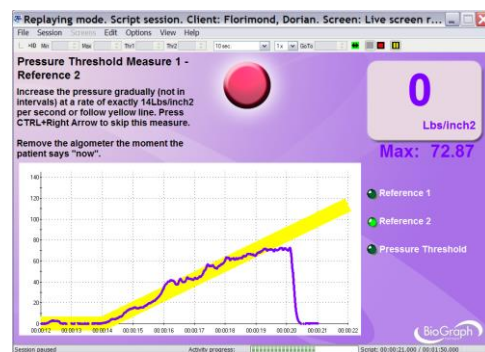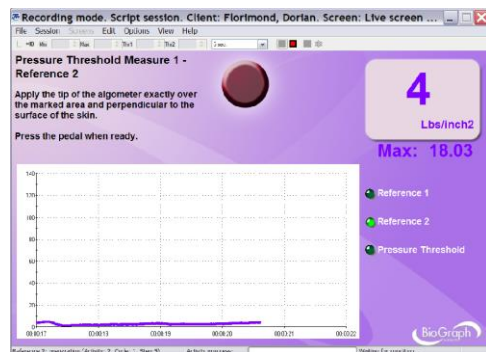

- Measure of reference 2:** Apply the tip of the algometer over the marked area and perpendicular to the surface of the skin. When ready to measure, press the foot pedal.

- Increase the pressure gradually at a rate of exactly 14Lbs/inch² or 10Kg/cm² or follow the yellow line. Remove the algometer at the moment the patient says "now".

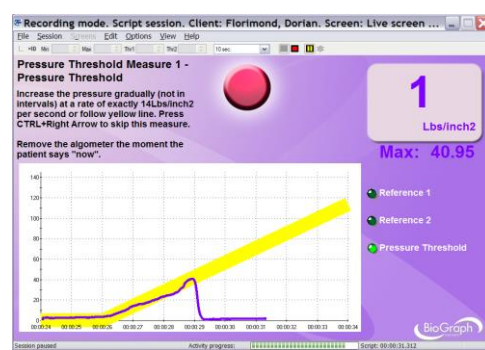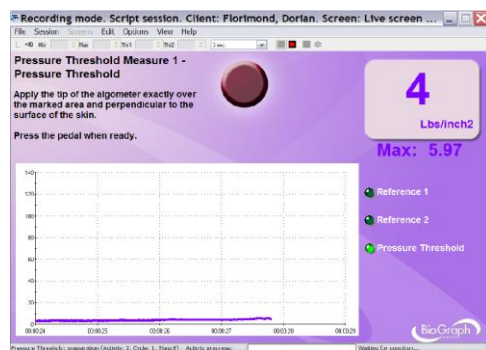


- Measure of pressure threshold:** Apply the tip of the algometer over the marked area and perpendicular to the surface of the skin. When ready to measure, press the foot pedal.

- Increase the pressure gradually at a rate of exactly 14Lbs/inch² or 10Kg/cm² or follow the yellow line. Remove the algometer at the moment the patient says "now".



- When the session ends, a series of message prompts guides you through the process of saving data, adding notes to the session file, and switching to reviewing mode so that you can generate a session report.

Generating a report

- After the review screen opens, you can generate an Algometry report. This is an Excel report.

- Click the **Session Report** icon  in the tool bar.
- Select **Generate Excel Report**.

Microsoft Excel opens and displays your report as a workbook. You can print or save the report using Excel's **Print** and **Save** functions.

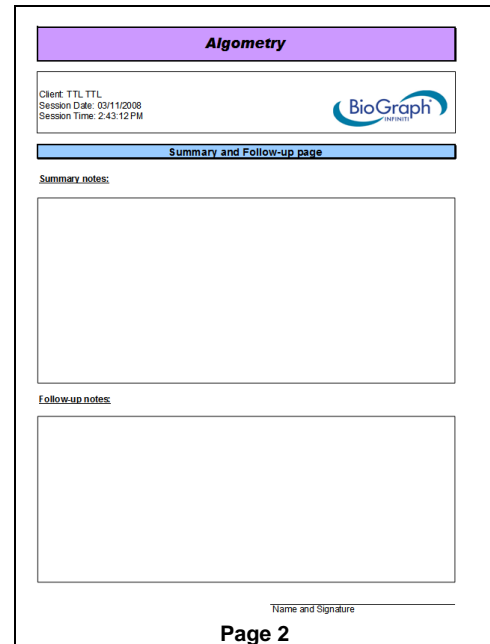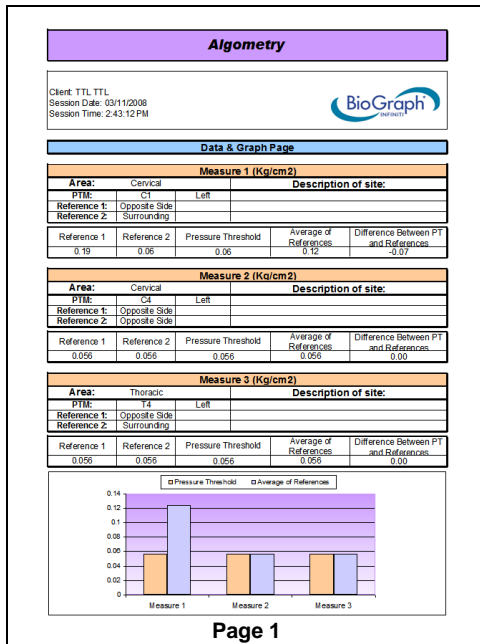

Selecting the sites

Area:	Not used
PTM:	<select>
Reference 1:	Not used
Reference 2:	Cervical
	Thoracic
	Lumbar
	Sacral
Reference 1	Reference 2

For each measure, first select the area (cervical, thoracic, lumbar or sacral). If the measure is not used, select **Not used**.

Area:	Cervical
PTM:	<select>
Reference 1:	C1
Reference 2:	C2
	C3
	C4
Reference 1	C5
	C6
	C7
79.33	

Then select the PTM site...

PTM:	C3	<select>
ference 1:	<select>	<select>
ference 2:	<select>	Left
		Center
		Right

... and the side.

PTM:	C3
Reference 1:	<select>
Reference 2:	<select> Opposite Side Surrounding
Reference 1:	Opposite Side
Reference 2:	<select>
Reference 1:	<select> Opposite Side Surrounding

Then select the reference sites (opposite side or surrounding muscle).

You can also add a description of the location of the site.

Area:	Cervical	Description of site:	
PTM:	C3	Left	one inch from the spine
Reference 1:	Opposite Side		

Data table

Reference 1	Reference 2	Pressure Threshold	Average of References	Difference Between PT and References
79.33	72.87	64.04	76.10	-12.06

The data table shows the three measures, the average of the references and the difference between the references and the pressure threshold.

According to the literature, the following conditions are considered a positive finding:

- The difference between the PTM and the opposite side or surrounding muscle, in the same patient, exceeds 2 kg/cm² (28.5 Lbs/inch²).
- The PTM is less than 3 kg/cm² (42.7 Lbs/inch²).

Summary and Follow-up notes

You can enter notes in the summary/follow-up notes boxes:

Summary Notes:
Marked improvement

- In **Excel 2000** and **Excel 2003**, place your cursor in the text box and start typing.
- In **Excel 2007**, place your cursor in the text box.
 1. Right-click the mouse button.
 2. Select **Edit text**.
 3. Start typing.



Hardware Specifications

MyoScan/MyoScan-Z EMG Sensors (SA9503M/SA9503Z)



Size (approx.)	37mm x 37mm x 12mm (1.45" x 1.45" x 0.45")
Weight	15g (0.5 oz)
Input impedance	$\geq 10G\Omega$ in parallel with 10pF
Input range	0 – 2000 μ VRMS
Sensitivity	<0.1 μ VRMS
CMRR	>130dB
Channel bandwidth	10Hz – 1kHz
Signal output range	0 – 1.0VRMS
Input / output gain	500
Supply voltage	7.26V (\pm 0.02V)
Current consumption	0.7mA (\pm 0.25mA)
Accuracy	$\pm 0.3\mu$ VRMS $\pm 4\%$ of reading @25°C to 30°C

MyoScan-Pro EMG Sensor (SA9401M-60 or SA9401M-50)



Size (Approx.)	37mm x 37mm x 15mm (1.45" x 1.45" x 0.60")
Weight	25g (1 oz)
Input impedance	10G Ω in parallel with 10pF
Input range	0 – 400 μ VRMS, 0 – 1600 μ VRMS
Sensitivity	<0.1 μ VRMS
Bandwidth	20Hz – 500Hz
Accuracy	$\pm 5\%$, $\pm 0.3\mu$ VRMS

EKG Sensor (SA9306M)



Length (approx.)	152cm (60")
Weight	10g (0.33oz)
Temperature range	10°C - 45°C (50°F – 115°F)
Accuracy	$\pm 1.0^\circ\text{C}$ ($\pm 1.8^\circ\text{F}$) 20°C – 40°C (68°F – 104°F)

HR/BVP Flex/Pro Sensor (SA9308M)



Length (approx.)	20mm x 34mm x 10mm (0.72" x 1.33" x 0.41")
Weight	20g (0.66 oz)
Input range	Unit less quantity displayed as 0% – 100%
Accuracy	$\pm 5\%$

Respiration Sensor (SA9311M)



Size of sensor (approx.)	33mm x 33mm x 11mm (1.28" x 1.28" x 0.44")
Weight (approx.)	90g (3.3 oz)
Input range	0 – 9cm (0 – 3.55")
Supply voltage	7.26V ($\pm 0.05V$)
Current consumption	$\leq 0.5mA$
Straps:	
Length, sensor strap (approx.)	18cm (7")
Length, elastic strap (approx.)	14cm (5.5")
Max. length, small extender strap (approx.)	51cm (20")
Max. length, medium extender strap (approx.)	84cm (33")

Skin Conductance Flex/Pro Sensor (SA9309M)



Size without electrode leads (approx.)	3.5 cm (1.4")
Size with electrode leads (approx.)	15 cm (6.0")
Cable length (approx.)	127 cm (50")
Weight (approx.)	25 g (1 oz)
Signal input range	0 – 30.0 μS
Accuracy	$\pm 5\%$ and $\pm 0.2 \mu S$

Skin Temperature Sensor (SA9310M)



Length (approx.)	152cm (60")
Weight	10g (0.33oz)
Temperature range	10°C - 45°C (50°F – 115°F)
Accuracy	$\pm 1.0^{\circ}C$ ($\pm 1.8^{\circ}F$) 20°C – 40°C (68°F – 104°F)

InclinoTrac/Dual-InclinoTrac (SA7650/ SA7655)



Dimensions	32mm x 18mm x 71mm
Weight	26g
Range	$\pm 180^{\circ}$
Accuracy	$\leq 1.0^{\circ}$ (standalone mode) $\leq 2.0^{\circ}$ (dual mode, angle difference)
Output gain	4.44mV / degree inclination
Output voltage span	2.200 $\pm 0.8V$
Power supply	7.26V
Current consumption, maximum	9.5 mA (standalone mode) 19.0 mA (dual mode)
Link cable	RJ-11, 2 pairs, reversed (this is not a standard telephone cable)

ForceTrac (SA7600)



Dimensions	93mm x 63mm x 25mm
Weight	94g
Input range (force)	0–100 lbf
Safe overload	250 lbf
Accuracy	± (0.1lbf + 5% of reading)
Zero-level output	2.048V
Full scale output swing	1V at 100 lbs load
Power supply	7.26V
Algometer Attachment:	
Dimensions	69mm x 11mm (diameter)
Weight	14g
Material	Aluminum
Flat Tester Attachment:	
Dimensions	9mm x 42mm (diameter)
Weight	39g
Material	Neoprene
Curved Tester Attachment:	
Dimensions	19mm x 42mm (diameter)
Weight	52g
Material	Neoprene

Placing Orders

Outside USA

Tel: 1-514-489-8251

Fax: 1-514-489-8255

In USA Toll-Free

Tel: 1-800-361-3651

E-Mail: mail@thoughttechnology.com

Or contact your local authorized distributor.

Technical Support

Outside USA

Tel: 1-514-489-8251

Fax: 1-514-489-8255

In USA Toll-Free

Tel: 1-800-361-3651

E-Mail: techsupport@thoughttechnology.com

Or contact your local authorized distributor.

Warranty

The hardware (encoder and sensors) is guaranteed to be free from defects in material and workmanship for 1 year from the date of purchase.

In the unlikely event that repair is necessary, contact Thought Technology Ltd. to receive a Return Authorization number. Then send the unit back by a traceable method. Thought Technology will not be responsible for items not received. We will repair or replace your unit(s) that are still under warranty free of charge.

This warranty does not apply to damage incurred through accident, alteration, or abuse.

This warranty does not cover damage to the Ininiti encoder or the sensors caused by obvious mechanical mistreatment of the system.

Returning Equipment for Repair

Before returning the equipment, please contact first our service department and get an authorization number (RA number).



Canada and International +1 514 489-8251



USA 1-800-361- 3651



service@thoughttechnology.com

Then fill-in the return form (the form can be found at the end of the manual). You must provide a detailed description of the problem you are experiencing, and your telephone/fax number and e-mail.

The unit(s) must be sent **postage prepaid** and **insured**, with proof of purchase to one of the addresses below.

All customs and duties charges will be billed to the customer if incurred by sending the unit to the **wrong** address.

In the USA, ship insured to:

Thought Technology Ltd.
Cimetra LLC
8396 State Route 9
West Chazy, New York
12992 USA

In Canada, ship insured to:

Thought Technology Ltd.
8205 Montreal/ Toronto Blvd. Suite 223
Montreal West, Quebec
Canada H4X 1N1

For international:

- Package must be marked "Broker: Livingston International – 133461"
- Ship insured to:

Thought Technology Ltd.
8205 Montreal/ Toronto Blvd. Suite 223
Montreal West, Quebec
Canada H4X 1N1

Repair Return Form

Be sure to call for authorization before returning any equipment!

Copy and complete this form and include it with the unit(s).

Include a copy of original invoice and return to the address in the Returning Equipment section.

Name

Company

Address

Phone No.

Fax No.

Date Purchased

From Whom

Model Name

Serial No.

Problem

